


Squiggles Stander Mobile Chassis - Suspension

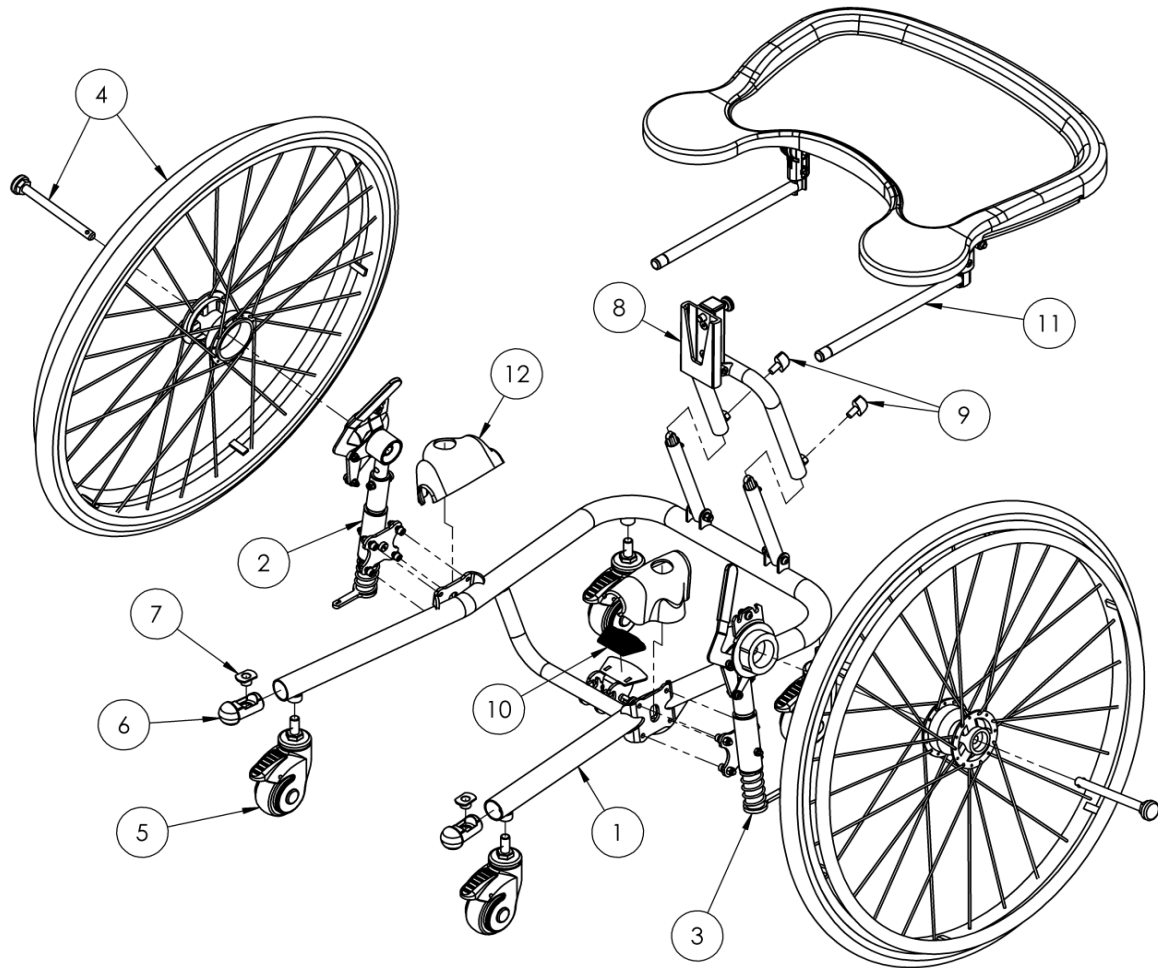

Pos.	Leckey Code	Description	Qty
1	129-600	Squiggles Mobile Chassis	1
2	129-631	Suspension Arm LHS Assembly	1
3	129-632	Suspension Arm RHS Assembly	1
4	129-647	Wheel 24" Pair	1
4	129-646	Wheel 26" Pair	1
5	IA34	Castor	4
6	118-160	Castor Tube Insert	2
7	SJ136	Jetpress Nut	1
8	129-625	Angle Tube Outer Assembly	1
9	SF19	Locking Key (M8x16)	1
10	102-1232	Rubber Pad	1
11	118-619	Tray	1
12	129-062	Moulded Shock Cover	2

