


## Contoured Advanced Seat - Hi Low Frame Power

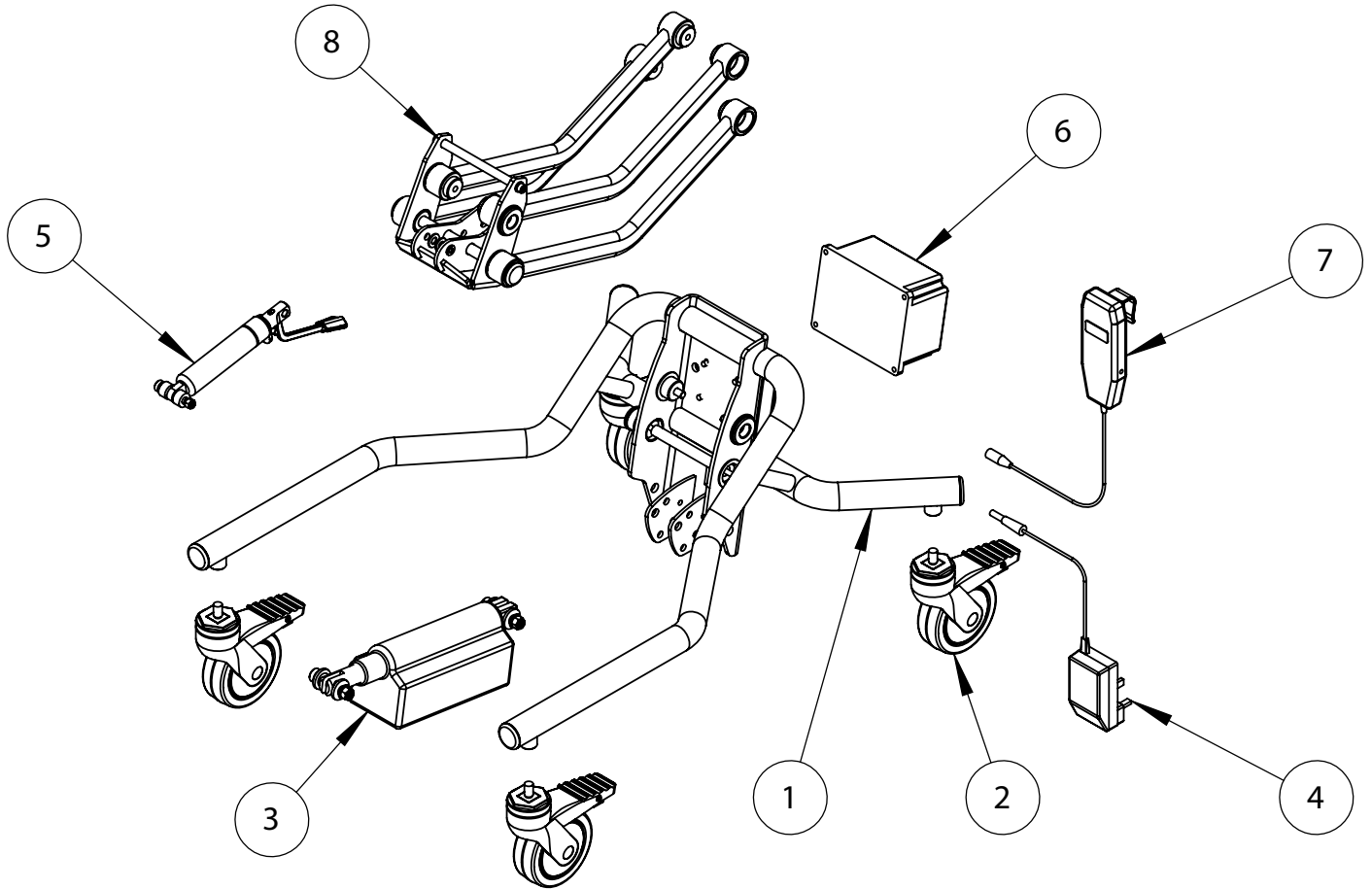

Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	102-1693	Chassis Welded Assy Size 1&2	1	✓	✓	
1	102-2693	Chassis Welded Assy Size 3	1			✓
2	IA13	Castor	4	✓	✓	✓
3	102-1694	Actuator Assy	1	✓	✓	✓
4	SI74	Charger UK	1	✓	✓	✓
4	SI125	Charger EUR	1	✓	✓	✓
4	SI126	Charger USA/JAPAN	1	✓	✓	✓
4	SI127	Charger AUS	1	✓	✓	✓
5	102-1695	500N Blocolift Size 1&2 Assy	1	✓	✓	
5	102-1696	800N Blocolift Size 3 Assy	1			✓
6	SI64	Battery	1	✓	✓	✓
7	SI67	Handset	1	✓	✓	✓
8	117-651	Lever Arm Assembly Size 1-3	1	✓	✓	✓