


Comfee Armrests

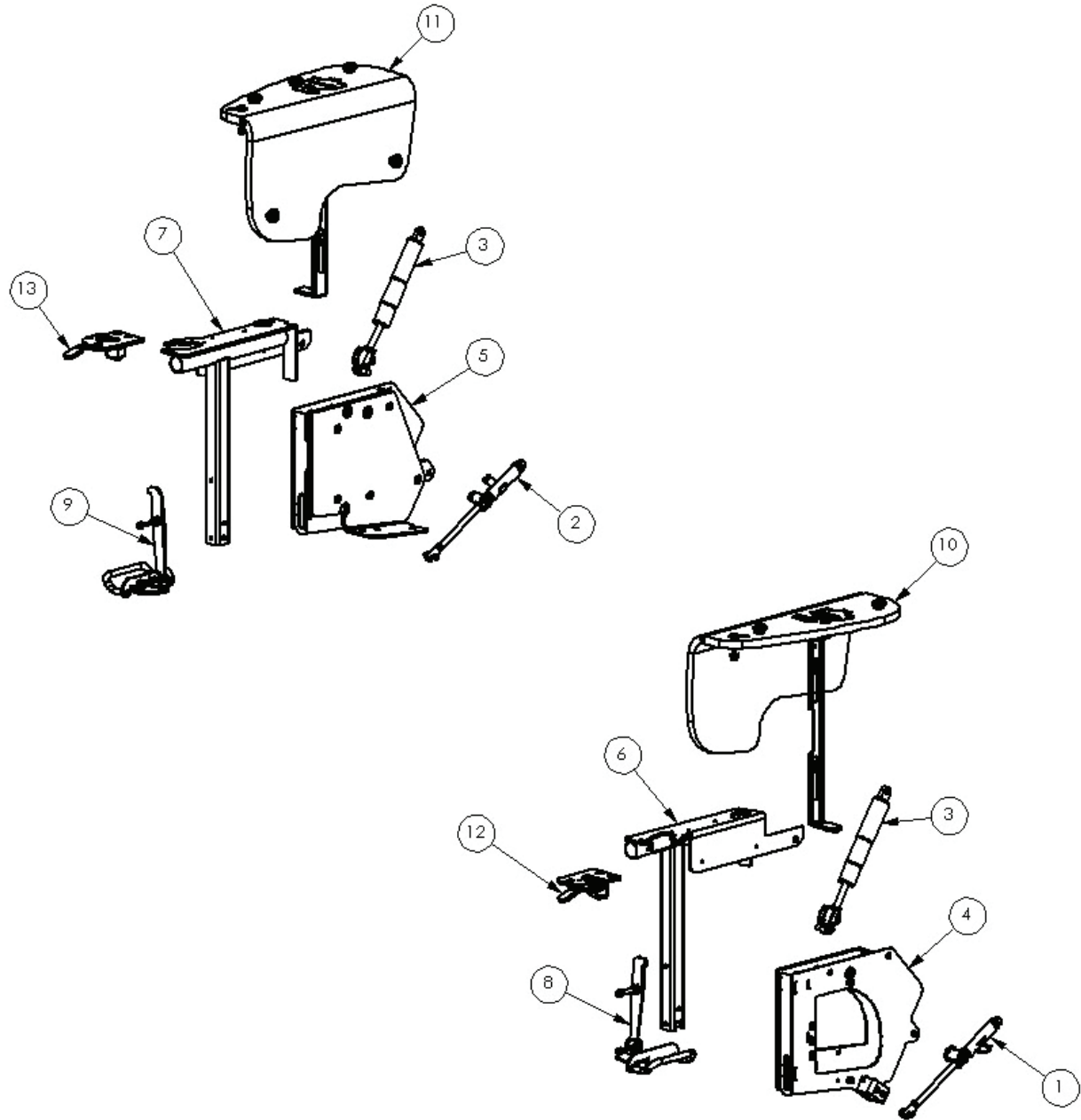

Comfee Armrests

| Pos. | Leckey Code | Description | Qty | Compatible with | | |
|------|-------------|------------------------|-----|-----------------|--------|--------|
| | | | | Size 1 | Size 2 | Size 3 |
| 1 | 149-1980 | Mechlock Assy LHS | 1 | ✓ | ✓ | ✓ |
| 2 | 149-1979 | Mechlock Assy RHS | 1 | ✓ | ✓ | ✓ |
| 3 | 149-1981 | Gas Spring Assy | 2 | ✓ | ✓ | ✓ |
| 4 | 149-1946 | Swinging Box LHS | 1 | ✓ | | |
| 4 | 149-2946 | Swinging Box LHS | 1 | | ✓ | |
| 4 | 149-3946 | Swinging Box LHS | 1 | ✓ | | ✓ |
| 5 | 149-1944 | Swinging Box RHS | 1 | ✓ | | |
| 5 | 149-2944 | Swinging Box RHS | 1 | | ✓ | |
| 5 | 149-3944 | Swinging Box RHS | 1 | ✓ | | |
| 6 | 149-1607 | Armrest Inner Assy LHS | 1 | ✓ | | |
| 6 | 149-2607 | Armrest Inner Assy LHS | 1 | | ✓ | |
| 6 | 149-3607 | Armrest Inner Assy LHS | 1 | ✓ | | |
| 7 | 149-1608 | Armrest Inner Assy RHS | 1 | ✓ | | |
| 7 | 149-2608 | Armrest Inner Assy RHS | 1 | | ✓ | |
| 7 | 149-3608 | Armrest Inner Assy RHS | 1 | ✓ | | |
| 8 | 149-1945 | Height Latch Assy LHS | 1 | ✓ | | |
| 8 | 149-2945 | Height Latch Assy LHS | 1 | | ✓ | |
| 8 | 149-3945 | Height Latch Assy LHS | 1 | ✓ | | |
| 9 | 149-1943 | Height Latch Assy RHS | 1 | ✓ | | |
| 9 | 149-2943 | Height Latch Assy RHS | 1 | | ✓ | |
| 9 | 149-3943 | Height Latch Assy RHS | 1 | ✓ | | |
| 10 | 149-1985 | Armrest Assy LHS | 1 | ✓ | | |
| 10 | 149-2985 | Armrest Assy LHS | 1 | | ✓ | |
| 10 | 149-3985 | Armrest Assy LHS | 1 | ✓ | | |
| 11 | 149-1983 | Armrest Assy RHS | 1 | ✓ | | |
| 11 | 149-2983 | Armrest Assy RHS | 1 | | ✓ | |
| 11 | 149-3983 | Armrest Assy RHS | 1 | ✓ | | |
| 12 | 149-1988 | Tray Locking Assy LHS | 1 | ✓ | ✓ | ✓ |
| 13 | 149-1987 | Tray Locking Assy LHS | 1 | ✓ | ✓ | ✓ |