

LECKEY[®]


Leckey Tour Mobility Base Spare Parts Catalogue



Contents

03-04 Leckey Tour Mobility Base

Leckey Tour Mobility Base

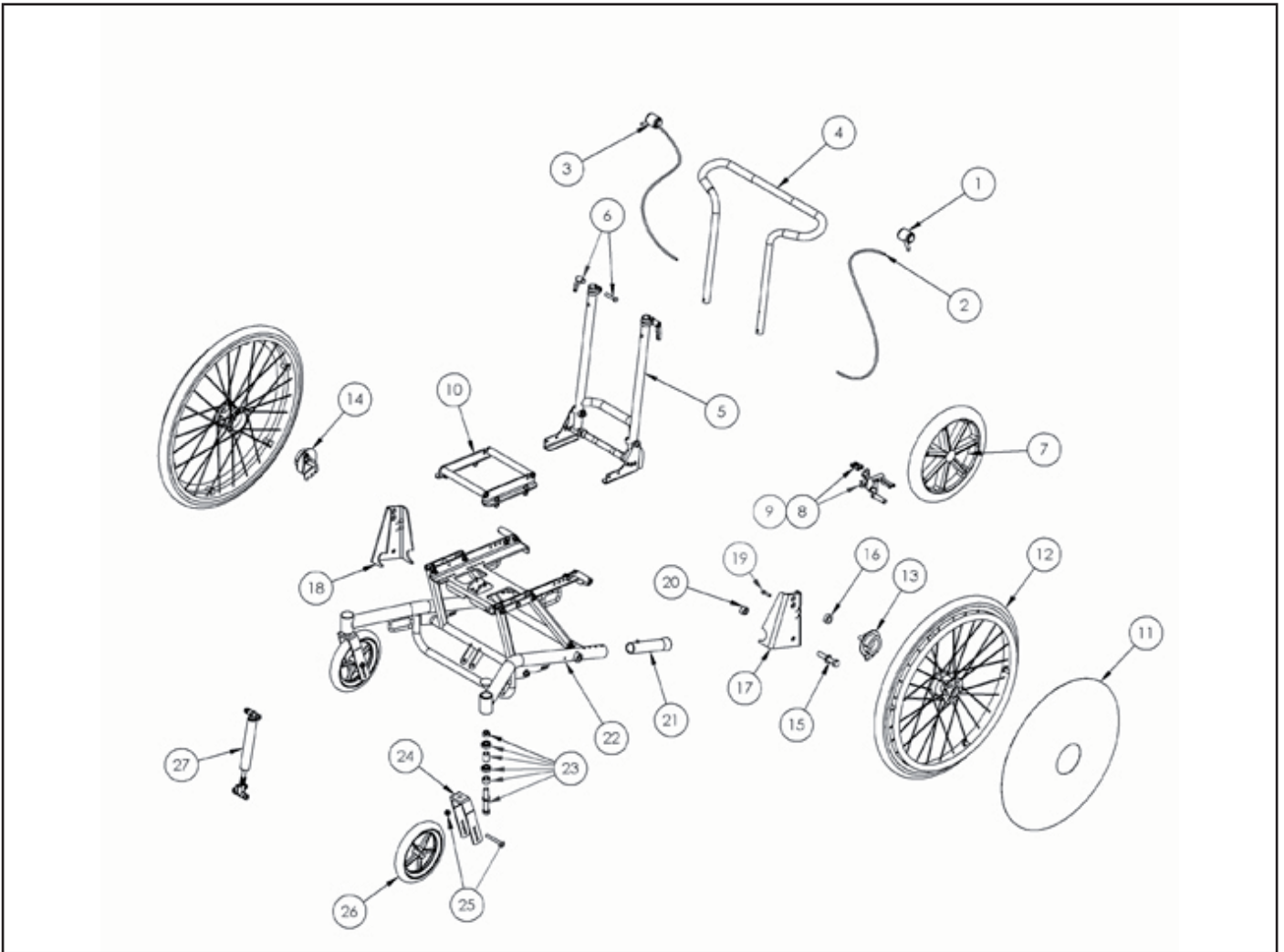

Pos.	Leckey Code	Description	Qty
1	133-851	Tilt Lever	1
2	133-856	Tilt Cable	1
3	133-2130	Brake Assembly	1
4	133-2131	Upper Handlebar Assembly	1
5	133-2132	Lower Pushbar Assembly	1
6	133-2092	Pushbar Clamp	1
7	133-2087	Rear 12" Wheel Assy	2
8	133-2085	Brake Assy LH	1
9	133-2086	Brake Assy RH	1
10	133-2133	Interface KIT	1
10	133-2134	Interface Mygo	1
11	IA54	Spoke Shield 22"	2
11	IA55	Spoke Shield 24"	2
12	IA52	Wheel 22"	2
12	IA53	Wheel 24"	2

Leckey Tour Mobility Base

Pos.	Leckey Code	Description	Qty
13	IA50	Braked Hub LH	1
14	IA51	Braked Hub RH	1
15	133-2134	Bolt Assembly	2
16	133-427	Hub Spacer	2
17	133-498	Wheel Plate LH KIT	1
17	137-498	Wheel Plate LH Mygo	1
18	133-426	Wheel Plate RH KIT	1
18	137-426	Wheel Plate RH Mygo	1
19	133-2136	Hub Screw Assembly	2
20	133-387	Plate Spacer	2
21	133-778	Tipper	2
22	133-2137	Chassis Assembly	1
23	133-2138	Bearing Assembly	2
24	IA48	Fork	2
25	133-2140	Wheel Bolt Assembly	2
26	IA45	Front Castor 4"/100mm	2
26	IA46	Front Castor 6"/150mm	2
26	IA47	Front Castor 8"/200mm	2
27	133-2144	Tilt Shock Assembly	1