

LECKEY[®] 
Squiggles+ 



Introducing the NEW
Squiggles+ Stander
from Leckey



Standing is very important

An upright posture allows gravity to assist the function of our internal systems. By standing, children bear their weight through their bones and joints, which helps shape them while increasing their strength.

-
- Evidence would suggest that standing has a beneficial effect on hip joint formation.
-
- Standing stretches tight muscles and provides an alternative position.
-
- Socially, standing is also important as it enables interaction with their peers at the same level.



The new Squiggles+ Stander offers new, additional features to the extremely versatile three-in-one Squiggles Stander for children aged 1-5 years.

Still offering:

- Prone, upright and supine standing in one product.
- Wide range of adjustability by the chest, hip, and knee supports; and the head support in supine provides clinicians with the tools to position a large range of children in the same product.
- Colourful design, available in four colours with machine washable covers.
- Lightweight and robust stander support frame which can be easily transported or disassembled for storage.



With new, added features



- 1 Hip abduction of up to 60° in prone, upright and supine standing.
- 2 Fully horizontal loading in supine.
- 3 Height-adjustable, foldable push handle for ergonomic use.
- 4 New wrap around knee supports to better accommodate all sizes, with added comfort and support.
- 5 Individually adjustable footplates to accommodate leg length discrepancy.
- 6 New, improved, pivot chassis.



- The height, depth, width and angle of the cushioned pelvic positioning harness can be changed to support the child comfortably in a secure position.
- The height, depth, width and angle of the chest positioning support can also be adjusted.
- Cushioned sternum pad offers extra support which encourages extension and gives a wide range of freedom for the arms thus allowing a greater range of activities.
- Comfortable, padded headrest which can also be removed for prone standing.
- Larger, more comfortable medial knee supports to distribute pressure, especially in abducted standing. Improved wrap support to maintain knee in neutral flexion or extension.

- Hip Abduction now achievable, 60 degrees bilaterally. With a measurement gauge to monitor and aid repeatability. Hip laterals also shift in sync with the legs as they move through the arc of abduction.
- Individual footplates to accommodate leg length discrepancy.

- Improved pivot chassis facilitates horizontal loading, accommodates hip abduction, with new foot pedal to ease carer effort.
- New height-adjustable push handle, which can be lowered for prone use.
- Activity tray with improved height and adjustability which can be used in either prone or supine position.

Available in 3 new colours, with new Vinyl option also available



Red



Purple



Blue

Covers Include:

Support Harnesses (2),
Chest Cushion (1),
Hip Cushion (1),
Chest Lateral Cushions (Pair),
Knee Supports (Pair),
Pelvic Support Harness (1),
Spine Cap Cover (1).



Red Vinyl option

Find out more

For further information on the clinical benefits associated with Squiggles+ Stander, please visit our website.

www.leckey.com

The Squiggles Stander Plus System

The standard product includes:

Hip and chest support with sternum prompt; chest and hip lateral supports; knee supports; footplate; tray attachment and head support.

Squiggles+ Stander dimensions

Age (approx)	1-5
Max User Weight	22kg/48.4lbs
User Height	Min 750mm / 29.5 inches Max 1110mm / 43.5 inches
Chest Support	Angle adjustment +/- 30° Depth adjustment 50mm / 2 inches
Chest Support Height (footplate to armpit)	Min 550mm / 21.5 inches Max 820mm / 32 inches
Chest Width (distance between laterals)	Min 160mm / 6.3 inches Max 230mm / 9 inches
Hip Support (height from footplate to mid buttocks)	Min 300mm / 11.8 inches Max 530mm / 20.8 inches
Distance between midline of knees	Min 140mm / 5.5 inches Max 215mm / 8.4 inches
Footplate Angle	Plantarflexion 10° / Dorsiflexion 10°

Tray Size

Tray height – Prone (footplate to top of tray)	Min 555mm / 21.9 inches Max 800mm / 31.5 inches
Tray height – Supine (footplate to top of tray)	Min 480mm / 18.5 inches Max 800mm / 31.5 inches
Tray angle adjustment	Prone 30° / Supine 40°

Support frame and Pivot Chassis Dimensions

Stander support frame	Weight 7.5kg / 16.5lbs Width 390mm / 15.4 inches	Length 680mm / 26.8 inches Height 390mm / 15.4 inches
Chassis footprint (unfolded)	Weight 6.5kg / 14.3lbs Width 590mm / 23.2 inches	Length 950mm / 37.4 inches Height 515mm / 20.3 inches
Chassis footprint (folded)	Weight 6.5kg / 14.3lbs Width 590mm / 23.2 inches	Length 660mm / 26.0 inches Height 515mm / 20.3 inches
Chassis angle range	90 - 160°	90 - 70°

For further information on
the Squiggles+ Stander
check out our website
www.leckey.com

LS823-01

Leckey
19 Ballinderry Road
Lisburn
BT28 2SA

T: +44 (0)28 9260 0750
E: Info@leckey.com

