


## Rigid Lateral Extended Height Assembly

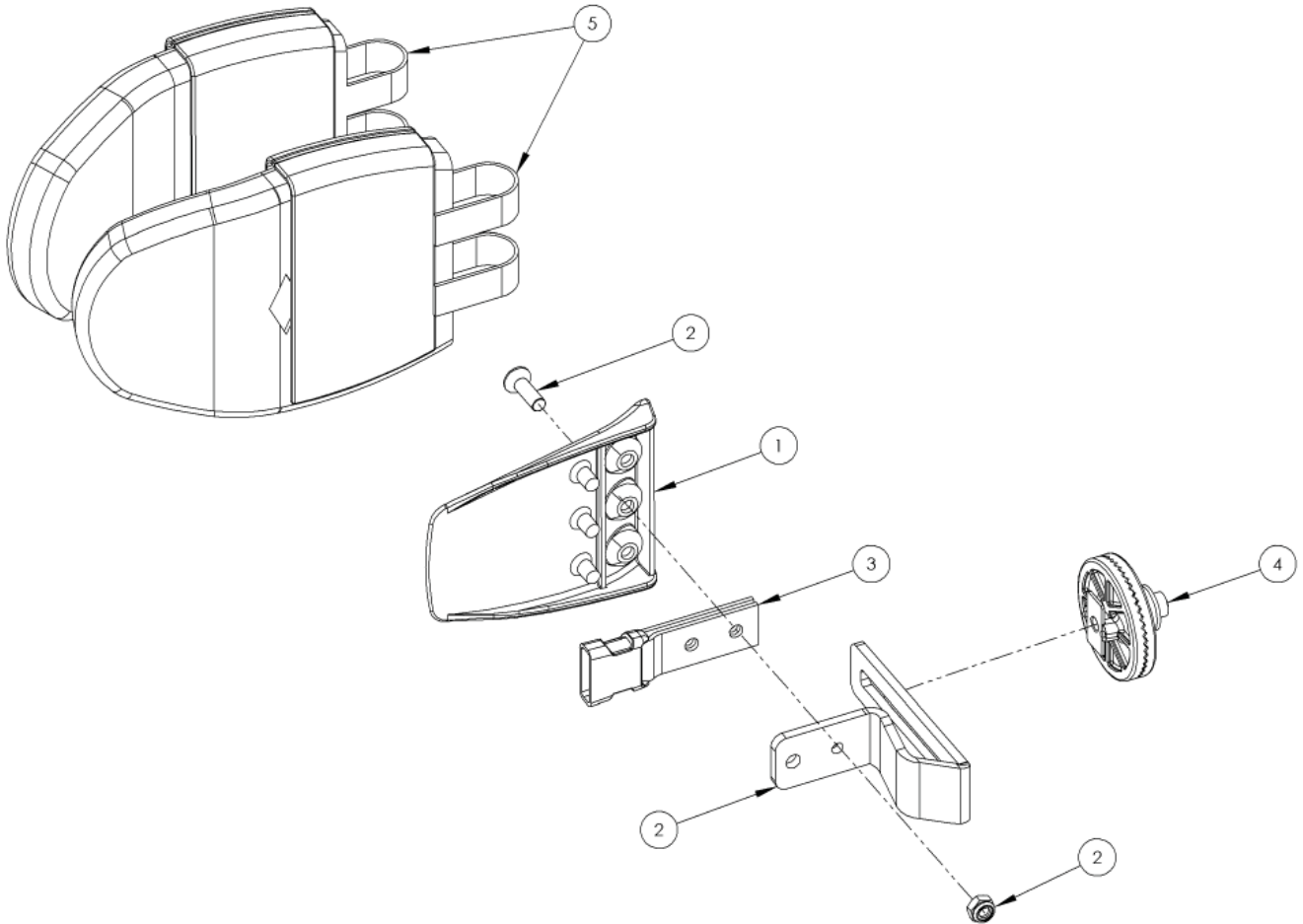

Pos.	Leckey Code	Description	Qty	Compatible with	
				Size 1	Size 2
1	117-176	Lateral	1	✓	✓
2	137-9018	Lateral Bracket & Screw Kit LH	1	✓	✓
2	137-9019	Lateral Bracket & Screw Kit RH	1	✓	✓
3	117-700	Harness Buckle Attachment	1	✓	✓
4	120-9027	Angle Lock Assembly	1	✓	✓
5	137-806-05	Lateral Covers Pair	1	✓	✓