



 **MyWay**

My independence.
My freedom.
My choice.
My movement.
MyWay!

LECKEY[®]

Leckey are passionate about designing products that give children the opportunity for inclusion and to access their full potential and we have taken this philosophy to a new level with MyWay.

MyWay is an upright mobility device which is designed to support children in exploring, interacting and participating. With MyWay, children can say 'I'm ready to go, my way!'

I'm ready
to go
MyWay!



Taking upright movement therapy in a new direction



MyWay brings something new to the world of upright movement via its revolutionary harness support system:

- The MyWay harness delivers the core stability required to support head control and limb movement, enabling children to achieve optimum stepping ability.
- The structure and comfortable postural support delivered by the MyWay harness means it works for children with a range of needs e.g. those with scoliosis or lordosis, those with low truncal tone and those who experience dystonic movements. The MyWay harness provides a 'hug' like effect via the corset-style design, which can increase sensory feedback, dampen involuntary movement and provide comfort.
- The MyWay harness supports children in experiencing an upright view of the world, supporting their cognitive, visual and social development.
- If a child is unable to complete a standing transfer, they can be hoisted directly in/out of the MyWay frame in their harness using the optional hoisting straps accessory.



Great features & benefits for both children and caregivers

The MyWay frame is packed with features:

- MyWay's frame design is completely open! Using MyWay, children can easily touch, feel, learn, discover, take part, play...the choice is theirs! The open-back allows therapists to get close to the child and work with their lower limbs whilst they are in MyWay, helping maximise the therapeutic benefits delivered by the product.
- The frame's castors are multi-functional offering brakes, direction lock, anti-roll back and resistance. This allows MyWay's castors to be set to assist children with their individual needs.
- Prone angle along with overall, fine and shoulder height can be adjusted, meaning MyWay can be fine-tuned to achieve the optimum position for mobility for each child.
- The MyWay frame can be split into two parts* and the wheeled frame is foldable, facilitating convenient storage and transportation.

PLEASE NOTE

*Not applicable to MyWay Size 1 due to its small footprint.



Choosing your MyWay

A standard MyWay configuration starts with your choice of frame and harness. The below can then be added as optional accessories if required.

- A. Saddle
- B. User Handles
- C. Ankle Guides
- D. Headrest
- E. Hoist Straps*
- F. Adult Handles
- G. Adult Connection Belt**

The Adult Connection Belt allows an adult to attach to the MyWay frame and control the frame's movements, providing additional support to the child if required. Using the connection belt allows the adult to facilitate walking and therapy exercises whilst leaving their hands free to interact with the child.

PLEASE NOTE

* E indicates the hoisting loops on the MyWay harness.

** The Adult Handles must also be purchased in order to use the Adult Connection Belt.



Find Out More

For further information on the clinical benefits associated with MyWay, please visit our website and download the MyWay Know How document.

www.leckeymyway.com





There's always more to MyWay

MYWAY FRAME SIZING / TECHNICAL INFORMATION

Frame Size	1	2	3
Age (approx.)	1-5	4-11	9-16
User Shoulder Height (to floor) (mm)	635 – 920	870 – 1190	1070 – 1490
User Saddle Height (inside leg) (mm)	220 – 440	400 – 610	550 - 810
Max. User Weight (kg)	25	50	80
Frame Weight (kg)	12	6.5	9.5
Upper Support Unit Weight (kg)	Size 1 frame and upper support unit do not separate		11
Frame Width (mm)	570	670	720
Frame Length (mm)	720	850	1100

MYWAY HARNESS SIZING / TECHNICAL INFORMATION

Harness Size	1	2	3	4	5	6
Age (approx.)	1-4	2-6	5-9	8-12	12-15	16-18
Waist at navel level (cm)	46-56	50-60	55-65	60-70	70-80	80-90
Colour	Purple	Green	Blue	Red	Orange	Grey



Now you can do it your way, in MyWay!

1. Play games with friends
2. Discover new environments
3. Just go have fun!
4. Take part in activities
5. Open up a world of possibilities
6. Explore new surroundings





LECKEY
19C Ballinderry Road
Lisburn BT28 2SA
Northern Ireland
United Kingdom

T: 028 9260 0750
F: 028 9260 0799
E: info@leckey.com



LECKEYMYWAY.COM

LECKEY[®]

LS901-01