


PAL Seat Assembly

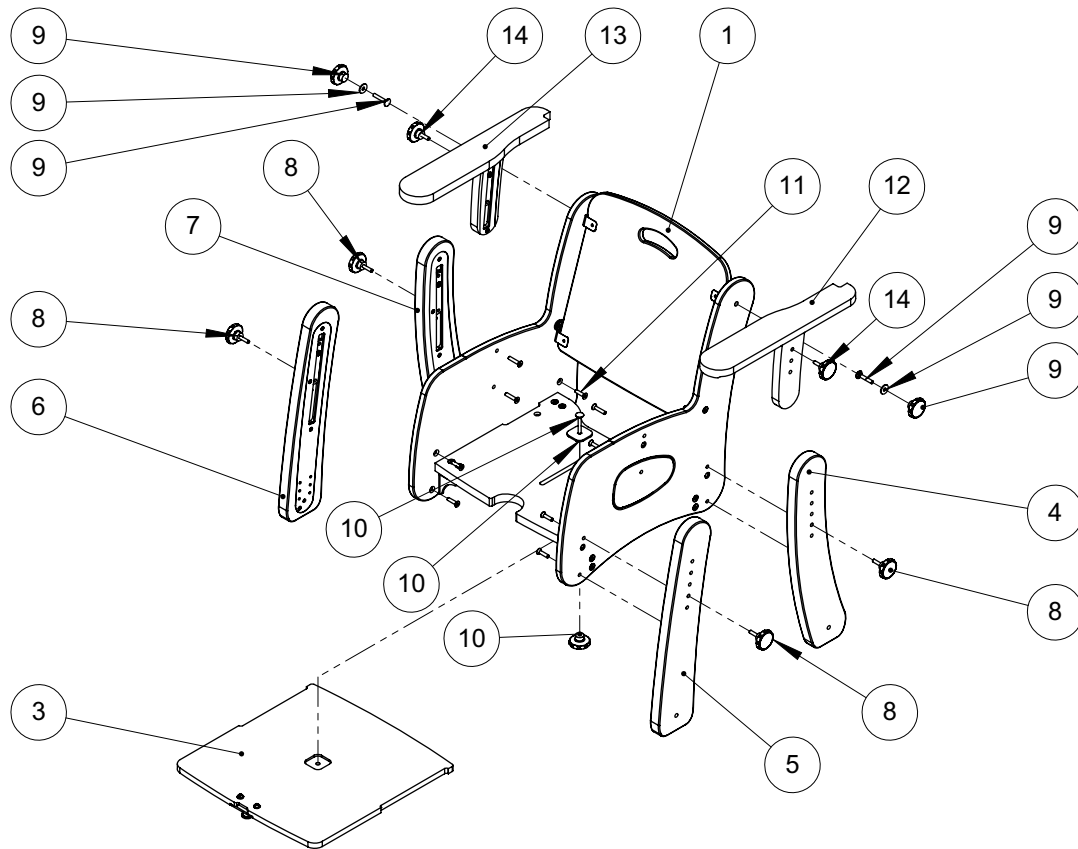

Pos.	Leckey Code	Description	Qty	Compatible with			
				Size 1	Size 2	Size 3	Size 4
1	135-9001-XX	Seat Size 1	1	✓			
1	135-9002-XX	Seat Size 2	1		✓		
1	135-9003-XX	Seat Size 3	1			✓	
1	135-9004-XX	Seat Size 4	1				✓
2	135-1657	Lap Belt Size 1	1	✓			
2	135-2657	Lap Belt Size 2	1		✓		
2	135-3657	Lap Belt Size 3	1			✓	
2	135-4657	Lap Belt Size 4	1				✓
3	135-1108	Sliding Seatbase Size 1	1	✓			
3	135-2108	Sliding Seatbase Size 2	1		✓		
3	135-3108	Sliding Seatbase Size 3	1			✓	
3	135-4108	Sliding Seatbase Size 4	1				✓
4	135-1612	Leg Rear Left Size 1	1	✓			
4	135-2612	Leg Rear Left Size 2	1		✓		
4	135-3612	Leg Rear Left Size 3	1			✓	
4	135-4612	Leg Rear Left Size 4	1				✓

PAL Seat Assembly

Pos.	Leckey Code	Description	Qty	Compatible with			
				Size 1	Size 2	Size 3	Size 4
5	135-1618	Leg Front Left Size 1	1	✓			
5	135-2618	Leg Front Left Size 2	1		✓		
5	135-3618	Leg Front Left Size 3	1			✓	
5	135-4618	Leg Front Left Size 4	1				✓
6	135-1611	Leg Front Right Size 1	1	✓			
6	135-2611	Leg Front Right Size 2	1		✓		
6	135-3611	Leg Front Right Size 3	1			✓	
6	135-4611	Leg Front Right Size 4	1				✓
7	135-1613	Leg Rear Right Size 1	1	✓			
7	135-2613	Leg Rear Right Size 2	1		✓		
7	135-3613	Leg Rear Right Size 3	1			✓	
7	135-4613	Leg Rear Right Size 4	1				✓
8	SF90	M6 Knob Male X 25 Thread	4	✓	✓	✓	✓
9	SP-1041	Backrest Knob & Screw Kit	4	✓	✓	✓	✓
10	135-9000	Sliding Seatbase Knob & Screw Kit	1	✓	✓	✓	✓
11	SP-1042	Leg Attachment Screw Kit	4	✓	✓	✓	✓
12	135-1617	Armrest Left Size 1	1	✓			
12	135-2617	Armrest Left Size 2	1		✓		
12	135-3617	Armrest Left Size 3	1			✓	
12	135-4617	Armrest Left Size 4	1				✓
13	135-1616	Armrest Right Size 1	1	✓			
13	135-2616	Armrest Right Size 1	1		✓		
13	135-3616	Armrest Right Size 1	1			✓	
13	135-4616	Armrest Right Size 1	1				✓
14	SF90	M6 Knob Male X 20 Thread	2	✓	✓	✓	✓