


## Squiggles Stander Mobile Chassis (With Fixed Wheels)

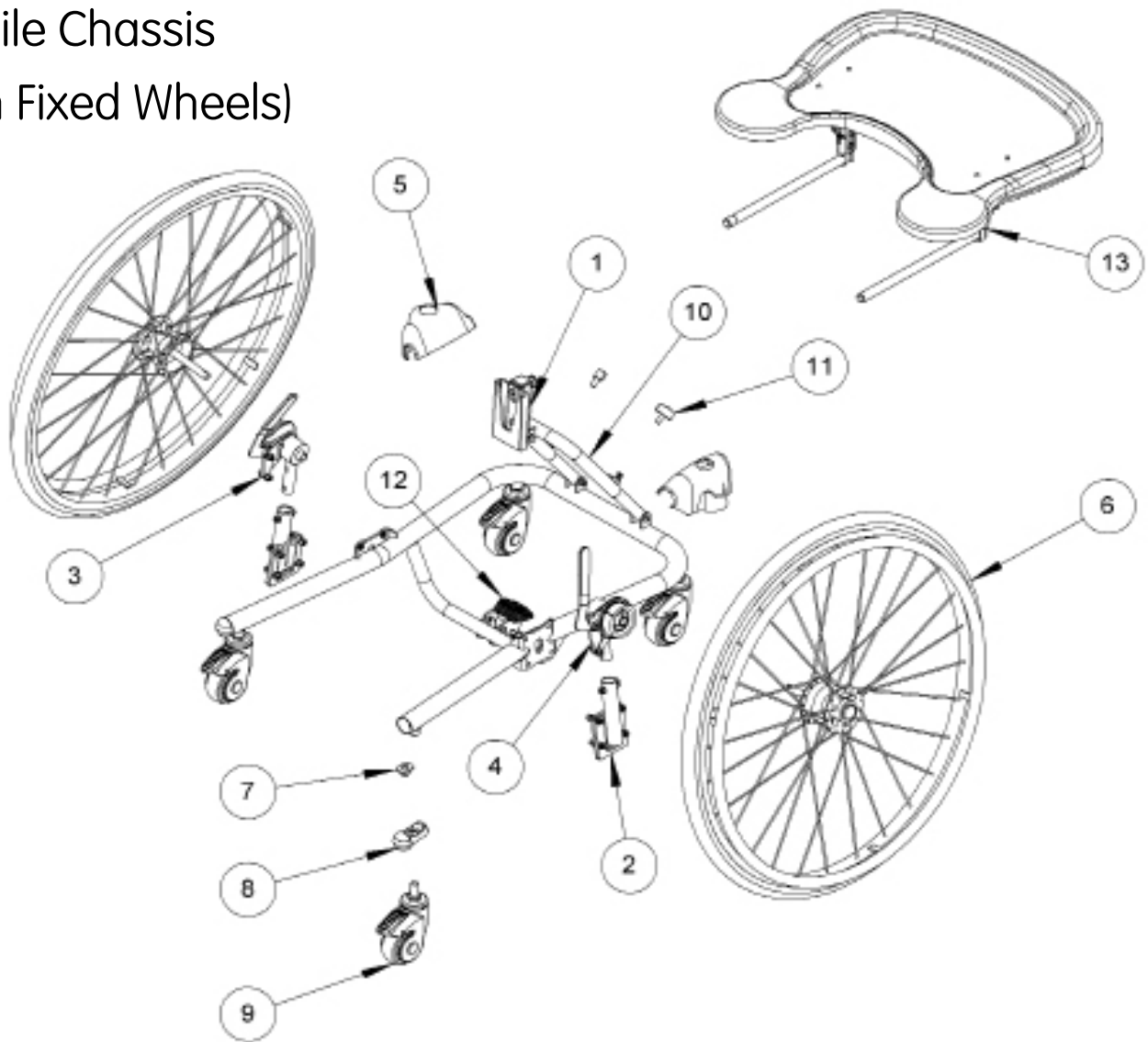

Pos.	Leckey Code	Description	Qty
1	129-600	Squiggles Mobility Chassis	1
2	129-650	Suspension Plate Assy - Fixed Wheel	1
3	129-634	RH Drum Brake Assy	1
4	129-635	LH Drum Brake Assy	1
5	129-062	Moulded Shock Cover	2
6	129-647	Wheel 24" Pair	1
6	129-646	Wheel 26" Pair	1
7	SJ136	Jetpress Nut	4
8	118-160	Castor Tube Insert	4
9	IA34	Castor	4
10	129-625	Angle Tube Outer Assembly	1
11	SF19	Locking Key M8 x 16	4
12	102-1232	Rubber Pad	1
13	118-619	Tray	1