

LECKEY^o


Squiggles Early Sitting System Spare Parts Catalogue




Contents

03-05	Squiggles Early Sitting System
06	Squiggles Early Sitting System Tray Assembly

Squiggles Early Sitting System

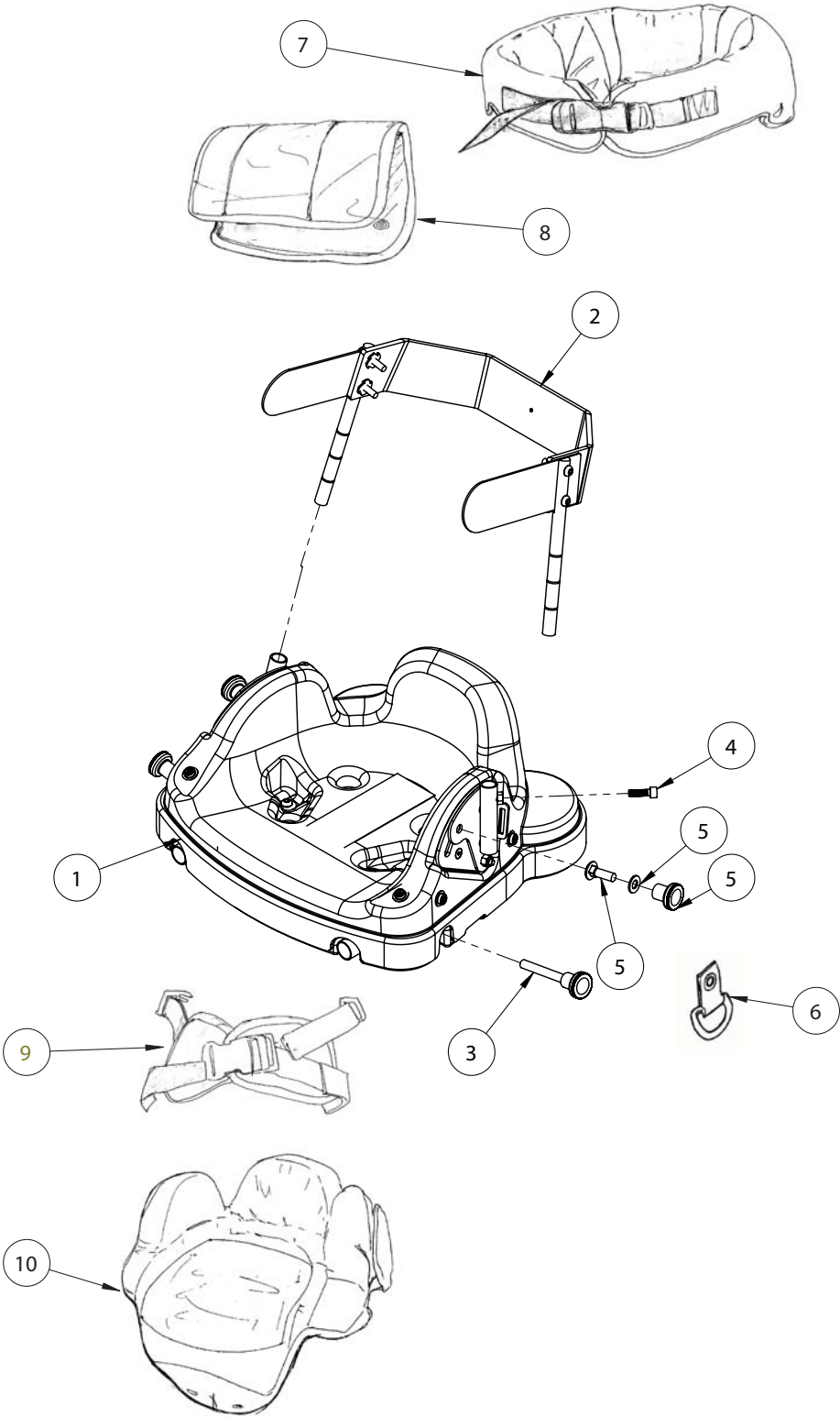


Squiggles Early Sitting System

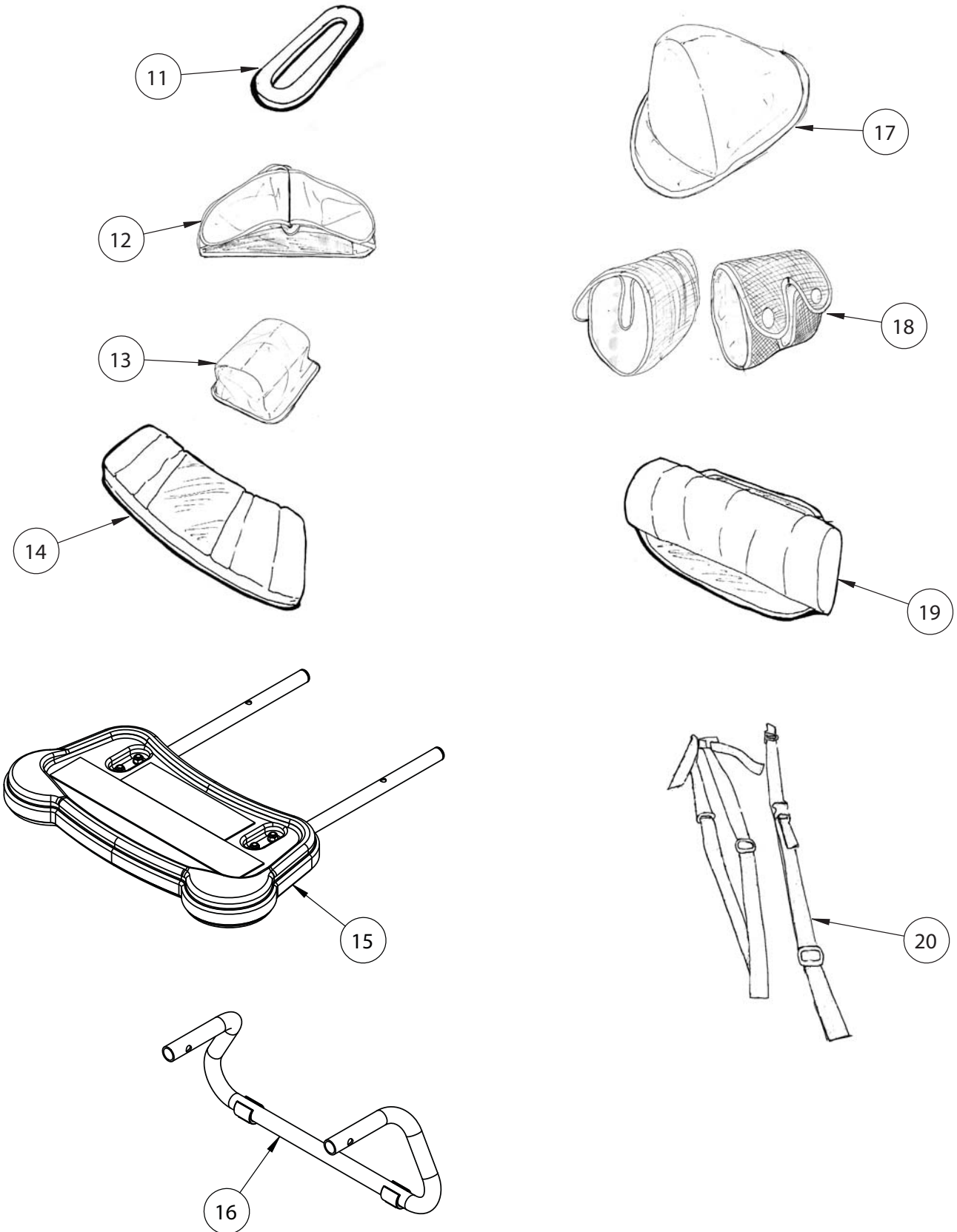

Squiggles Early Sitting System

Pos.	Leckey Code	Description	Qty
1	119-601	Base Assembly	1
2	119-609	Trunk Assembly	1
3	SF100	Elesa Knurled Locking Knob	2
4	SJ60	M6 X 12 Socket Cap	1
5	SP-1024	Ess Screw Kit	1
6	119-674	Seat Side Strap	2
7	117-421	Trunk Support Upholstery Surround	1
8	119-422	Trunk Support Upholstery Front Flap	1
9	119-651	Pelvic Harness	1
10	119-667	Seat Cover	1
11	119-044	Positioning Strap Slide	1
12	119-671	Positioning Strap	1
13	119-668	Leg Support Pommel	1
14	119-665	Leg Support Cushion	1
15	119-621	Leg Support Assembly	1
16	119-062	Prone Bar	1
17	119-669	Abduction Pommel	1
18	119-672	Spacer Pads	2
19	119-670	Flexion Roll	1
20	119-653	Chair Attachment Straps	1

Squiggles Early Sitting System Tray Assembly

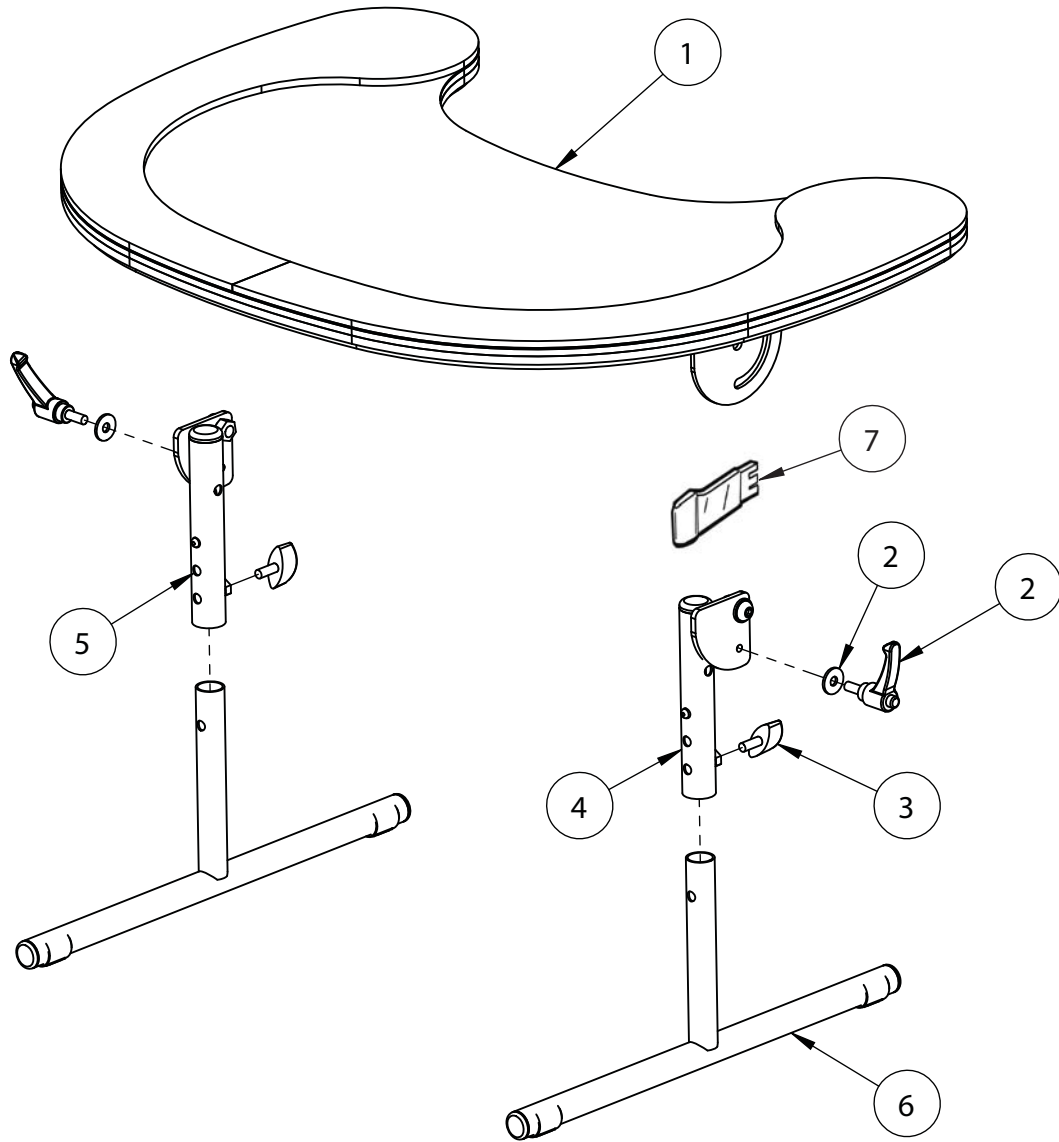

Pos.	Leckey Code	Description	Qty
1	119-629	Tray Assembly Top	1
2	SP-1025	Tray Adjustment Screw Kit	1
3	SF87	Locking Key M6 X 10	2
4	119-645	Upper Leg Assembly Lh	1
5	119-644	Upper Leg Assembly Rh	1
6	119-639	Lower Leg Assembly	2
7	119-078	Tray Attachment Strap	1