

LECKEY^o


Squiggles Seat Spare Parts Catalogue



Contents


03	Squiggles Seat - Product Overview
04	Squiggles Seat - Sliding Seatbase Assembly
05	Squiggles Seat - Static Seatbase Assembly
06	Squiggles Seat - Footrest Assembly
07	Squiggles Seat - Backrest Assembly
08	Squiggles Seat - Lateral Assembly
09	Squiggles Seat - Tray Assembly
10	Squiggles Seat - Hi-Low Base Assembly
11	Squiggles Seat - Easy Chassis

Squiggles Seat - Product Overview

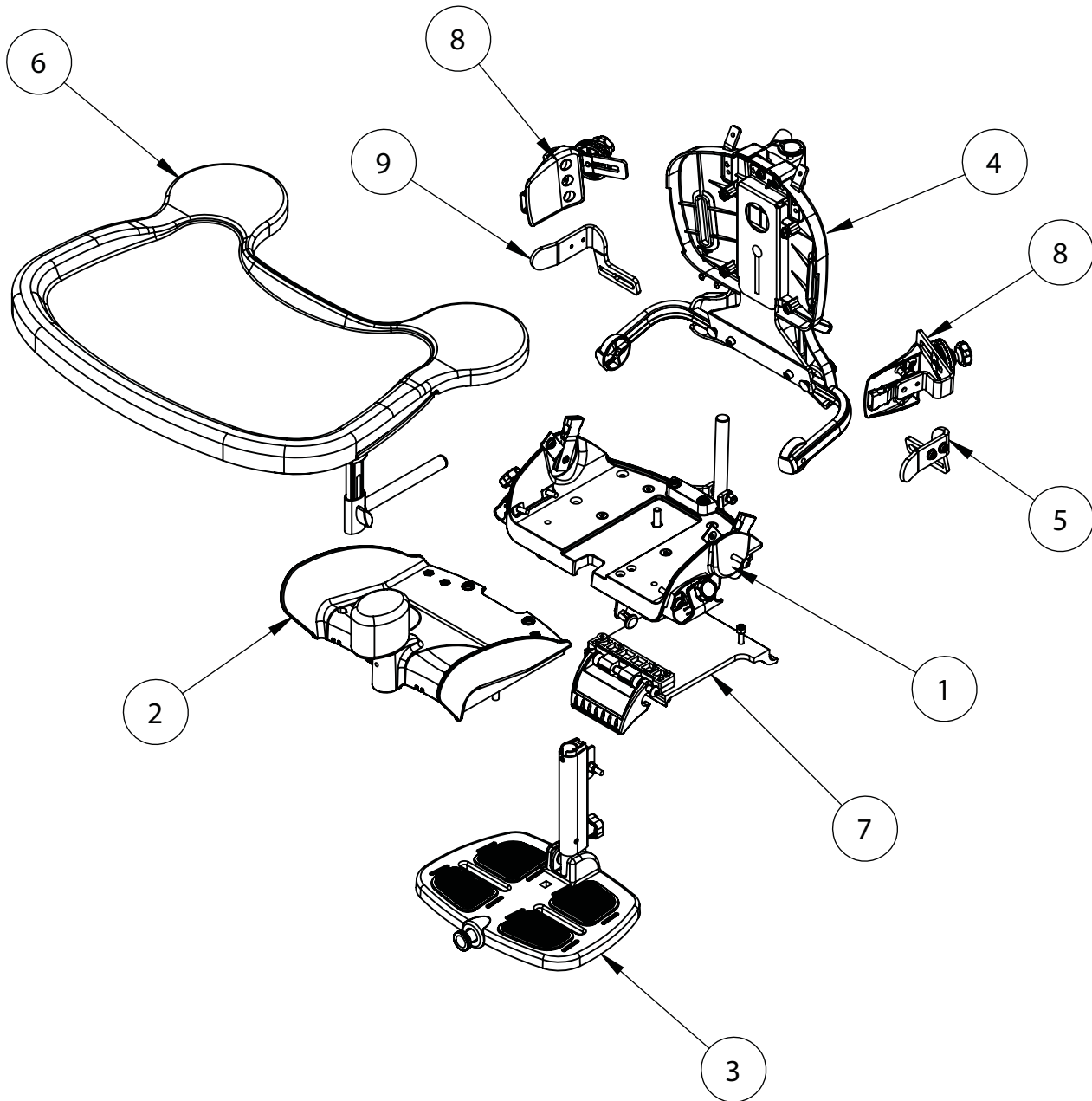


Pos.	Leckey Code	Description	Qty
1	120-635	Static Seat Base Assembly	1
2	120-634	Sliding Seat Base Assembly	1
3	120-637	Footrest Assembly	1
4	120-640	Backrest Assembly	1
5	120-195	LHS Hip Guide	1
6	120-672	Tray Assembly	1
7	117-720	Interface Assembly	1
8	120-656	Rigid Lateral Assembly	1
9	120-201	RHS Hip Guide	1

Squiggles Seat - Sliding Seatbase Assembly

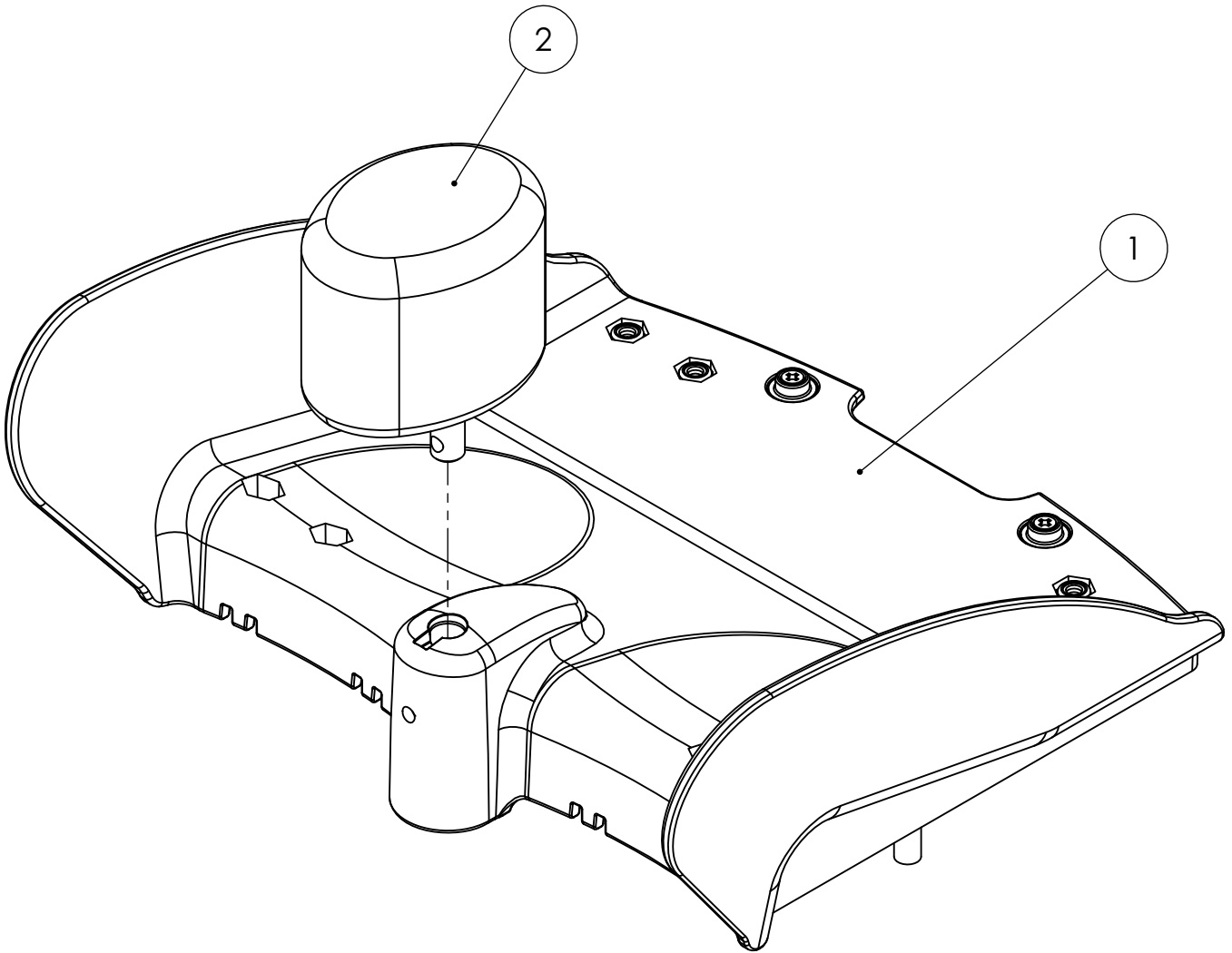


Pos.	Leckey Code	Description	Qty
1	120-634	Sliding Seat Base Assembly	1
2	120-641	Pommel	1

Squiggles Seat - Static Seatbase Assembly

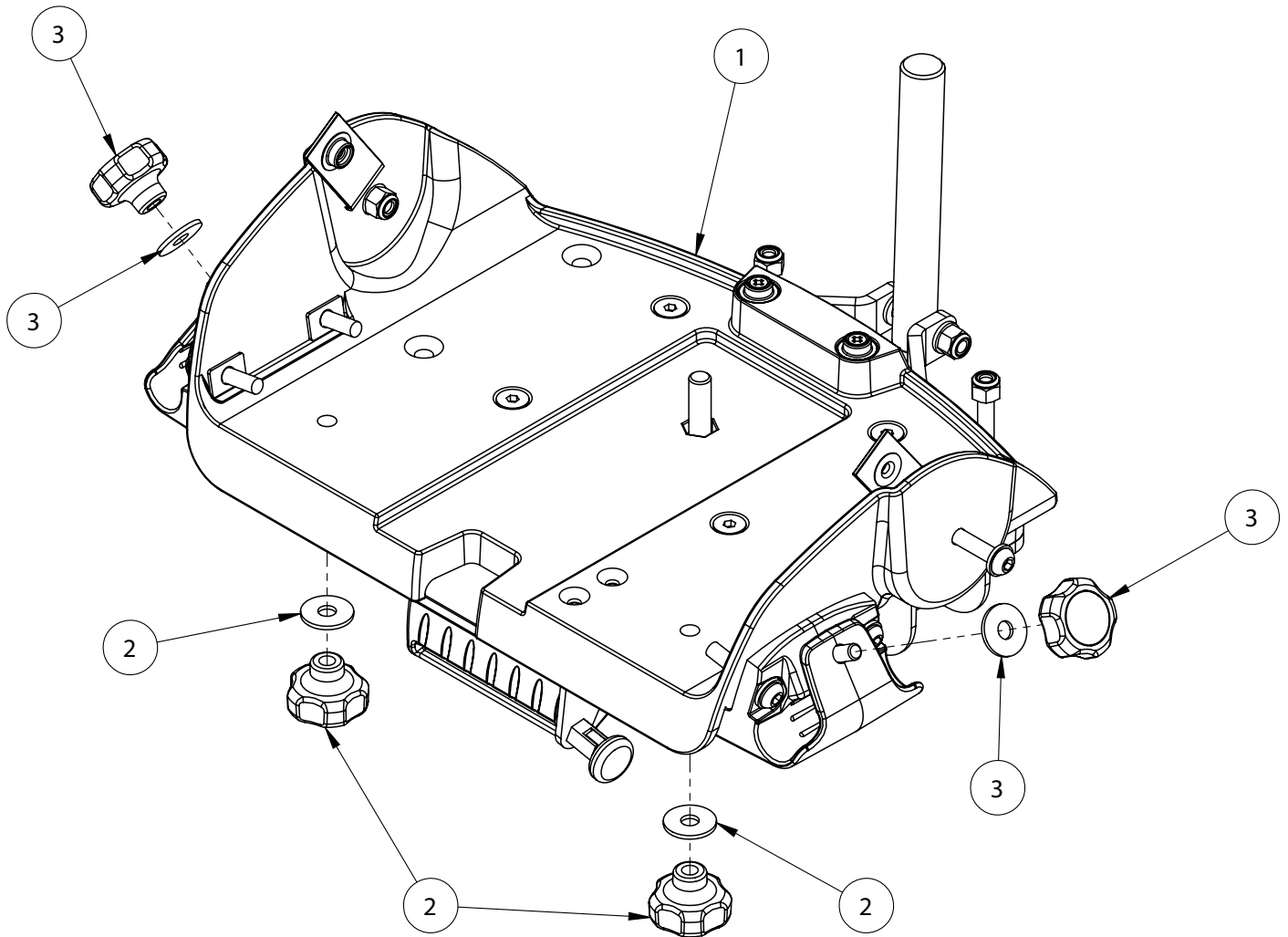


Pos.	Leckey Code	Description	Qty
1	120-635	Static Seat Base Assembly	1
2	SP-1027	Squiggles Seatbase Depth Adjustment Kit	1
3	SP-1028	Tray Assembly Adjustment Kit	1

Squiggles Seat - Footrest Assembly

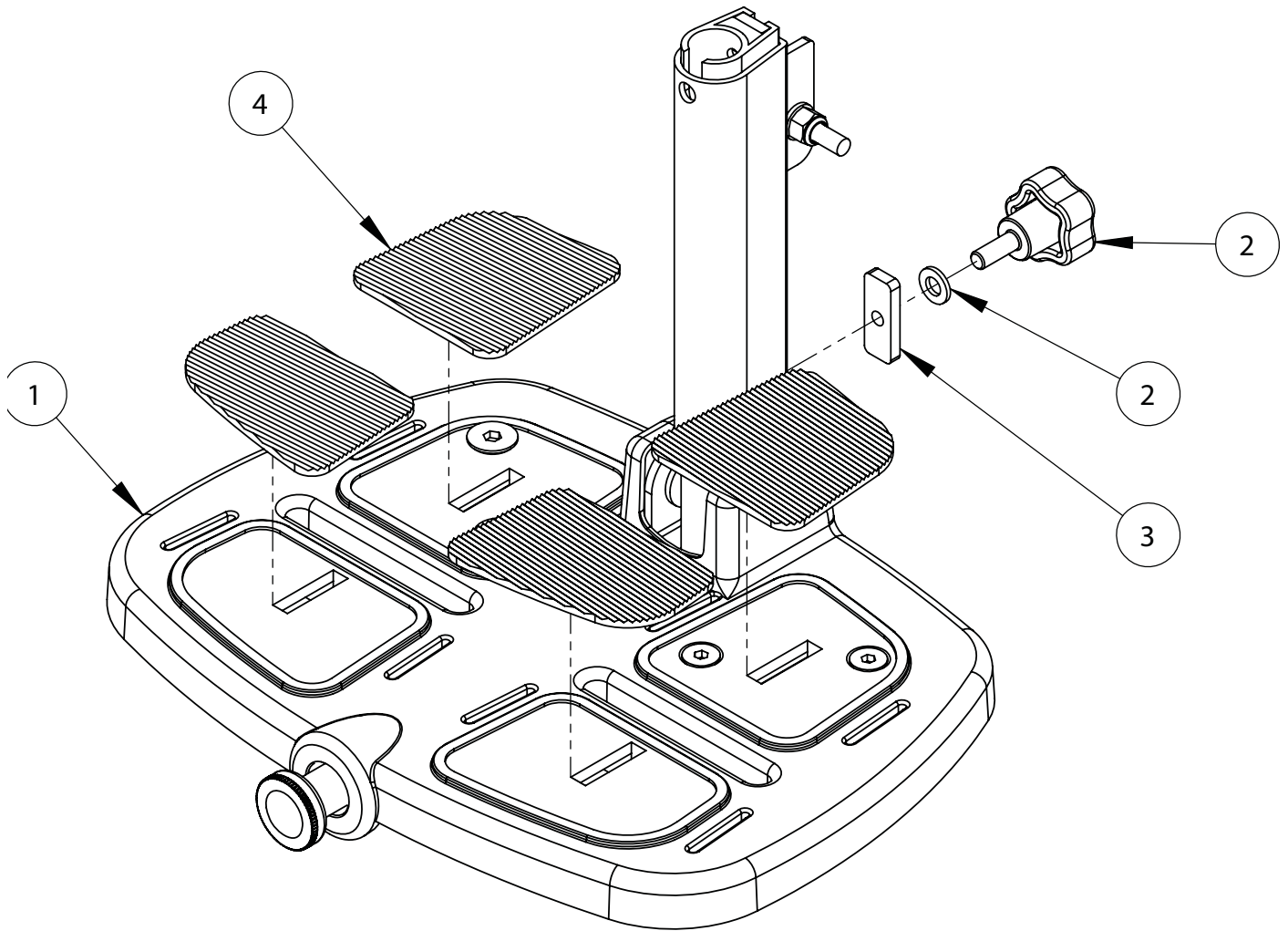


Pos.	Leckey Code	Description	Qty
1	120-637	Footrest Assembly	1
2	SP-1029	Footrest Locking Kit	1
3	117-115	Extrusion Locking Plate	1
4	120-214	Footplate Pad Kit	1

Squiggles Seat - Backrest Assembly

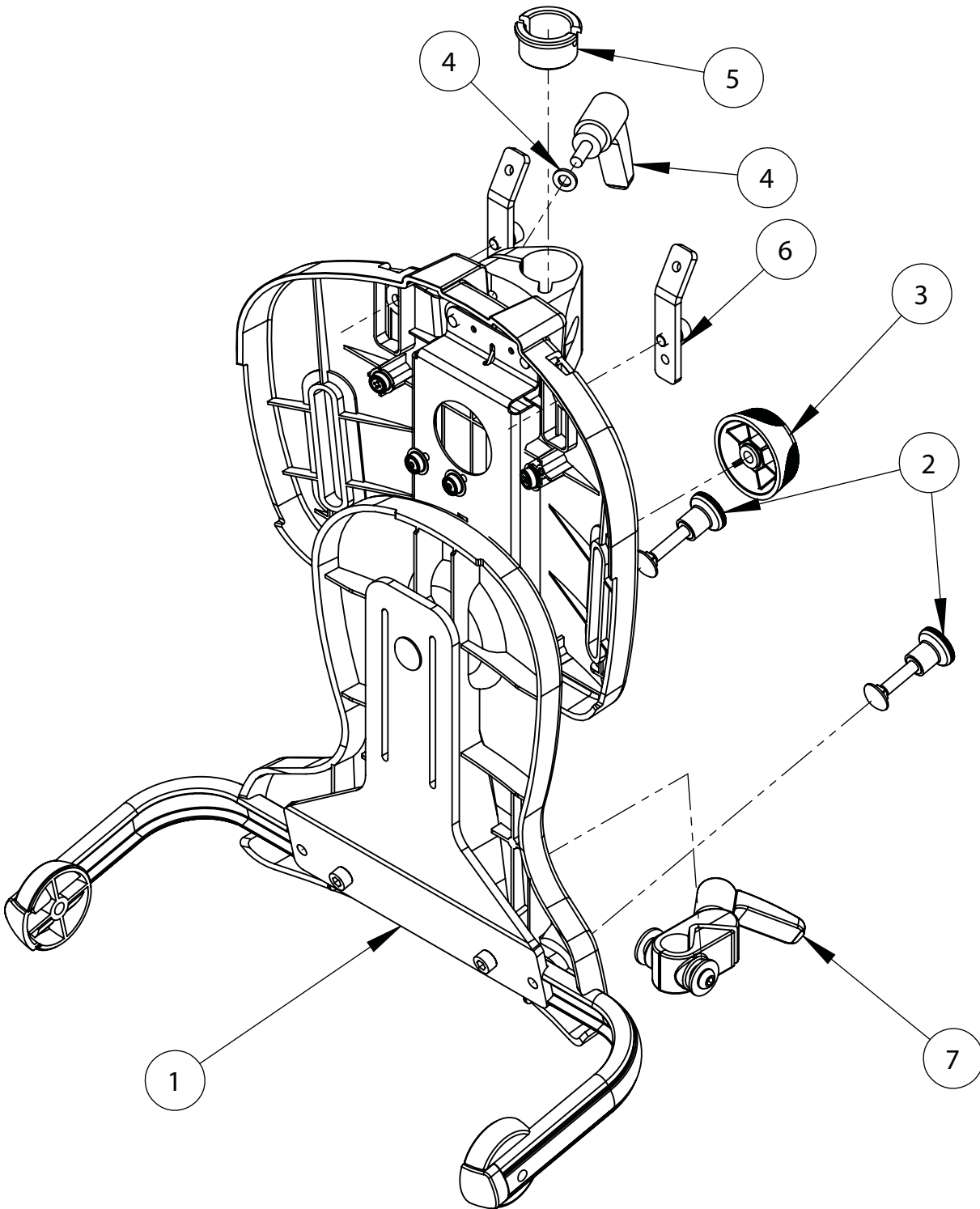


Pos.	Leckey Code	Description	Qty
1	120-640	Backrest Assembly	1
2	SF01	Elesa Knurled Knob Kit	1
3	118-716	Orb Knob	1
4	120-1020	Head Support Ratchet Assembly	1
5	118-166	Head Support Collar	1
6	120-9029	Trunk Harness Attachment Kit	1
7	120-665	Backrest Angle Clamp Assembly	1

Squiggles Seat - Lateral Assembly

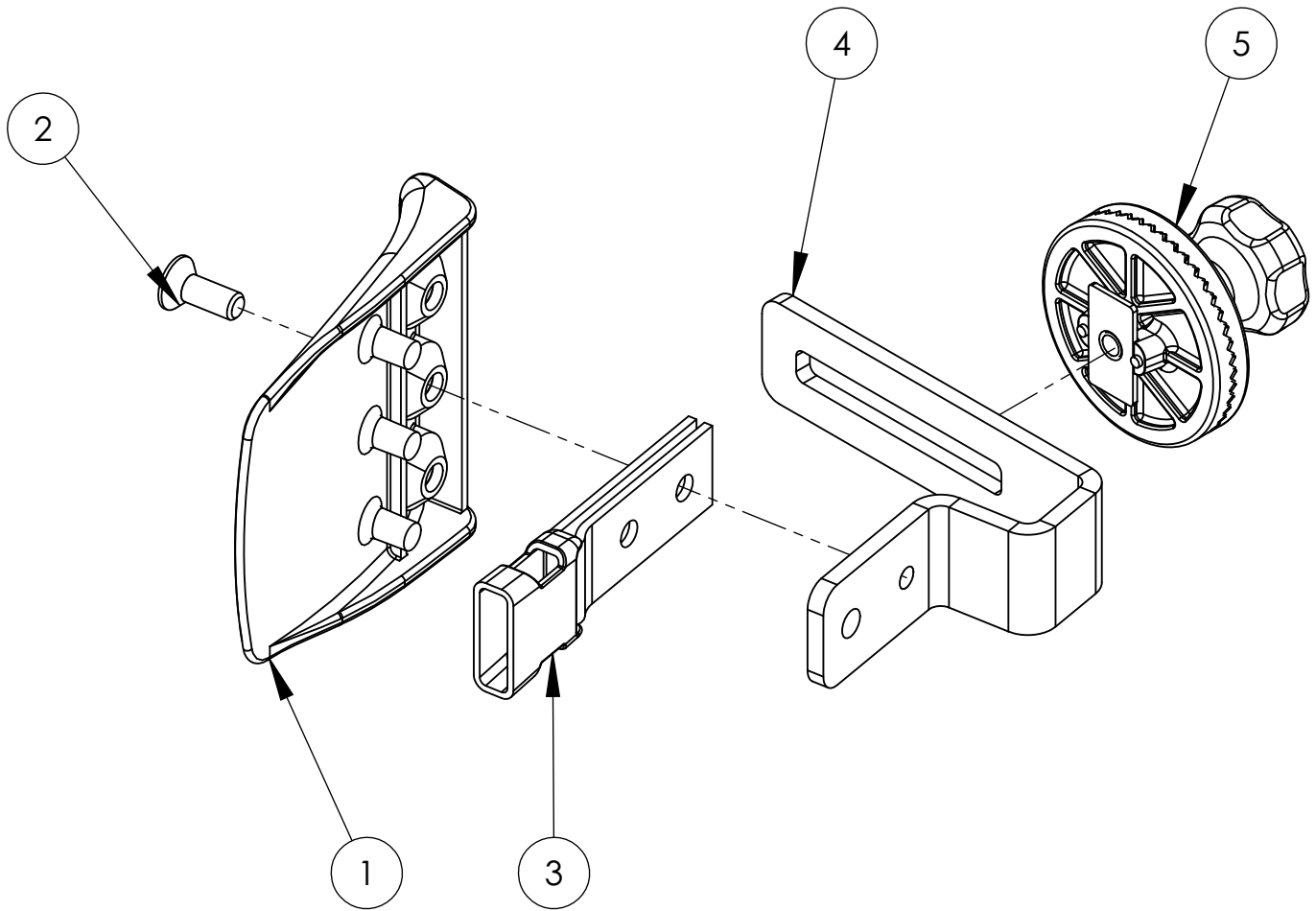


Pos.	Leckey Code	Description	Qty
1	118-049	Lateral	1
2	SL19	Csk Socket Screw M6 X 16	1
3	117-700	Harness Buckle Attachment	1
4	120-175	Lateral Bracket	1
5	117-139	Angle Lock Assembly	1

Squiggles Seat - Tray Assembly

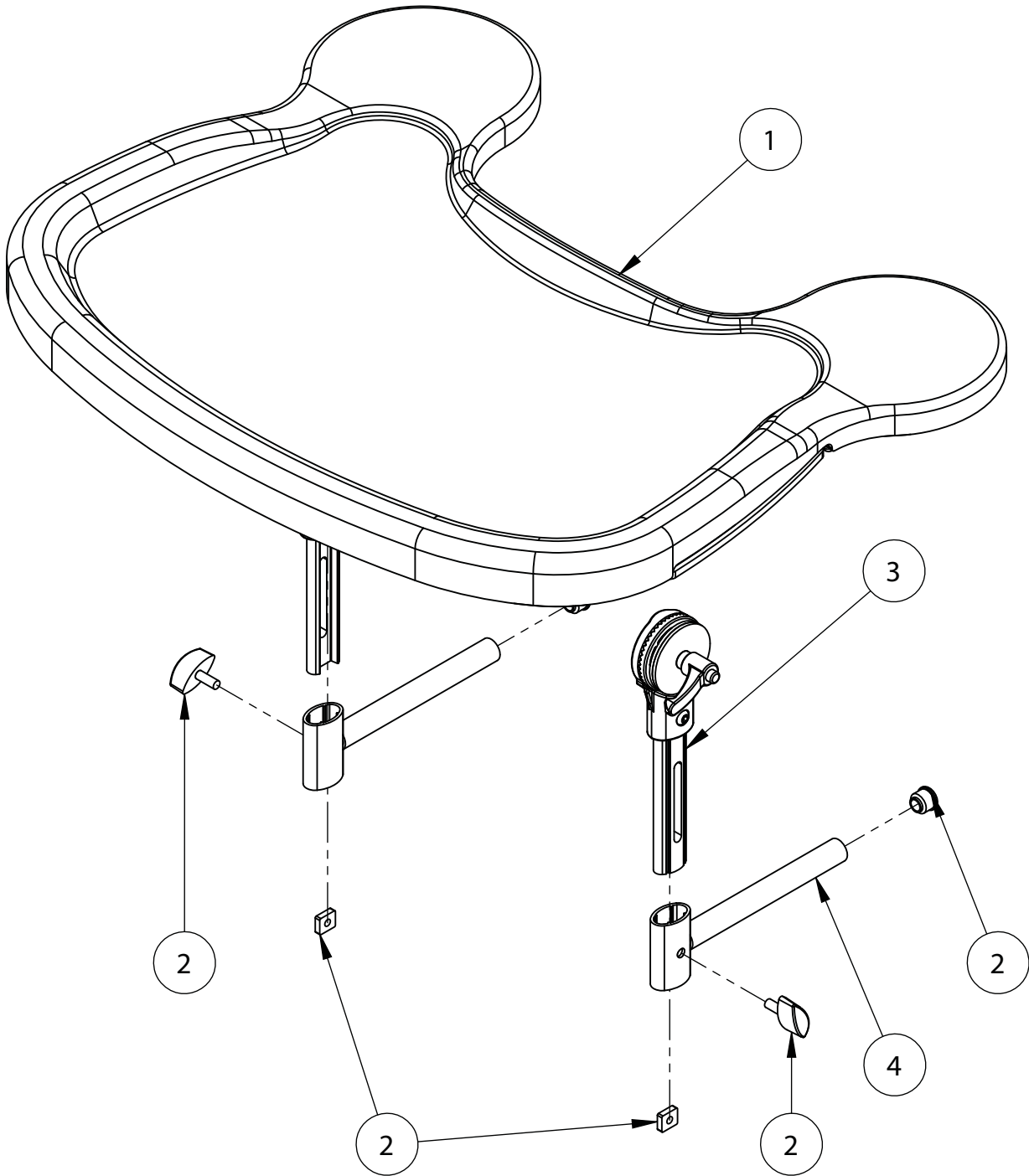


Pos.	Leckey Code	Description	Qty
1	120-672	Tray Assembly	1
2	120-9032	Tray Adjustment Kit	2
3	120-671	Tray Height Adjustment Assembly	2
4	120-638	Tray Depth Adjustment Assembly	2

Squiggles Seat - Hi-Low Base

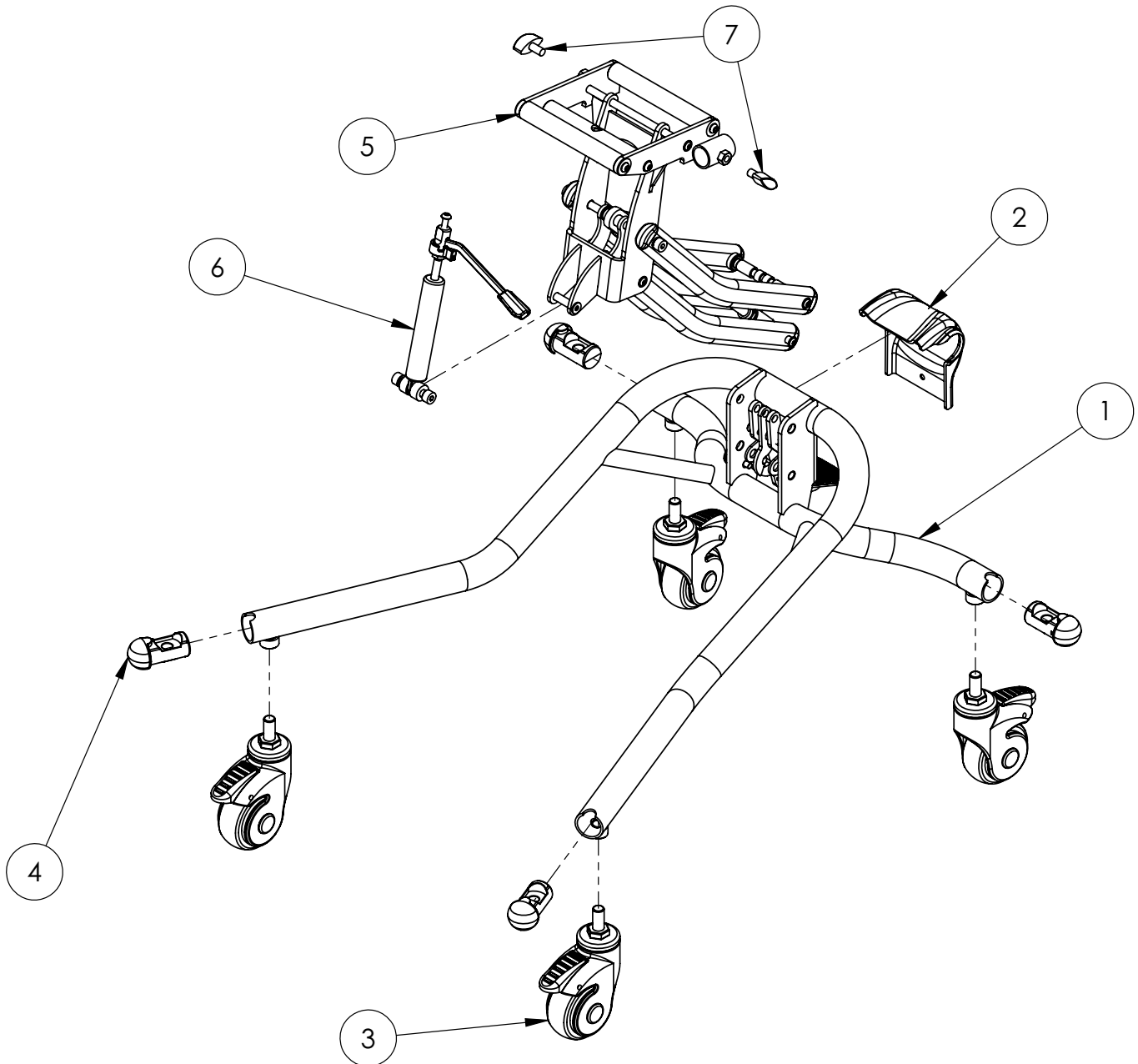

Pos.	Leckey Code	Description	Qty
1	120-800	Squiggles Chassis Assembly	1
2	120-371	Chassis Rear Moulding	1
3	IA32	Castor	4
4	118-160	Castor Tube Insert	4
5	120-802	Chassis Arm Assembly	1
6	120-812	Bansback Tilt Spring Assembly	1
7	SF19	Locking Key (M8 X 16)	2

Squiggles Seat - Easy Chassis

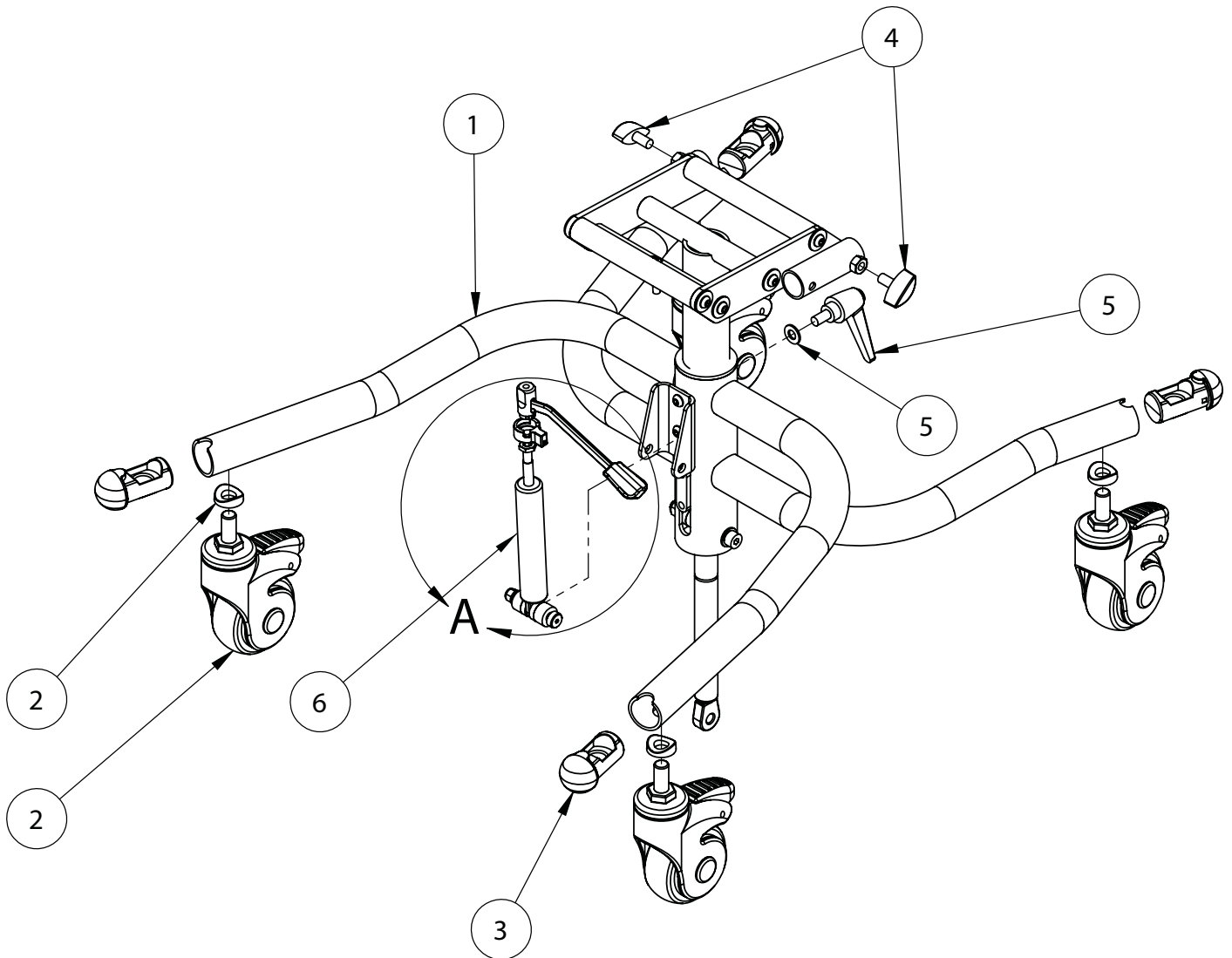

Pos.	Leckey Code	Description	Qty
1	122-717	Easy Chassis	1
2	120-9031	Easy Chassis Castor Kit	4
3	118-160	Castor Tube Insert	4
4	SF19	Locking Key (M8 X 16)	2
5	SP-1030	Easy Chassis Ratchet Kit	1
6	120-812	Gas Strut Activation	1