


Stander Frame Assembly

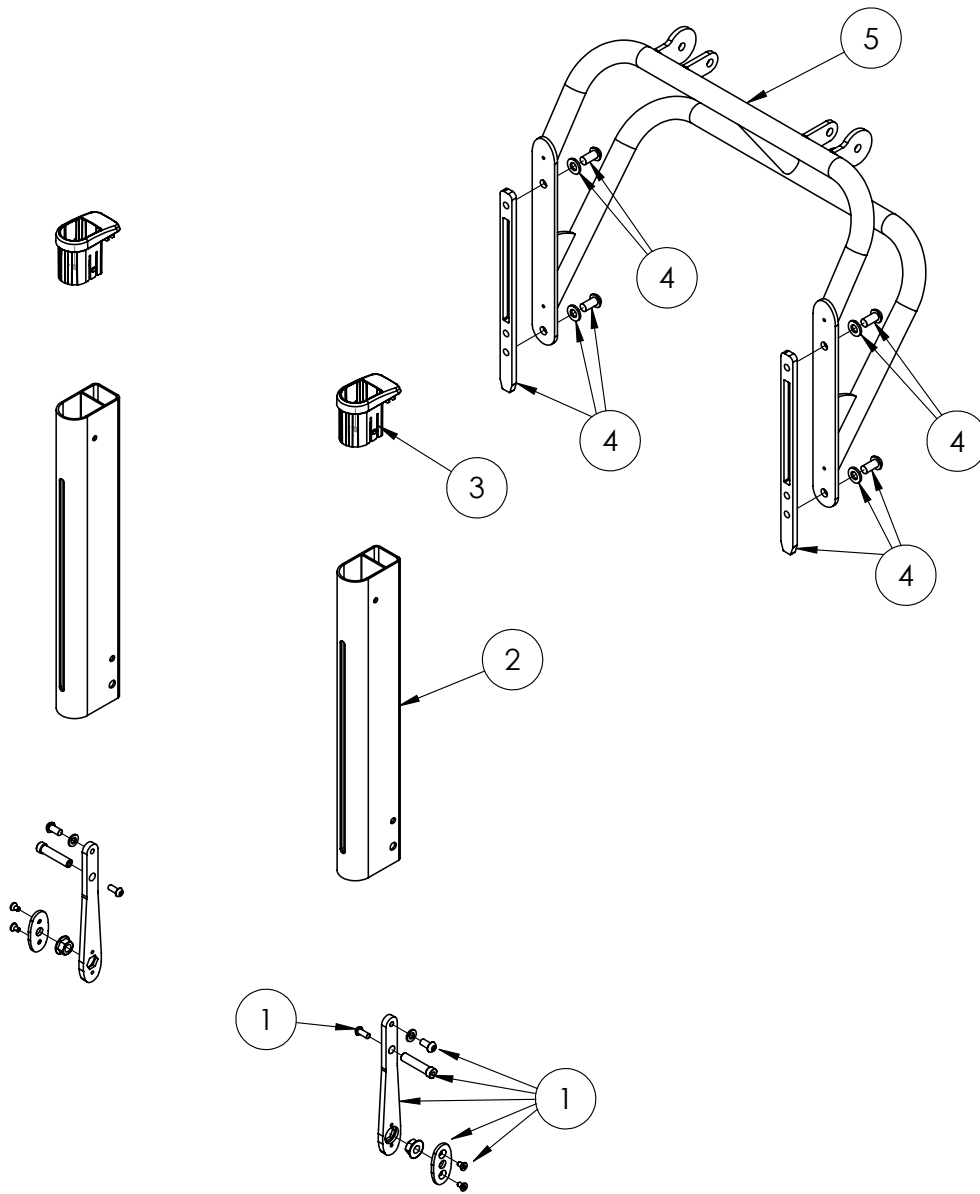

Pos.	Leckey Code	Description	Qty	Compatible with	
				Size 1	Size 2
1	143-9012	FOOTPLATE HINGE FIXINGS KIT	2	✓	✓
2	143-1108	SIDE EXTRUSION - SIZE 1	2	✓	
2	143-2108	SIDE EXTRUSION - SIZE 2	2		✓
3	143-1110	EXTRUSION TOP AND BUSH	2	✓	✓
4	143-9013	FRAME ASSY FIXINGS KIT	2	✓	✓
5	143-1612	CROSS BAR WELD ASSY - SIZE 1	1	✓	
5	143-2612	CROSS BAR WELD ASSY - SIZE 2	1		✓