

Toilet Chair


Spare Parts Catalogue



Contents


03	Toilet Chair
04	Toilet Chair Frame

Toilet Chair

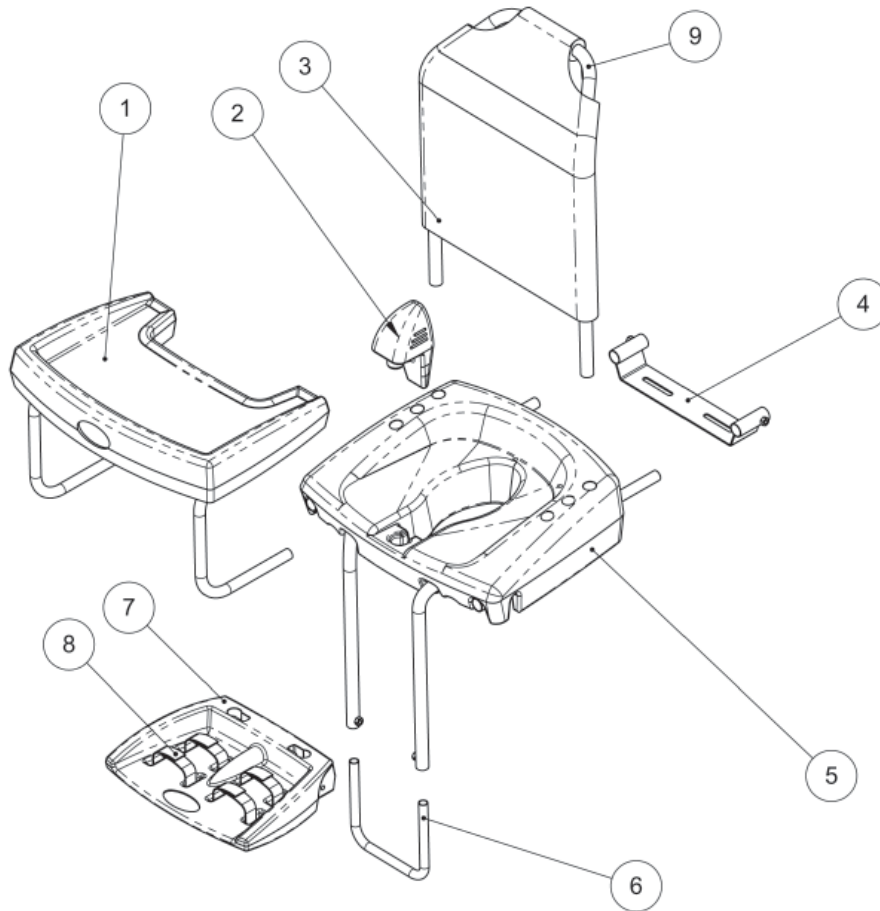

Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	LTS/1/C	Tray Assembly Size 1	1	✓		
1	LTS/2/C	Tray Assembly Size 2	1		✓	
1	LTS/3/C	Tray Assembly Size 3	1			✓
2	RAP-4-1032	Toilet Chair Pommel Size 1	1	✓		
2	RAP-4-3032	Toilet Chair Pommel Size 2 & 3	1		✓	✓
3	LTS/1/E	Toilet Chair Backrest Cover Size 1	1	✓		
3	LTS/2/E	Toilet Chair Backrest Cover Size 2 & 3	1		✓	✓
4	OTS/1/A	Toilet Chair Bracket	1	✓	✓	✓
5	RAP-4-1963	Toilet Chair Seat Size 1	1	✓		
5	RAP-4-2963	Toilet Chair Seat Size 2	1		✓	
5	RAP-4-3963	Toilet Chair Seat Size 3	1			✓
6	RAP-4-1011	U Bend Footplate Stability Bar All Sizes	1	✓	✓	✓
7	RAP-4-1033	Footrest All Sizes	1	✓	✓	✓
8	LTS/1/D	Footrest Straps All Sizes	1	✓	✓	✓
9	RAP-4-1041	Backrest Tube Size 1	1	✓		
9	RAP-4-2041	Backrest Tube Size 2	1		✓	
9	RAP-4-3041	Backrest Tube Size 3	1			✓
	LTS Care Kit	Toilet Seat Care Kit	1	✓	✓	✓

Toilet Chair Frame

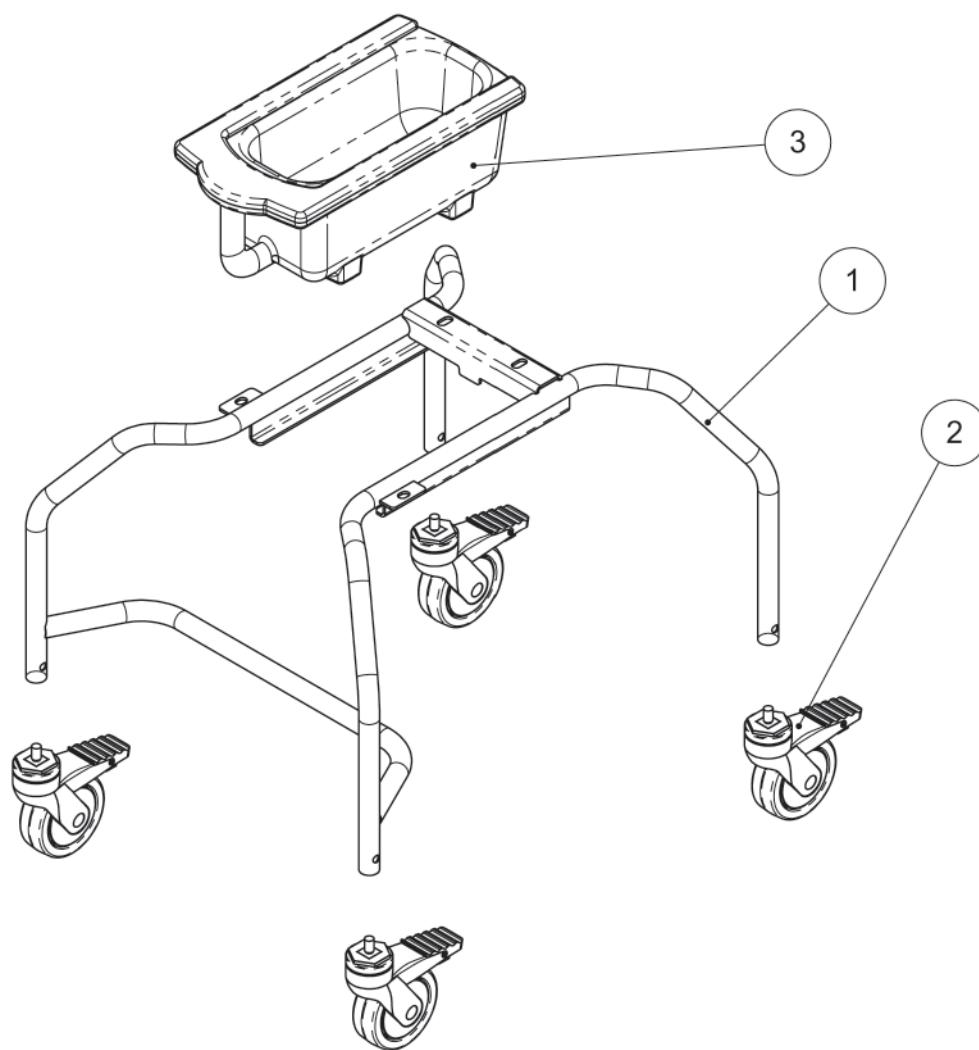

Pos.	Leckey Code	Description	Qty
1	RAP-4-1601	Welded Frame Assembly	1
2	IA14	Castor	1
3	RAP-4-1030	Toilet Chair Potty	1