


Armrest Assembly

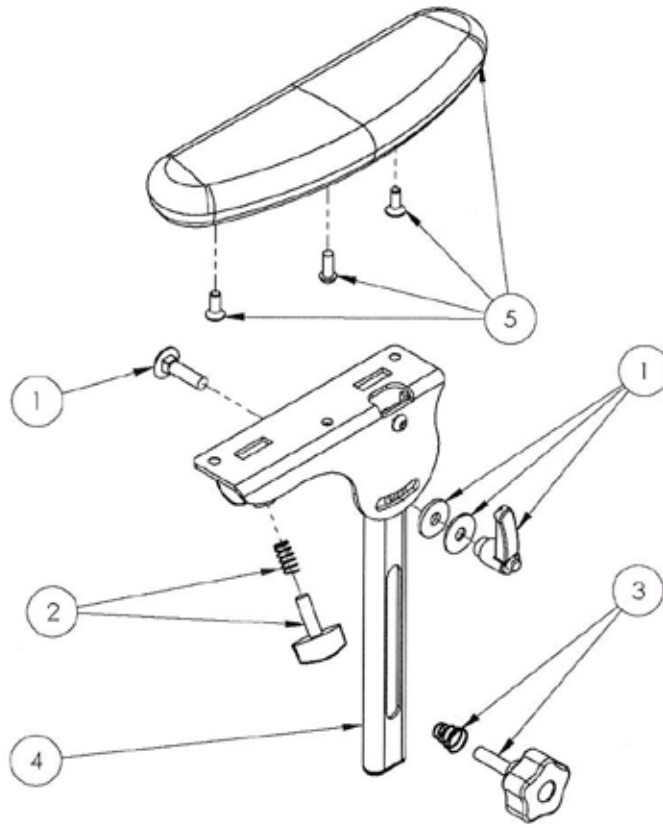

Pos.	Leckey Code	Description	Qty	Compatible with		
				Size 1	Size 2	Size 3
1	151-1812	Armrest Adjustment Kit	1	✓	✓	
1	151-3812	Armrest Adjustment Kit	1			✓
2	151-1813	Tray to Armrest Locking Wingnut	1	✓	✓	
2	151-3813	Tray to Armrest Locking Wingnut	1			✓
3	151-1814	Armrest Lock Kit	1	✓	✓	
3	151-3814	Armrest Lock Kit	1			✓
4	151-1820	Armrest Frame LHS Size 1	1	✓		
4	151-2820	Armrest Frame LHS Size 2	1		✓	
4	151-3820	Armrest Frame LHS Size 3	1			✓
4	151-1821	Armrest Frame RHS Size 1	1	✓		
4	151-2821	Armrest Frame RHS Size 2	1		✓	
4	151-3821	Armrest Frame RHS Size 3	1			✓
5	151-1822	Armrest PU Size 1	1	✓		
5	151-2822	Armrest PU Size 2	1		✓	
5	151-3822	Armrest PU Size 3	1			✓