


## Indoor / Outdoor Hi-low Chassis – Power

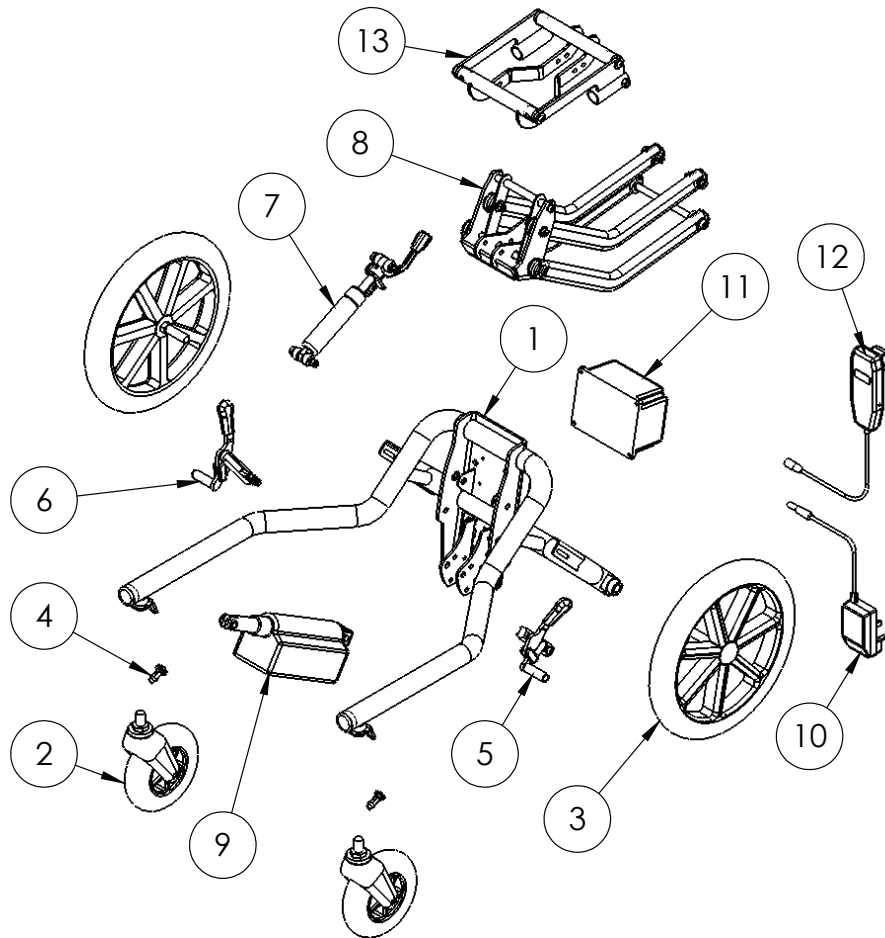

Pos.	Leckey Code	Description	Qty	Compatible with	
				Size 1	Size 2
1	102-1770	Chassis Welded Assy Size 1&2	1	✓	✓
2	133-2084	150mm Front Caster	1	✓	✓
3	133-2087	315mm Rear Wheel	1	✓	✓
4	la19	Lock Pin	1	✓	✓
5	133-2085	Brake Assy Lh	1	✓	✓
6	133-2086	Brake Assy Rh	1	✓	✓
7	102-1695	500N Blocolift Size 1&2 Assy	1	✓	✓
8	117-651	Lever Arm Assembly Size 1-2	1	✓	✓
9	102-1694	Actuator Assy	1	✓	✓
10	SI74	Charger UK	1	✓	✓
10	SI125	Charger EUR	1	✓	✓
10	SI126	Charger USA/Japan	1	✓	✓
10	SI127	Charger AUS	1	✓	✓
11	SI64	Battery	1	✓	✓
12	SI67	Handset	1	✓	✓
13	117-633	Mygo Interface Assembly	1	✓	✓