


Posterior Support Assembly (incl Pommel)

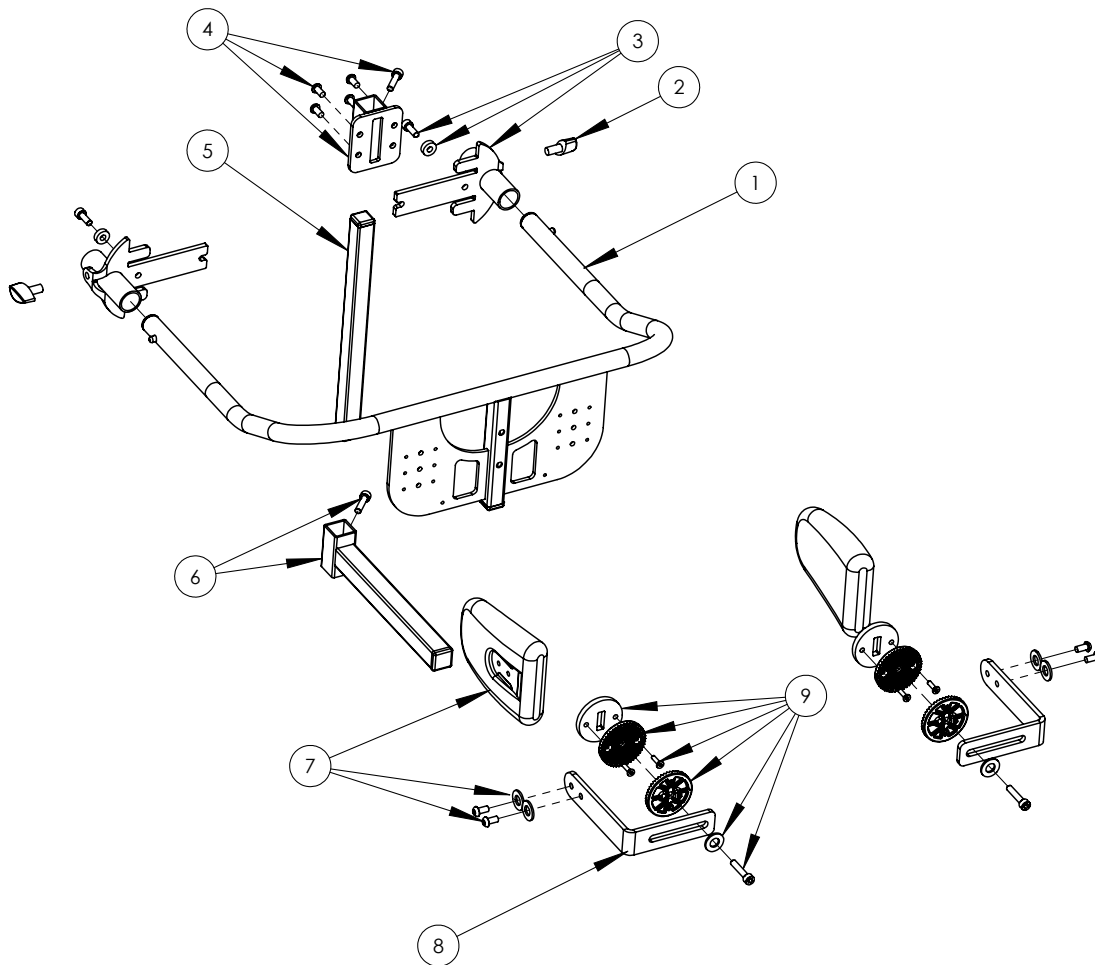

Pos.	Leckey Code	Description	Qty	Compatible with	
				Size 1	Size 2
1	143-9059	POSTERIOR SUPPORT ASSY - SIZE 1	1	✓	
	143-9063	POSTERIOR SUPPORT ASSY - SIZE 2	1		✓
2	SF19	LOCKING KEY	2	✓	✓
3	143-9056	ATTACHMENT KIT	2	✓	✓
4	143-9090	POMMEL ATTACHMENT KIT		✓	✓
	143-9091	POMMEL VERTICAL BAR - SIZE 1		✓	
	143-9092	POMMEL VERTICAL BAR - SIZE 2			✓
6	143-9060	POMMEL ASSY	1	✓	✓
7	143-9061	LATERAL ATTACHMENT ASSY	2	✓	✓
8	143-1226	LATERAL BRACKET	2	✓	✓
9	143-9062	ANGLE ADJUST FIXINGS KIT	2	✓	✓
10	143-1250	POMMEL FOAM (NOT SHOWN)	1	✓	✓