


LECKEY^o

Squiggles Seat Upholstery - various

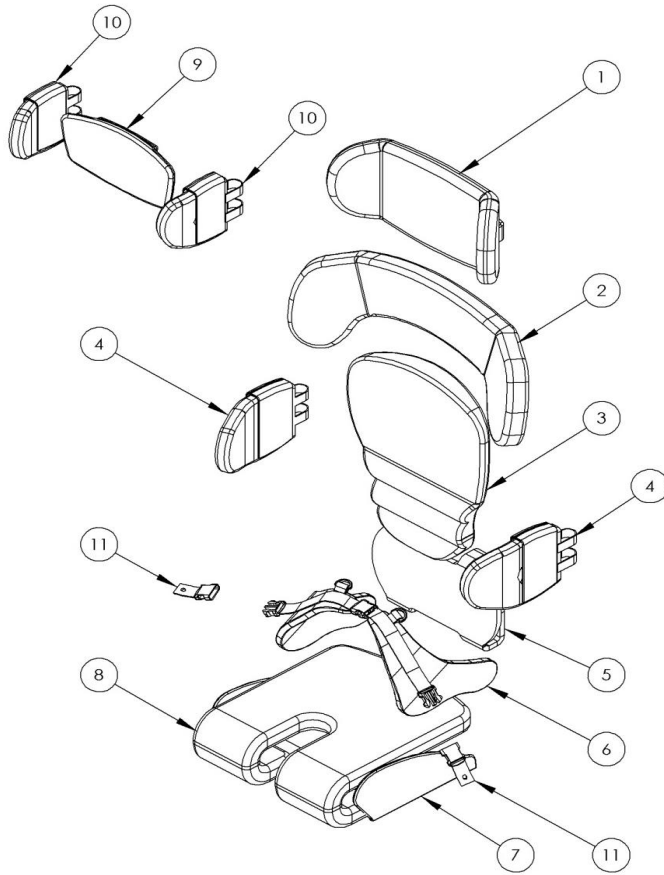


Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	137-800-01		Contoured Head Support Cushion - Green	1	✓
	137-800-02		Contoured Head Support Cushion - Orange	1	✓
	137-800-03		Contoured Head Support Cushion - Blue	1	✓
	137-800-04		Contoured Head Support Cushion - Pink	1	✓
	137-800-05		Contoured Head Support Cushion -Black	1	✓
2	120-828-01		Shoulder Support Cushion - Green	1	✓
	120-828-02		Shoulder Support Cushion - Orange	1	✓
	120-828-03		Shoulder Support Cushion - Blue	1	✓
	120-828-04		Shoulder Support Cushion - Pink	1	✓

3	120-845-01		Backrest Cushion - Green	1	✓
	120-845-02		Backrest Cushion - Orange	1	✓
	120-845-03		Backrest Cushion - Blue	1	✓
	120-845-04		Backrest Cushion - Pink	1	✓
4	120-838-01		Lateral Covers (Pair) - Green	1	✓
	120-838-02		Lateral Covers (Pair) - Orange	1	✓
	120-838-03		Lateral Covers (Pair) - Blue	1	✓
	120-838-04		Lateral Covers (Pair) - Pink	1	✓
5	120-833-01		Sacral Cushions - Green	1	✓
	120-833-02		Sacral Cushions - Orange	1	✓
	120-833-03		Sacral Cushions - Blue	1	✓
	120-833-04		Sacral Cushions - Pink	1	✓
6	120-844-01		Pelvic Harness Small - Green	1	✓
	120-844-02		Pelvic Harness Small - Orange	1	✓
	120-844-03		Pelvic Harness Small - Blue	1	✓
	120-844-04		Pelvic Harness Small - Pink	1	✓
6	117-848-01		Pelvic Harness Medium - Green	1	✓
	117-848-02		Pelvic Harness Medium - Orange	1	✓
	117-848-03		Pelvic Harness Medium - Blue	1	✓
	117-848-04		Pelvic Harness Medium - Pink	1	✓
	117-848-05		Pelvic Harness Medium - Black	1	✓
7	120-836-01		1-Piece Femoral Guide - Green	1	✓
	120-836-02		1-Piece Femoral Guide - Orange	1	✓
	120-836-03		1-Piece Femoral Guide - Blue	1	✓
	120-836-04		1-Piece Femoral Guide - Pink	1	✓
8	120-834-01		Seat Base Cushions - Green	1	✓
	120-834-02		Seat Base Cushions - Orange	1	✓
	120-834-03		Seat Base Cushions - Blue	1	✓
	120-834-04		Seat Base Cushions - Pink	1	✓
9	117-850-01		Flat Headrest Cushion - Green	1	✓
	117-850-02		Flat Headrest Cushion - Orange	1	✓
	117-850-03		Flat Headrest Cushion - Blue	1	✓
	117-850-03VIN		Flat Headrest Cushion - Blue Vinyl	1	✓
	117-850-04		Flat Headrest Cushion - Pink	1	✓
	117-850-05		Flat Headrest Cushion - Black	1	✓
10	137-807-01		Head/Hip Lateral Covers - Green	1	✓
	137-807-02		Head/Hip Lateral Covers - Orange	1	✓
	137-807-03		Head/Hip Lateral Covers - Blue	1	✓
	137-807-03VIN		Head/Hip Lateral Covers - Blue Vinyl	1	✓
	137-807-04		Head/Hip Lateral Covers - Pink	1	✓
	137-807-05		Head/Hip Lateral Covers - Black	1	✓
11	120-9004		Pelvic Harness Attachment Kit	1	✓

LECKEY^o


Squiggles Seat Backrest Assembly 120-645

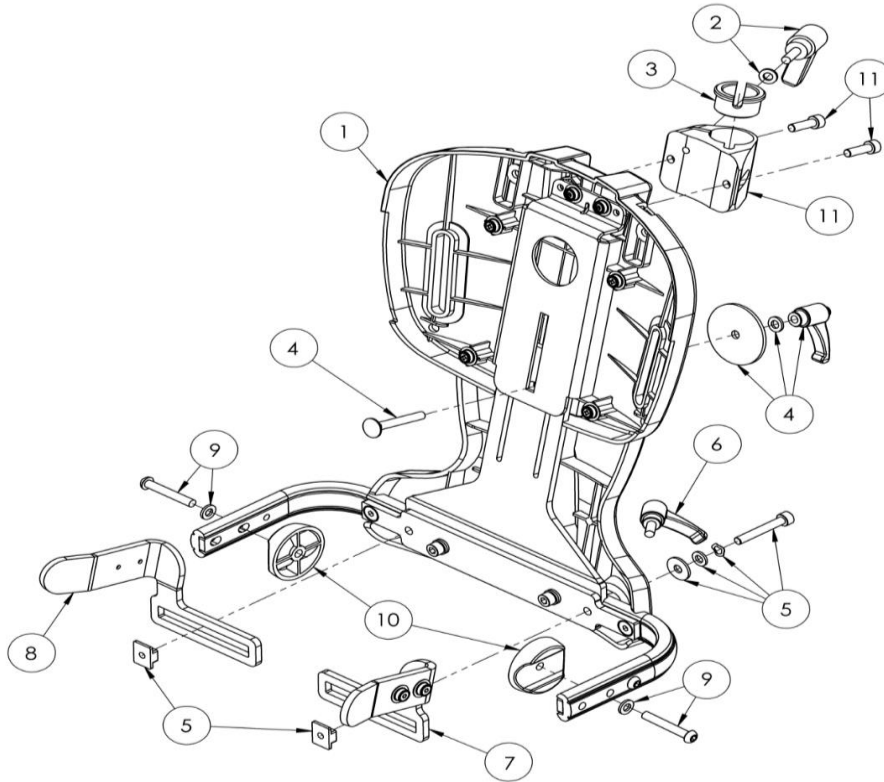

Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	120-645		Backrest Assembly	1	✓
2	SP-1020		Head Support Ratchet Assembly	1	✓
3	118-166		Head Support Collar	1	✓
4	120-9002		Backrest Adjustment Assembly	1	✓
5	120-9003		Hip Guide Locking Assembly	1	✓
6	SF101		M6 Male Ratchet Handle	1	✓
7	120-678		Hip Guide Bracket Assembly LHS	1	✓
8	120-679		Hip Guide Bracket Assembly RHS	1	✓
9	SP-1014		Seatbase Attachment Screw Kit	1	✓
10	120-190		Backrest Pivot Mould	1	✓
11	120-9005		Whitmayer Casting Assembly	1	✓

LECKEY^o

Squiggles Seat Contoured Head Support Assembly 120-859

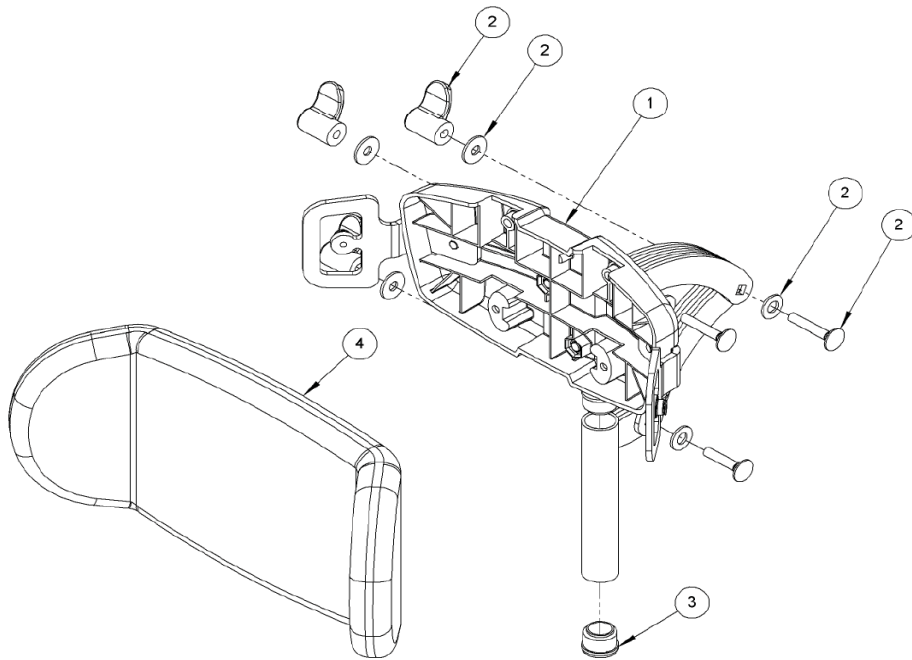


Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	120-859		Contoured Headrest (Hardware Only)	1	✓
2		137-9056	Wing Nut Assembly	30	✓
3		SF53	Round Tube Insert	30	✓
4	137-800-01		Contoured Headrest Cushion - Green	1	✓
	137-800-02		Contoured Headrest Cushion - Orange	1	✓
	137-800-03		Contoured Headrest Cushion - Blue	1	✓
	137-800-03 VIN		Contoured Headrest Cushion - Blue Vinyl	1	✓
	137-800-04		Contoured Headrest Cushion - Pink	1	✓
	137-800-05		Contoured Headrest Cushion - Black	1	✓
	137-800-15		Contoured Headrest Cushion - Grey	1	✓

Order SF88, S109, S114, SK03, SK04

LECKEY^o

Squiggles Seat Flat Head Support Assembly 117-697

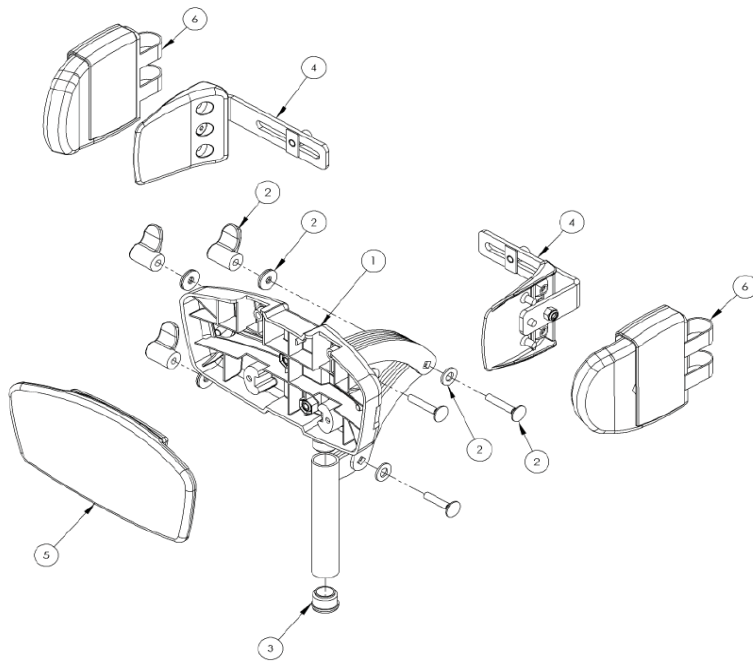


Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	117-697		Flat Headrest Support (Hardware Only)	1	✓
2		137-9056	Wing Nut Assembly	30	✓
3		SF53	Round Tube Insert (Ø22)	30	✓
4	118-666		Flat Headrest Lateral Bracket (Single)	1	✓
5	117-850-01		Flat Headrest Cover - Green	1	✓
	117-850-02		Flat Headrest Cover - Orange	1	✓
	117-850-03		Flat Headrest Cover - Blue	1	✓
	117-850-03 VIN		Flat Headrest Cover - Blue Vinyl	1	✓
	117-850-04		Flat Headrest Cover - Pink	1	✓
	117-850-05		Flat Headrest Cover - Black	1	✓
6	137-807-05		Headrest / Hip Laterals Covers (pair)	1	✓

Order SF88, S109, S114, SK03, SK04

LECKEY^o


Squiggles Seat Flipaway Lateral Assembly 120-855

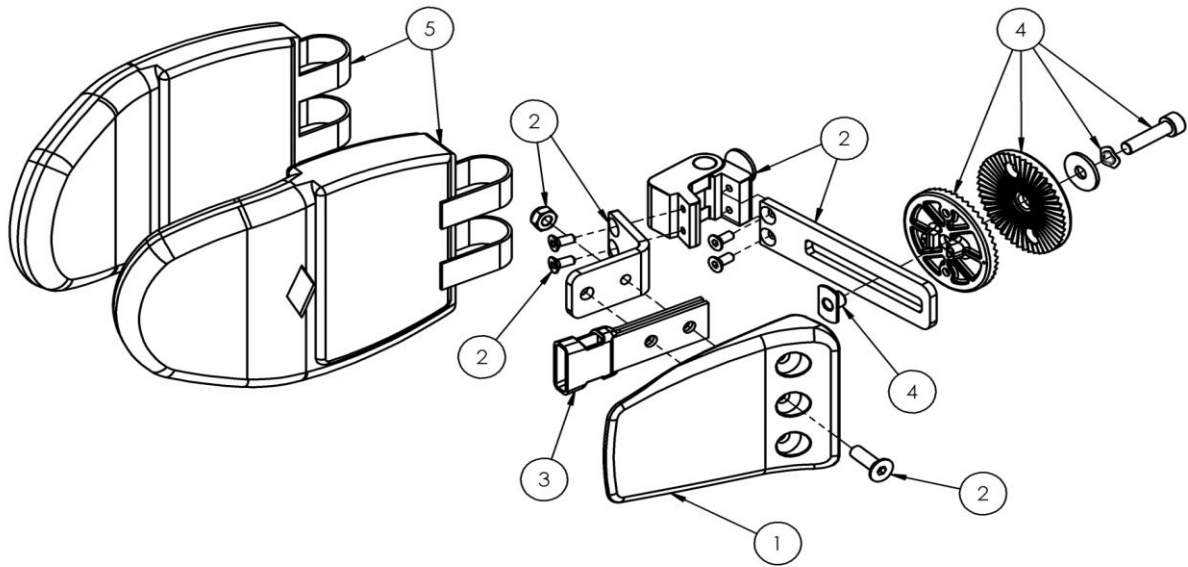

Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	120-855		Flipaway Lateral Support Hardware	1	✓
2	120-9035		Lateral Bracket & Screw Assembly	1	*
3	117-700		Harness Buckle Attachment	30	✓
4	120-9027		Angle Lock Assy	30	✓
5	120-838-01		Lateral Cover Set - Green	1	✓
	120-838-02		Lateral Cover Set - Orange	1	✓
	120-838-03		Lateral Cover Set - Blue	1	✓
	120-838-04		Lateral Cover Set - Pink	1	✓

LECKEY^o

Squiggles Seat Footplate Assembly 120-639

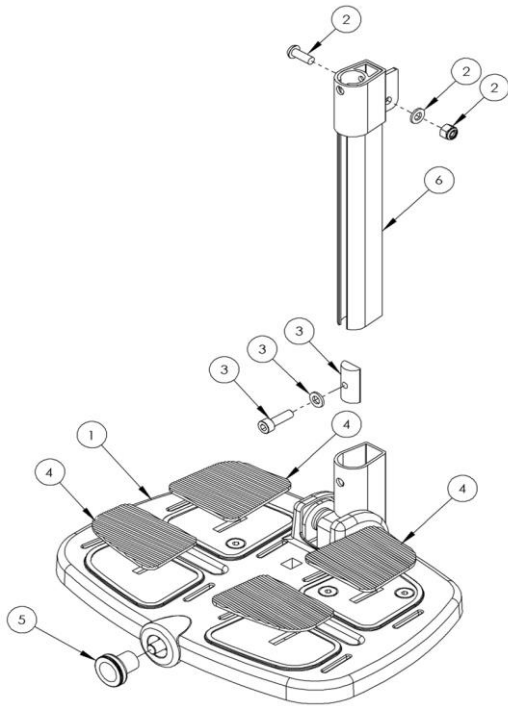


Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	120-639		Footplate Assembly	1	✓
2		SP-1019	Leg Extension Bolt Kit	50	✓
3		120-9015	Calf Locking Assy	20	✓
4	120-214		Footplate Pad Kit	1	✓
5		SF85	Elesa Knurled Knob M8	20	✓
6		120-700	Leg Extension Assy	1	✓

Order SI12, SI04, SJ94
Order 120-235, S112, SJ62

Order SF85

LECKEY^o

Squiggles Seat Hi-Low Chassis 120-800

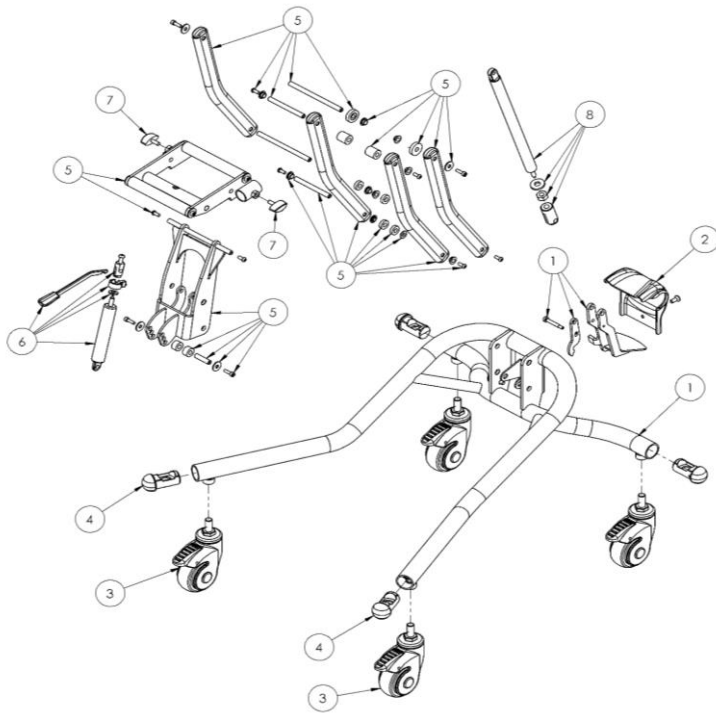


Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	120-800		Squiggles Chassis Assembly	1	✓
2		120-371	Chassis Rear Moulding	1	✓
3	IA34		Castor	1	✓
4		118-160	Castor Tube Insert	20	✓
5	120-802		Chassis Arm Assembly	1	✓
6	120-812		Bansback Tilt Spring Assembly	1	✓
7		SF19	Locking Key (M8 X 16)	50	✓
8	120-811		Lift Spring Assembly	1	✓

Order SJ220



Squiggles Seat Interface & Static Seatbase Assembly 120-666

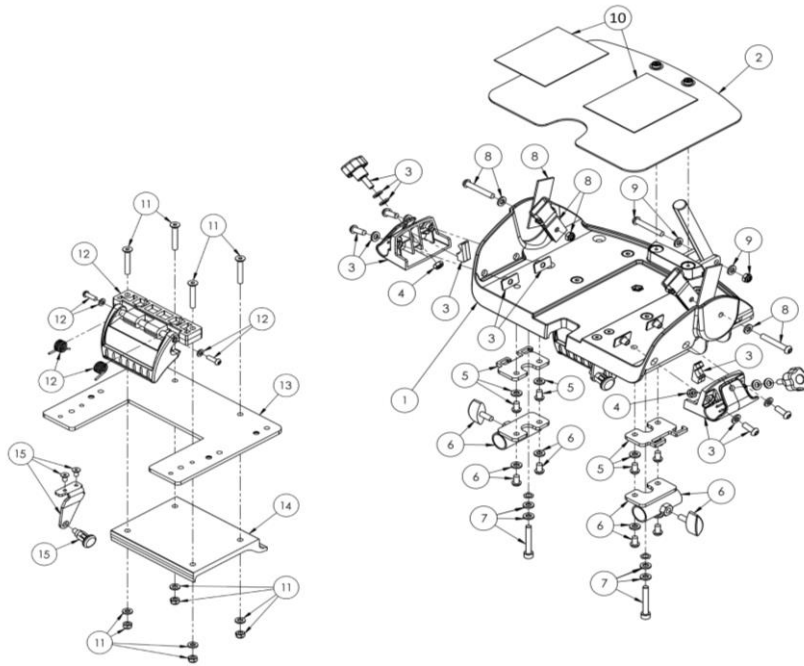

Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	120-666		Interface and Static Base Assembly	1	✓
2		120-851	Seatbase Shelf	1	✗
3		120-9006	Tray Holder Assembly	1	✓
4		120-9007	Tray Holder Knob Assembly	30	✓
5		120-9008	Long Sitting Spacer Plate Assembly	1	✗
6		120-9009	Long Sitting Receiver Bracket Assembly	1	✗
7		SP-1012	Sliding Seatbase Attachment Screw Kit	1	✗
8		120-9011	Seatbase & Strap Attachment Kit	1	✓
9		SP-1013	Back Angle Bar Screw Kit	50	✓
10		120-9013	Seatbase Shelf Velcro Kit	1	✗
11		SP-1018	Interface Attachment Screw Kit	50	✓
12		120-9016	Interface Handle Assy	1	✗
13		120-387	Interface Aluminium Plate	1	✗
14		120-326	Interface Plate	1	✗
15		120-9017	Interface Pin Assy	1	✗


Order 120-152, SI04, SL25, SL02, 120-246
Order SF81, SL25

117-749, 120-761, SJ92, SI12, SI04
Order SJ92, SI12, SI04

Order SI20, SI12, SI04

LECKEY^o

Squiggles Seat Protraction Pad Assembly 120-954

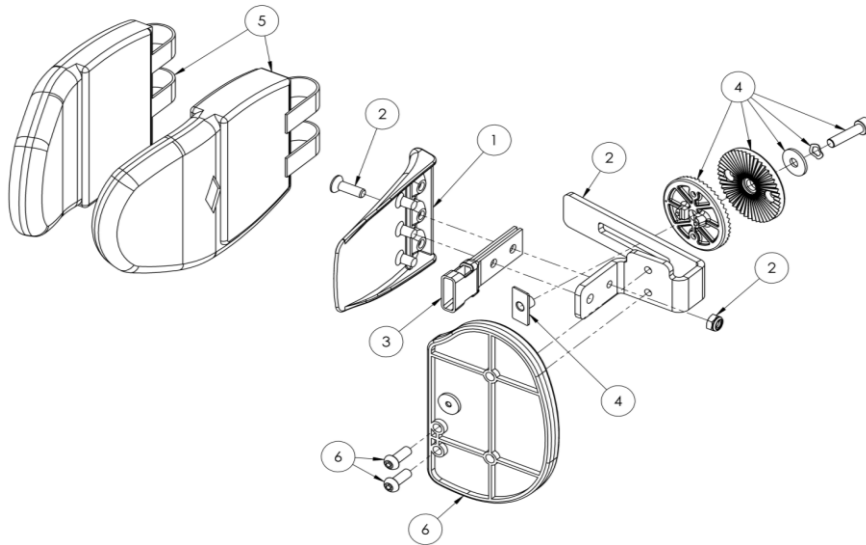


Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	118-049		Lateral	1	x
2		120-9056	Protraction Bracket Assy (RHS)	1	x
		120-9057	Protraction Bracket Assy (LHS)	1	x
3	117-700		Harness Buckle Attachment	1	✓
4	120-9027		Angle Lock Assy	1	✓
5	120-838-01		Lateral Cover Set - Green	1	x
	120-838-02		Lateral Cover Set - Orange	1	x
	120-838-03		Lateral Cover Set - Blue	1	x
	120-838-04		Lateral Cover Set - Pink	1	x
6		120-9058	Protraction Assy (with cover)	1	x

Order 117-138, SL36, SL66, SJ170, SJ17

LECKEY^o

Squiggles Seat Rigid Lateral Assembly 120-656

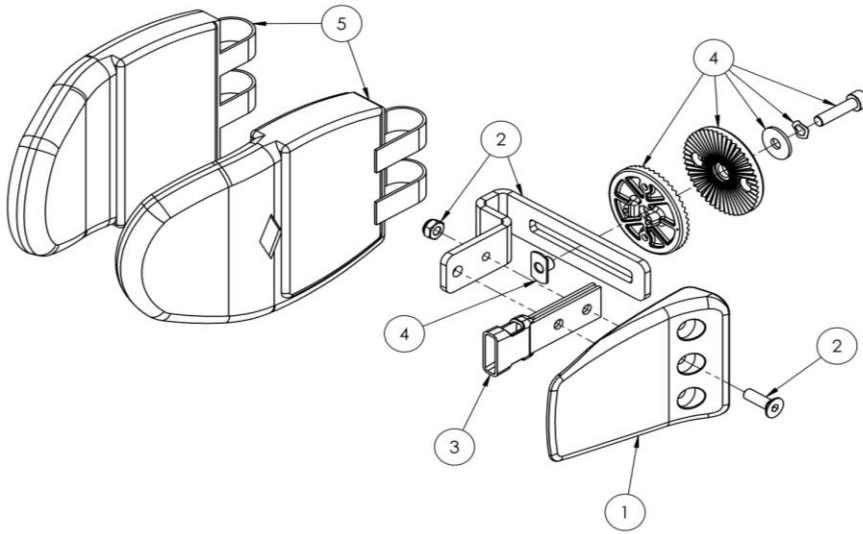


Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
	120-656		Top Level Assembly (No covers)		
1		118-049	Lateral	1	*
2		120-9032	Lateral Bracket & Screw Assembly	1	*
3		117-700	Harness Buckle Attachment	1	✓
4		120-9027	Angle Lock Assy	1	✓
5	120-838-01		Lateral Cover Set - Green	1	✓
	120-838-02		Lateral Cover Set - Orange	1	✓
	120-838-03		Lateral Cover Set - Blue	1	✓
	120-838-04		Lateral Cover Set - Pink	1	✓

Order 117-138, SL36, SL66, SJ170, SJ17

LECKEY^o

Squiggles Seat Shoulder Support Assembly 120-702

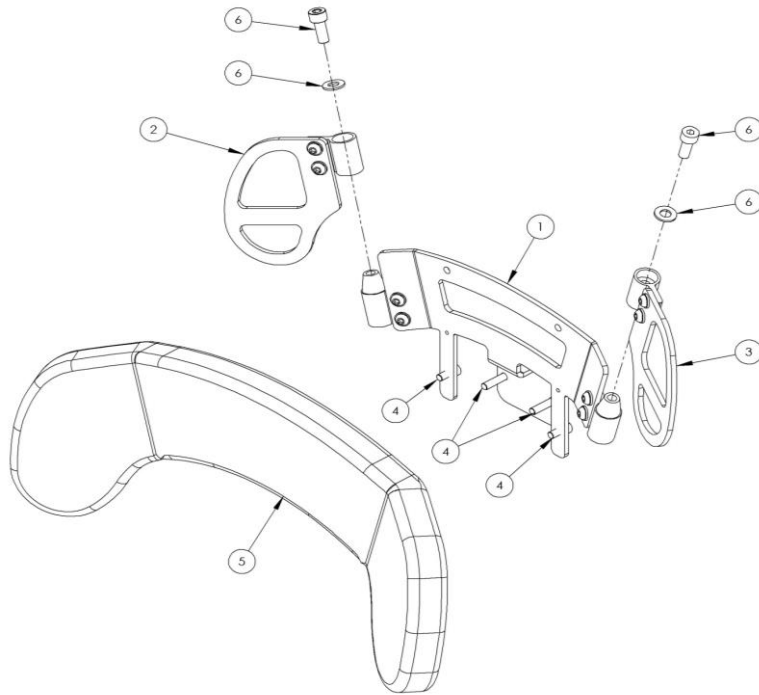


Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
	120-702		Top level assembly (No covers)		
1		120-9021	Shoulder Centre Plate Assembly	1	✗
2		120-9022	Shoulder Wing Plate Assembly RH	1	✗
3		120-9023	Shoulder Wing Plate Assembly LH	1	✗
4		SP-1015	Shoulder Attachment Bolt Kit	50	✓
5	120-828-01		Shoulder Cushion - Green	1	✗
	120-828-02		Shoulder Cushion - Orange	1	✗
	120-828-03		Shoulder Cushion - Blue	1	✗
	120-828-04		Shoulder Cushion - Pink	1	✗
6		SP-1016	Taperlock Adjustment Bolt Kit	50	✓

Order SJ17, SJ60

Order SJ98, SI14

LECKEY°


Squiggles Seat Sliding Seatbase Assembly - 120-634

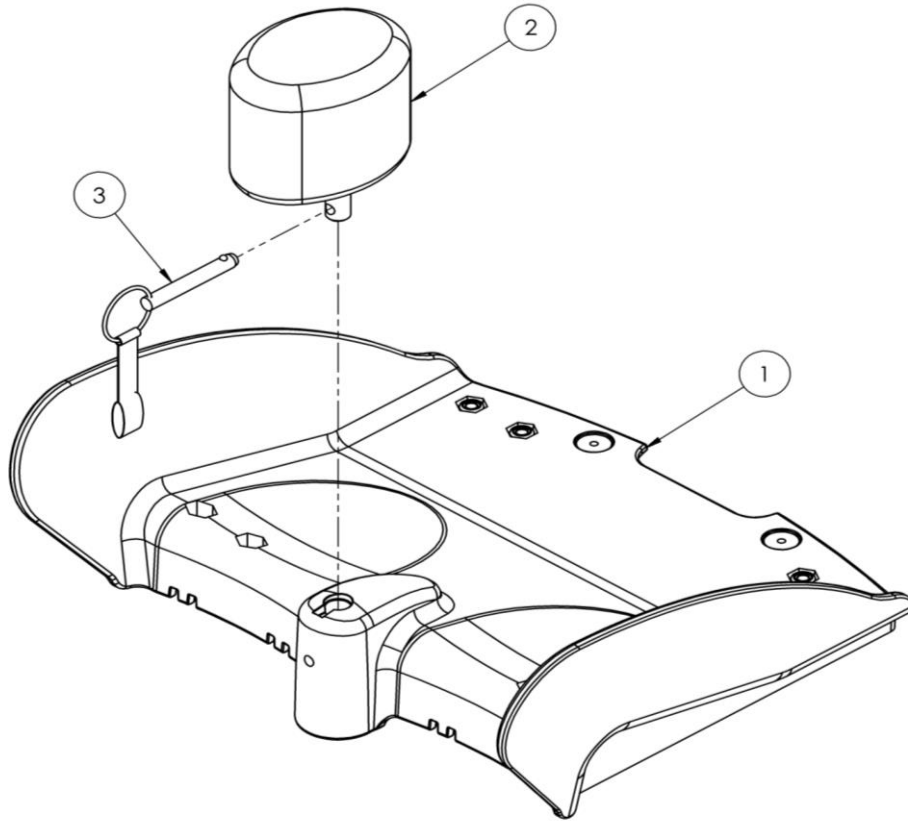

Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
1	120-634		Sliding SeatBase Assembly	1	✓
2	120-641		Pommel, Pin & Lanyard Complete Assembly	1	✓
3		120-162	Pin & Lanyard	1	✓

LECKEY^o

Squiggles Seat Tray Assembly 120-672 or 120-688 (extended)

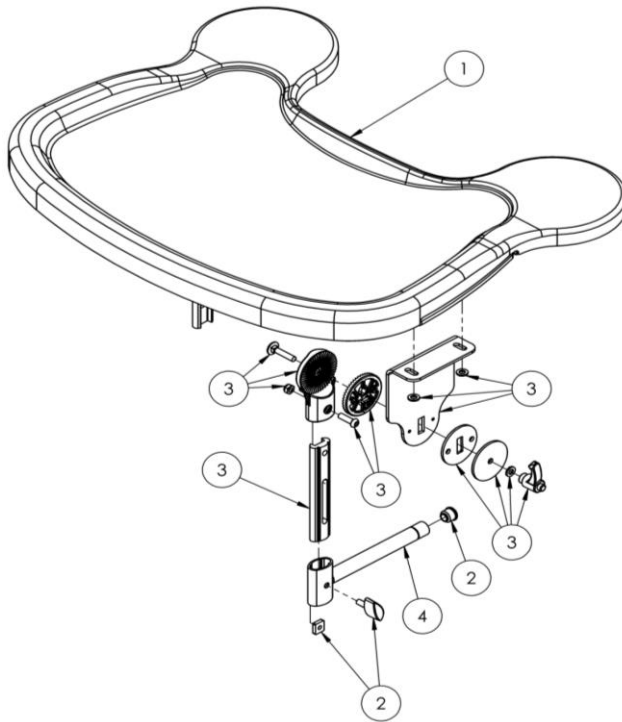

Pos.	Leckey Sales Code	Item Codes	Description	Min. Order Qty	Available for Sale
	120-672		Tray Assembly	1	✓
1	120-688		Ext. Height Tray Assembly	1	✓
2		120-9038	Tray Adjustment Kit	1	✓
3		120-671	Tray Height Adjustment Assembly	1	✗
4		120-638	Tray Depth Adjustment Assembly	1	✗

Order 117-268, SF48, SF51