




Leckey PAL Seat

Spare Parts Catalogue



Supporting
Children

Leckey PAL Seat Spare Parts - Pal Seat Assembly

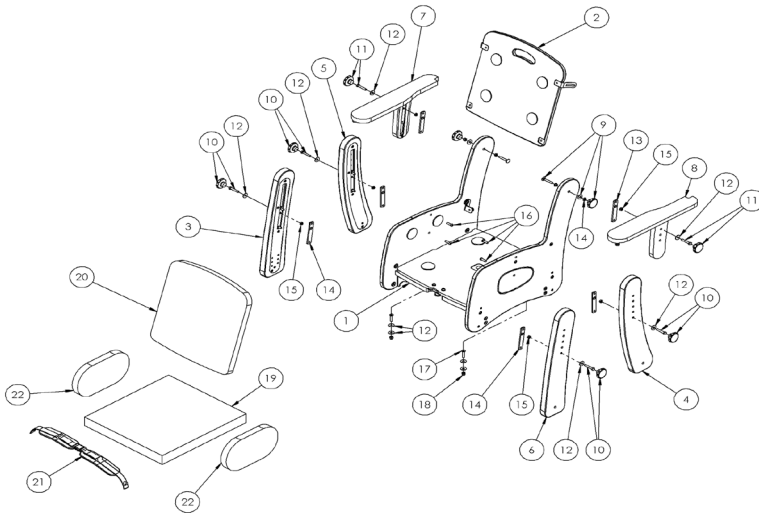

Pos.	Leckey Sales Code	Description
1	135-1615	Seat Base Assembly - Size 1 Green
1	135-1615-03	Seat Base Assembly - Size 1 Blue
1	135-2615-01	Seat Base Assembly - Size 2 Green
1	135-2615-03	Seat Base Assembly - Size 2 Blue
1	135-3615-01	Seat Base Assembly - Size 3 Green
1	135-3615-03	Seat Base Assembly - Size 3 Blue
1	135-4615-01	Seat Base Assembly - Size 4 Green
1	135-4615-03	Seat Base Assembly - Size 4 Blue
2	135-1614	Backrest Assembly - Size 1
2	135-2614	Backrest Assembly - Size 2
2	135-3614	Backrest Assembly - Size 3
2	135-4614	Backrest Assembly - Size 4
3	135-1611	Leg Front RH - Size 1
3	135-2611	Leg Front RH - Size 2
3	135-3611	Leg Front RH - Size 3
3	135-4611	Leg Front RH - Size 4


Leckey PAL Seat Spare Parts - Seat Assembly

Pos.	Leckey Sales Code	Description
4	135-1612	Leg Rear LH - Size 1
4	135-2612	Leg Rear LH - Size 2
4	135-3612	Leg Rear LH - Size 3
4	135-4612	Leg Rear LH - Size 4
5	135-1613	Leg Rear RH - Size 1
5	135-2613	Leg Rear RH - Size 2
5	135-3613	Leg Rear RH - Size 3
5	135-4613	Leg Rear RH - Size 4
6	135-1618	Leg Front LH - Size 1
6	135-2618	Leg Front LH - Size 2
6	135-3618	Leg Front LH - Size 3
6	135-4618	Leg Front LH - Size 4
7	135-1616	Armrest RH - Size 1
7	135-2616	Armrest RH - Size 2
7	135-3616	Armrest RH - Size 3
7	135-4616	Armrest RH - Size 4
8	135-1617	Armrest LH - Size 1
8	135-2617	Armrest LH - Size 2
8	135-3617	Armrest LH - Size 3
8	135-4617	Armrest LH - Size 4
9	135-9011	Backrest Handle Knob Kit
10	135-9009	Legrest Handle Knob Kit
11	135-9012	Armrest Handle Knob Kit
12	SI09 M6	Mudguard Washer
13	135-1194	Gable to Arm Plate - Size 1, 2 & 3
13	135-4113	Gable to Arm Plate - Size 4
14	135-1113	Gable to Leg Plate - Size 1, 2 & 3
14	135-4113	Gable to Leg Plate - Size 4
15	SI04	M6 Nyloc Slim Nut
16	SJ13	M6 x 20 Socket Button
17	SJ19	M6 x 30 CSK Socket Screw
18	SJ202	M6 Dome Nut

Leckey PAL Seat Spare Parts - **Seat Assembly**

Pos.	Leckey Sales Code	Description
19	135-1650-01	Seat Cushion - Size 1 Green
19	135-1650-03	Seat Cushion - Size 1 Blue
19	135-2650-01	Seat Cushion - Size 2 Green
19	135-2650-03	Seat Cushion - Size 2 Blue
19	135-3650-01	Seat Cushion - Size 3 Green
19	135-3650-03	Seat Cushion - Size 3 Blue
19	135-4650-01	Seat Cushion - Size 4 Green
19	135-4650-03	Seat Cushion - Size 4 Blue
20	135-1655-01	Backrest Cushion - Size 1 Green
20	135-1655-03	Backrest Cushion - Size 1 Blue
20	135-2655-01	Backrest Cushion - Size 2 Green
20	135-2655-03	Backrest Cushion - Size 2 Blue
20	135-3655-01	Backrest Cushion - Size 3 Green
20	135-3655-03	Backrest Cushion - Size 3 Blue
20	135-4655-01	Backrest Cushion - Size 4 Green
20	135-4655-03	Backrest Cushion - Size 4 Blue
21	135-1657	Lap Belt - Size 1
21	135-2657	Lap Belt - Size 2
21	135-3657	Lap Belt - Size 3
21	135-4657	Lap Belt - Size 4
22	135-1635-01	Side Pad (Pair) 25mm - Size 1 Green
22	135-1635-03	Side Pad (Pair) 25mm - Size 1 Blue
22	135-2635-01	Side Pad (Pair) 25mm - Size 2 Green
22	135-2635-03	Side Pad (Pair) 25mm - Size 2 Blue
22	135-3635-01	Side Pad (Pair) 25mm - Size 3 Green
22	135-3635-03	Side Pad (Pair) 25mm - Size 3 Blue
22	135-4635-01	Side Pad (Pair) 25mm - Size 4 Green
22	135-4635-03	Side Pad (Pair) 25mm - Size 4 Blue

Leckey PAL Seat Spare Parts - Seat Base Assembly Size 1 - 3

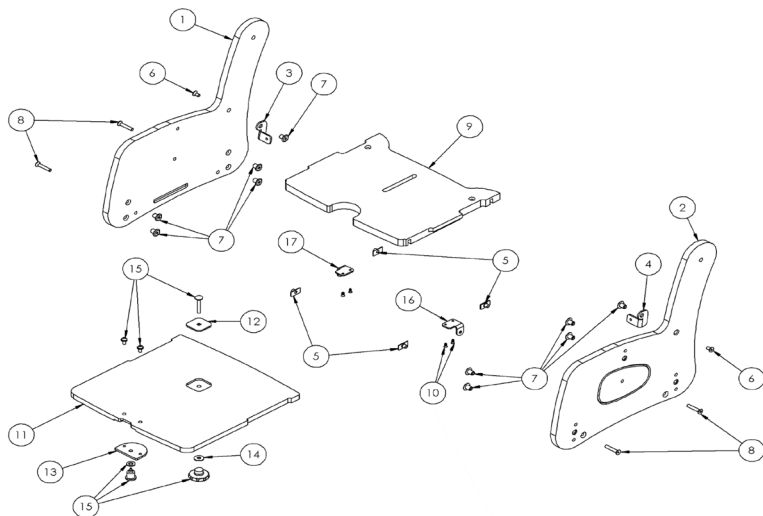


Pos.	Leckey Sales Code	Description
1	135-1186-01	Gable RH - Size 1 Green
1	135-1186-03	Gable RH - Size 1 Blue
1	135-2186-01	Gable RH - Size 2 Green
1	135-2186-03	Gable RH - Size 2 Blue
1	135-3186-01	Gable RH - Size 3 Green
1	135-3186-03	Gable RH - Size 3 Blue
2	135-1187-01	Gable LH - Size 1 Green
2	135-1187-03	Gable LH - Size 1 Blue
2	135-2187-01	Gable LH - Size 2 Green
2	135-2187-03	Gable LH - Size 2 Blue
2	135-3187-01	Gable LH - Size 3 Green
2	135-3187-03	Gable LH - Size 3 Blue
3	135-1114	Backrest Pivot Plate RH
4	135-1115	Backrest Pivot Plate LH
5	SJ170	M6 T Nut
6	SJ117	M6 x 12 CSK Socket Screw

Leckey PAL Seat Spare Parts - Seat Base Assembly Size 1 - 3


Pos.	Leckey Sales Code	Description
7	SL23	M6 Furniture Connector
8	SJ21	M6 x 35 CSK Socket Screw
9	135-1106	Static Seat Base - Size 1
9	135-2106	Static Seat Base - Size 2
9	135-3106	Static Seat Base - Size 3
10	SA16	3/4" X 6 CSK SCRULOX
11	135-1108	Sliding Seat Base - Size 1
11	135-2108	Sliding Seat Base - Size 2
11	135-3108	Sliding Seat Base - Size 3
12	135-2139	Cup Square Retaining Plate
13	135-1125	Pommel Plate
14	SI09 M6	Mudguard Washer
15	135-9000	Sliding Seatbase Knob & Screw Kit
16	135-1128	S1 Rigidity Bracket LH
17	135-1129	S1 Rigidity Bracket RH

Leckey PAL Seat Spare Parts - Seat Base Assembly Size 4 Only

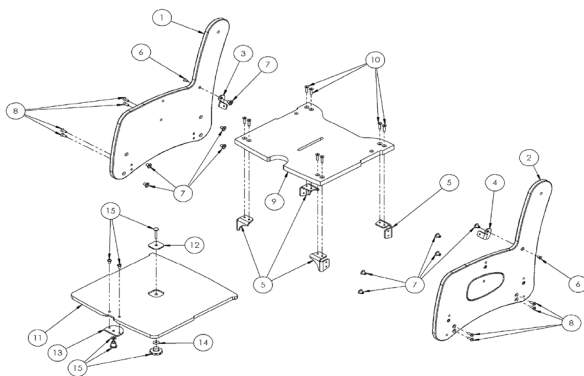

Pos.	Leckey Sales Code	Description
1	135-4186-01	Gable RH - Size 4 Green
1	135-4186-02	Gable RH - Size 4 Orange
1	135-4186-03	Gable RH - Size 4 Blue
1	135-4186-04	Gable RH - Size 4 Pink
2	135-4187-01	Gable LH - Size 4 Green
2	135-4187-02	Gable LH - Size 4 Orange
2	135-4187-03	Gable LH - Size 4 Blue
2	135-4187-04	Gable LH - Size 4 Pink
3	135-1114	Backrest Pivot Plate RH
4	135-1115	Backrest Pivot Plate LH
5	135-4185	Seat to Gable Bracket
6	SJ117	M6 x 12 CSK Socket Screw
7	SL23	M6 Furniture Connector
8	SJ18	M6 x 18 CSK Socket Screw
9	135-4106	Static Seat Base - Size 4
10	SJ06	M6 x 25 CSK Socket Screw
11	135-4108	Sliding Seat Base
12	135-2139	Cup Square Retaining Plate
13	135-1125	Pommel Plate
14	SI09	M6 Mudguard Washer
15	135-9000	Sliding Seatbase Knob & Screw Kit

Leckey PAL Seat Spare Parts - Accessories

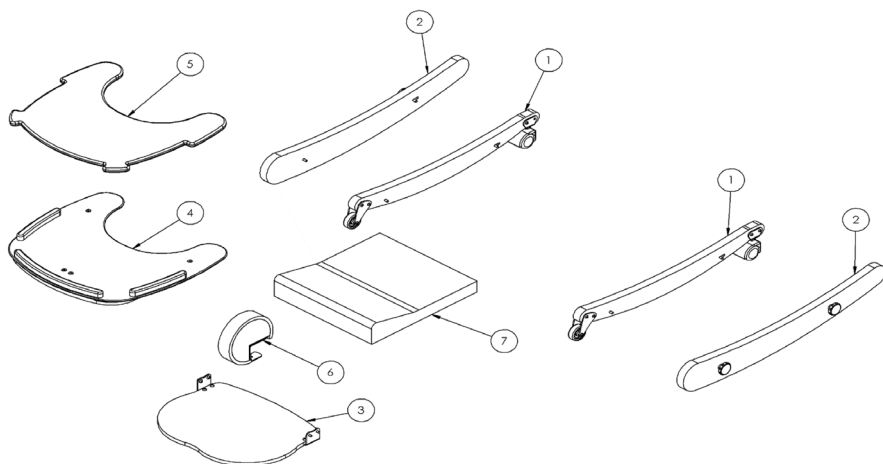


Pos.	Leckey Sales Code	Description
1	135-1610	Ski with Castors (Pair) - Size 1
1	135-2610	Ski with Castors (Pair) - Size 2
1	135-3610	Ski with Castors (Pair) - Size 3
1	135-4610	Ski with Castors (Pair) - Size 4
2	135-1604	Rocker/Stabiliser (Pair) - Size 1
2	135-2604	Rocker/Stabiliser (Pair) - Size 2
2	135-3604	Rocker/Stabiliser (Pair) - Size 3
2	135-4604	Rocker/Stabiliser (Pair) - Size 4
3	135-1603	Footrest - Size 1
3	135-2603	Footrest - Size 2
3	135-3603	Footrest - Size 3
3	135-4603	Footrest - Size 4
4	135-1620	Depth Adjustable Activity Tray - Size 1
4	135-2620	Depth Adjustable Activity Tray - Size 2
4	135-3620	Depth Adjustable Activity Tray - Size 3
4	135-4620	Depth Adjustable Activity Tray - Size 4
5	135-1656	Tray Pad - Size 1
5	135-2656	Tray Pad - Size 2
5	135-3656	Tray Pad - Size 3
5	135-4656	Tray Pad - Size 4

Leckey PAL Seat Spare Parts - Accessories

Pos.	Leckey Sales Code	Description
6	135-1602-01	Pommel Green - Size 1
6	135-1602-03	Pommel Blue - Size 1
6	135-3602-01	Pommel Green - Size 2,3 & 4
6	135-3602-03	Pommel Blue - Size 2,3 & 4
7	135-1651-01	Ramped Seat Cushion - Size 1 Green
7	135-1651-03	Ramped Seat Cushion - Size 1 Blue
7	135-2651-01	Ramped Seat Cushion - Size 2 Green
7	135-2651-03	Ramped Seat Cushion - Size 2 Blue
7	135-3651-01	Ramped Seat Cushion - Size 3 Green
7	135-3651-03	Ramped Seat Cushion - Size 3 Blue
7	135-4651-01	Ramped Seat Cushion - Size 4 Green
7	135-4651-03	Ramped Seat Cushion - Size 4 Blue

Leckey PAL Seat Spare Parts - **Armrest Assembly**

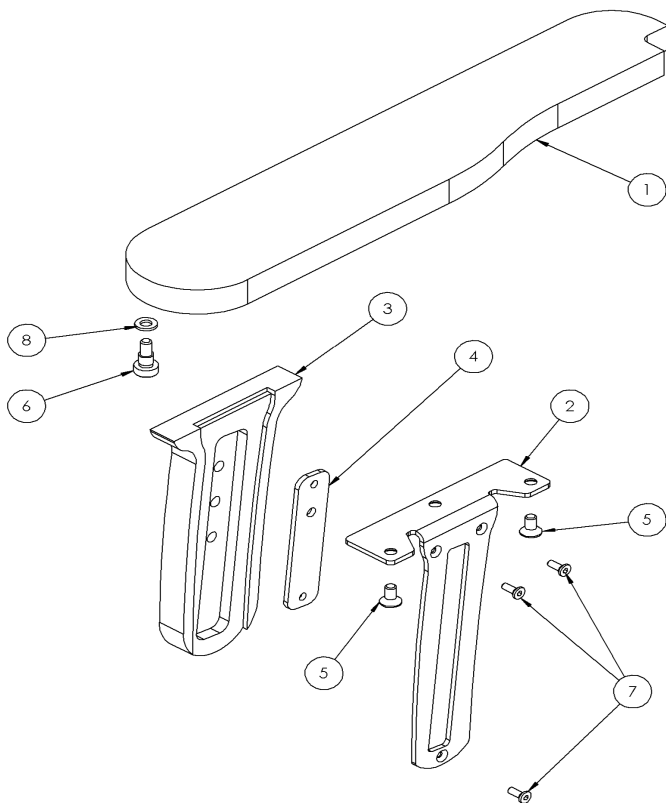


Pos.	Leckey Sales Code	Description
1	135-1188	Armrest RH - Size 1
1	135-2188	Armrest RH - Size 2
1	135-3188	Armrest RH - Size 3
1	135-4188	Armrest RH - Size 4
1	135-1189	Armrest LH - Size 1
1	135-2189	Armrest LH - Size 2
1	135-3189	Armrest LH - Size 3
1	135-4189	Armrest LH - Size 4

Leckey PAL Seat Spare Parts - **Armrest Assembly**


Pos.	Leckey Sales Code	Description
2	135-1192	Arm Bracket RH - Size 1 & 2
2	135-3192	Arm Bracket RH - Size 3 & 4
2	135-1193	Arm Bracket LH - Size 1 & 2
2	135-3193	Arm Bracket LH - Size 3 & 4
3	135-1190	Shroud RH - Size 1 & 2
3	135-3190	Shroud RH - Size 3 & 4
3	135-1191	Shroud LH - Size 1 & 2
3	135-3191	Shroud LH - Size 3 & 4
4	135-1195	Retaining Plate - Size 1 & 2
4	135-4126	Retaining Plate - Size 3 & 4
5	SJ117	M6 x 12 CSK Screw
6	SJ233	M6 x 8 x 8 Shoulder Screw
7	SA16	1/2" x 6 Scrulox
8	SI12	M6 Washer

Leckey PAL Seat Spare Parts - **Backrest Assembly**

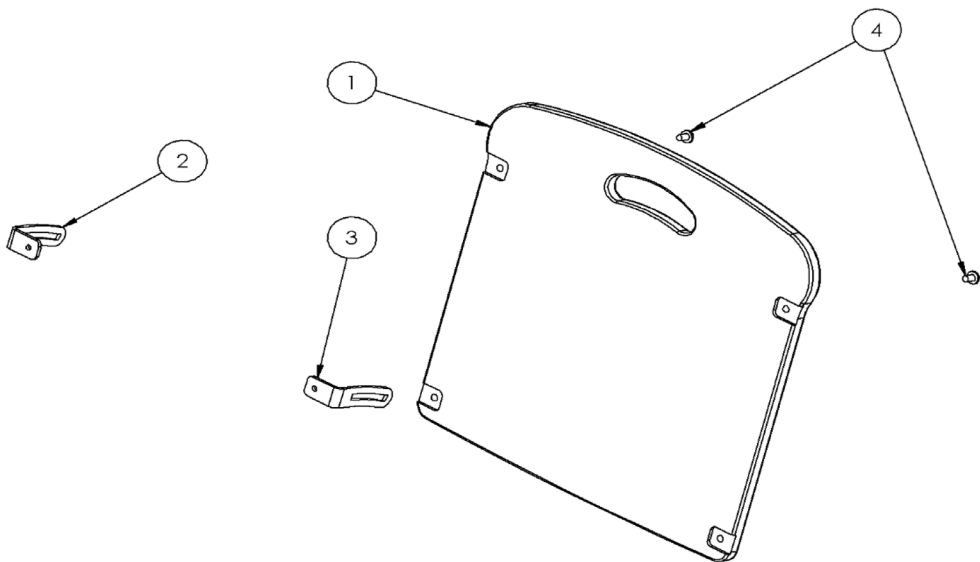


Pos.	Leckey Sales Code	Description
1	135-1107	Backrest - Size 1
1	135-2107	Backrest - Size 2
1	135-3107	Backrest - Size 3
1	135-4107	Backrest - Size 4
2	135-1111	Angle Bracket RH - Size 1
2	135-2111	Angle Bracket RH - Size 2
2	135-3111	Angle Bracket RH - Size 3
2	135-4111	Angle Bracket RH - Size 4
3	135-1110	Angle Bracket LH - Size 1
3	135-2110	Angle Bracket LH - Size 2
3	135-3110	Angle Bracket LH - Size 3
3	135-4110	Angle Bracket LH - Size 4
4	SJ201	M6 x 12 Flanged Head Socket

Leckey PAL Seat Spare Parts - Foot Plate Assembly

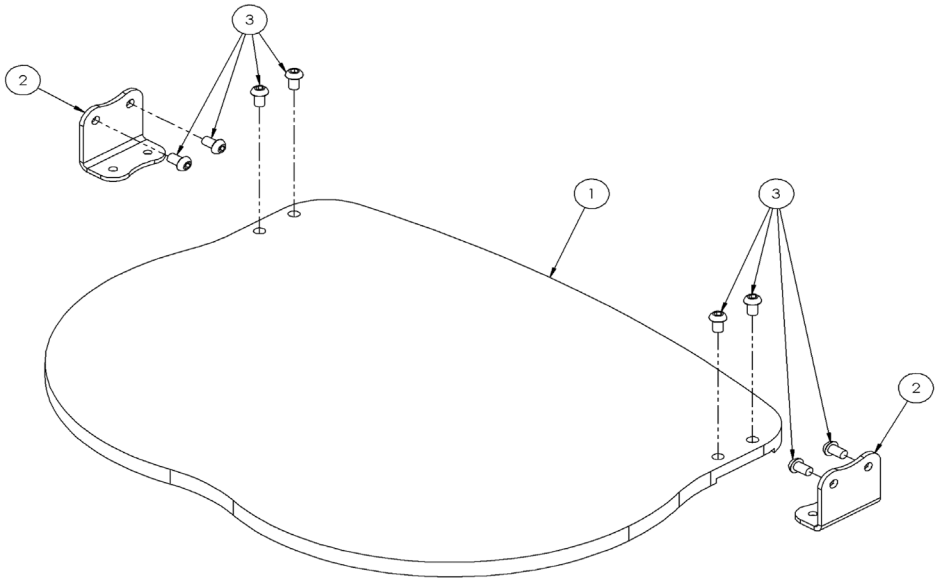

Pos.	Leckey Sales Code	Description
1	135-1109	Footplate - Size 1
1	135-2109	Footplate - Size 2
1	135-3109	Footplate - Size 3
1	135-4109	Footplate - Size 4
2	135-1112	Footplate Bracket
3	SJ201	M6 x 12 Flanged Head Button

Leckey PAL Seat Spare Parts - Front Legrest Assembly

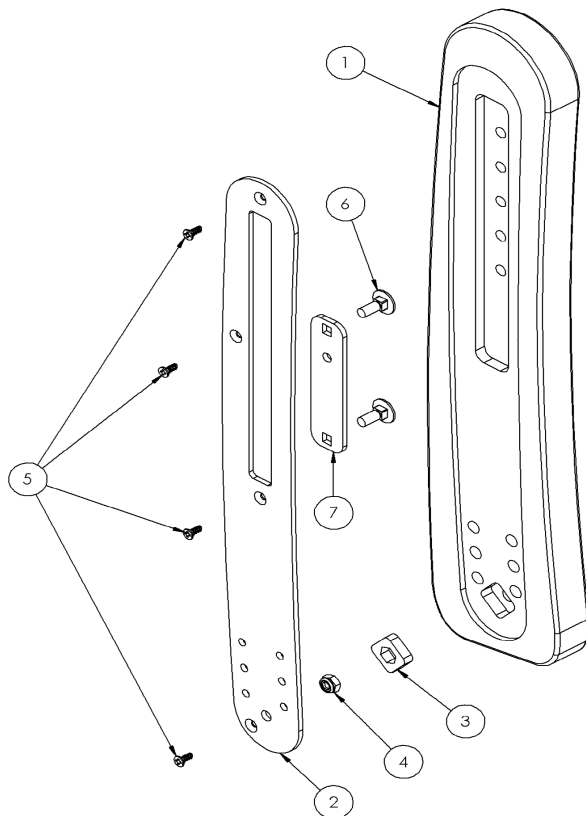


Pos.	Leckey Sales Code	Description
1	135-1102	Front Leg RH - Size 1
1	135-2102	Front Leg RH - Size 2
1	135-3102	Front Leg RH - Size 3
1	135-4102	Front Leg RH - Size 4
1	135-1103	Front Leg LH - Size 1
1	135-2103	Front Leg LH - Size 2
1	135-3103	Front Leg LH - Size 3
1	135-4103	Front Leg LH - Size 4

Leckey PAL Seat Spare Parts - Front Legrest Assembly

Pos.	Leckey Sales Code	Description
2	135-1116	Front Inner Plate RH - Size 1
2	135-2116	Front Inner Plate RH - Size 2
2	135-3116	Front Inner Plate RH - Size 3
2	135-4116	Front Inner Plate RH - Size 4
2	135-1141	Front Inner Plate LH - Size 1
2	135-2141	Front Inner Plate LH - Size 2
2	135-3141	Front Inner Plate LH - Size 3
2	135-4141	Front Inner Plate LH - Size 4
3	135-1130	Leg Nut Retaining Plate
4	S104	M6 Nyloc Slim Nut
5	SA18	3/4" x 6 Scrulox
6	SJ315	M6 x 16 Cup Square Bolt
7	135-1131	Leg Retaining Plate - Size 1, 2 & 3
7	135-4131	Leg Retaining Plate - Size 4

Leckey PAL Seat Spare Parts - Rear Legrest Assembly

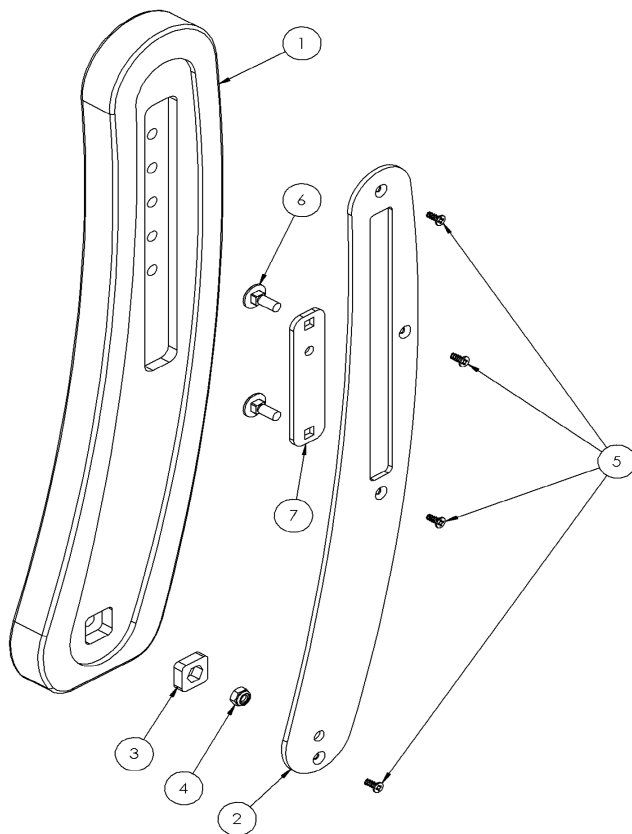


Pos.	Leckey Sales Code	Description
1	135-1104	Rear Leg RH - Size 1
1	135-2104	Rear Leg RH - Size 2
1	135-3104	Rear Leg RH - Size 3
1	135-4104	Rear Leg RH - Size 4
1	135-1105	Rear Leg LH - Size 1
1	135-2105	Rear Leg LH - Size 2
1	135-3105	Rear Leg LH - Size 3
1	135-4105	Rear Leg LH - Size 4

Leckey PAL Seat Spare Parts - Front Legrest Assembly


Pos.	Leckey Sales Code	Description
2	135-1117	Rear Inner Plate RH - Size 1
2	135-2117	Rear Inner Plate RH - Size 2
2	135-3117	Rear Inner Plate RH - Size 3
2	135-4117	Rear Inner Plate RH - Size 4
2	135-1142	Rear Inner Plate LH - Size 1
2	135-2142	Rear Inner Plate LH - Size 2
2	135-3142	Rear Inner Plate LH - Size 3
2	135-4142	Rear Inner Plate LH - Size 4
3	135-1130	Leg Nut Retaining Plate
4	SI04	M6 Nyloc Slim Nut
5	SA18	3/4" x 6 Scrulox
6	SJ315	M6 x 16 Cup Square Bolt
7	135-1131	Leg Retaining Plate - Size 1, 2 & 3
7	135-4131	Leg Retaining Plate - Size 4

Leckey PAL Seat Spare Parts - **Rocker/Stabiliser Assembly**

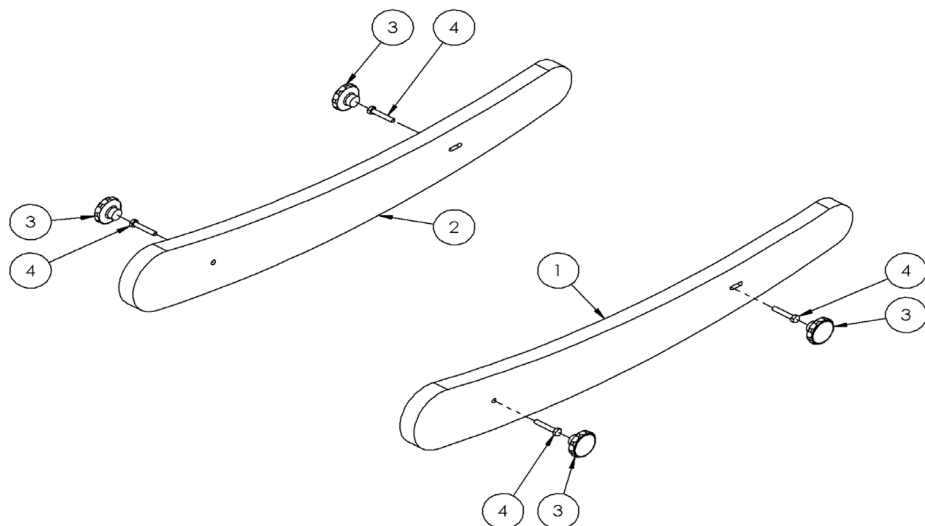

Pos.	Leckey Sales Code	Description
1	135-1120	Stabiliser RH/Rocker LH - Size 1
1	135-2120	Stabiliser RH/Rocker LH - Size 2
1	135-3120	Stabiliser RH/Rocker LH - Size 3
1	135-4120	Stabiliser RH/Rocker LH - Size 4
2	135-1121	Stabiliser LH/Rocker RH - Size 1
2	135-2121	Stabiliser LH/Rocker RH - Size 2
2	135-3121	Stabiliser LH/Rocker RH - Size 3
2	135-4121	Stabiliser LH/Rocker RH - Size 4
3	SF90	M6 Wixroyd Knob
4	SJ309	M6 x 45 Hex Head Screw

Leckey PAL Seat Spare Parts - Tray Assembly

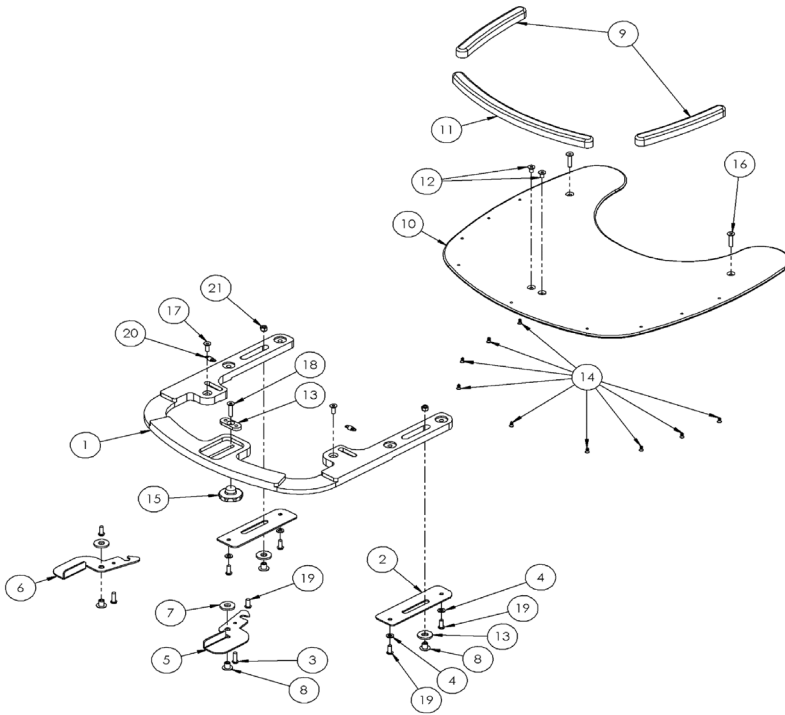


Pos.	Leckey Sales Code	Description
1	135-1127	Tray Horse Shoe - Size 1
1	135-2127	Tray Horse Shoe - Size 2
1	135-3127	Tray Horse Shoe - Size 3
1	135-4127	Tray Horse Shoe - Size 4
2	135-1136	Tray Guide Plate
3	SJ13	M6 x 20 Button Head
4	SI12	M6 Standard Washer
5	135-1132	Tray Latch LH
6	135-1133	Tray Latch RH
7	SL46	Nylon Washer

Leckey PAL Seat Spare Parts - Tray Assembly

Pos.	Leckey Sales Code	Description
8	SL23	Furniture Connector
9	135-1138	Tray Trim Side - Size 1
9	135-2138	Tray Trim Side - Size 2
9	135-3138	Tray Trim Side - Size 3
9	135-4138	Tray Trim Side - Size 4
10	135-1135	Tray Top - Size 1
10	135-2135	Tray Top - Size 2
10	135-3135	Tray Top - Size 3
10	135-4135	Tray Top - Size 4
11	135-1137	Tray Trim Front - Size 1
11	135-2137	Tray Trim Front - Size 2
11	135-3137	Tray Trim Front - Size 3
11	135-4137	Tray Trim Front - Size 4
12	SJ211	M6 x 12 CSK Socket Screw
13	135-1162	Tray Thread Plate
14	SA18	3/4" x 6 Scrulox
15	SF90	Wixroyd Knob M6
16	SJ216	M6 x 25 CSK Socket Screw Black
17	SJ18	M6 x 18 CSK Socket Screw
18	SJ19	M6 x 30 CSK Socket Screw
19	SJ94	M6 x 16 Button Head
20	SH63	Extension Spring
21	SI03	M6 Myloc Nut

Leckey PAL Seat Spare Parts - Wheel Ski Assembly

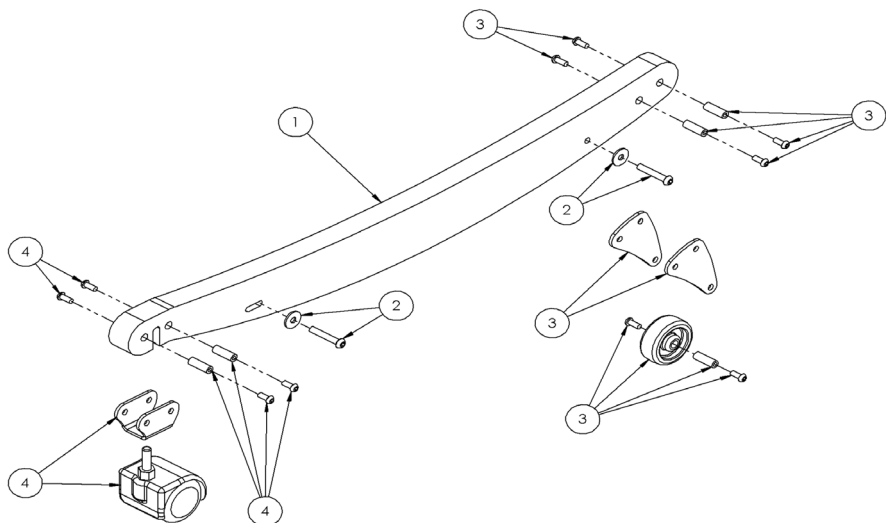

Pos.	Leckey Sales Code	Description
1	135-1610	Ski with Castors - Size 1
1	135-2610	Ski with Castors - Size 2
1	135-3610	Ski with Castors - Size 3
1	135-4610	Ski with Castors - Size 4
2	SP-1043	Ski Attachment Screw Kit
3	135-9004	Front Castor Assembly
4	135-9005	Rear Castor Assembly



Leckey
19c Ballinderry Road
Lisburn BT28 2SA
Northern Ireland

{+44} 28 9260 0750
hello@leckey.com

LS834-01

leckey.com

