



Squiggles+

Spare Parts Catalogue




Supporting
Children

Squiggles+ Spare Parts - Covers




Pos.	Leckey Sales Code	Description	Colour Options			
[Replace XX with colour]			Blue	Purple	Red	Red Vinyl
1	184-1800-XX	Full Cover Pack	-02	-04	-09	-09VIN
3	184-1820-XX	Knee Support [Pair]	-02	-04	-09	-09VIN
3	184-2820-XX	Knee Support [Pair] Size 2	-02	-04	-09	-09VIN
4	184-1822-XX	Support Harness	-02	-04	-09	-09VIN
5	184-1824-XX	Chest Cushion	-02	-04	-09	-09VIN
6	184-1826-XX	Hip Cushion	-02	-04	-09	-09VIN
7	184-1828-XX	Chest Lateral Covers	-02	-04	-09	-09VIN
8	184-1836-XX	Pelvic Support Harness	-02	-04	-09	-09VIN
9	184-1838-XX	Spine Cap Cover	-02	-04	-09	-09VIN

Squiggles+ Spare Parts - Handlebar

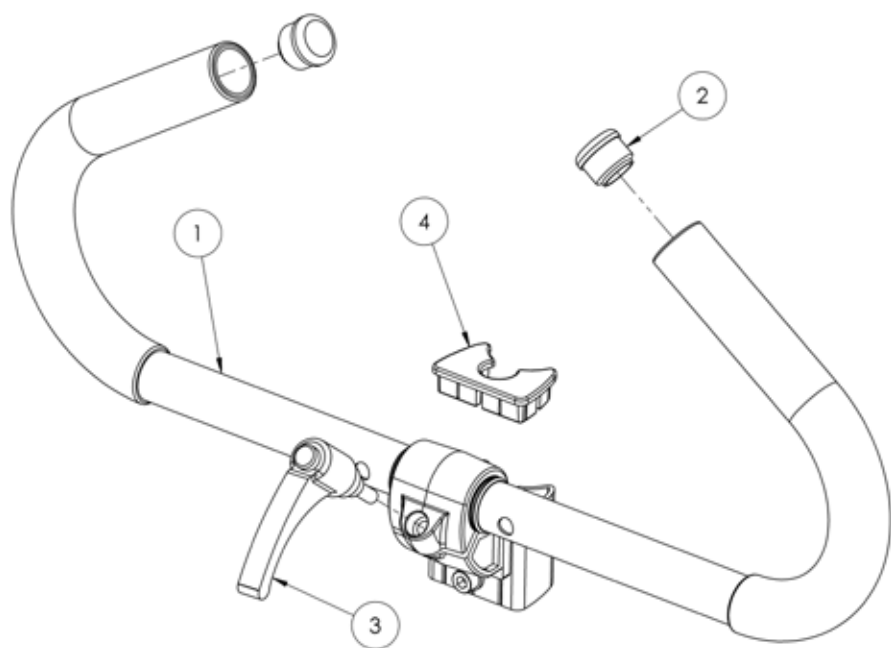


Pos.	Leckey Sales Code	Description
1	184-621	Push Handle Assembly
2	SF53	Round Tube Insert [Single]
3	SF95	Adjustable Handle
4	118-154	Extrusion Outer Cap

Squiggles+ Spare Parts - **Outer Spine**

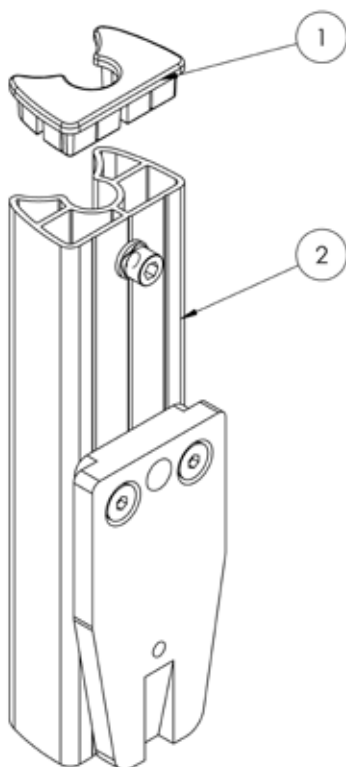


Pos.	Lecky Sales Code	Description
1	118-154	Extrusion Outer Cap
2	184-610	Outer Spine Assembly

Squiggles+ Spare Parts - Inner Spine

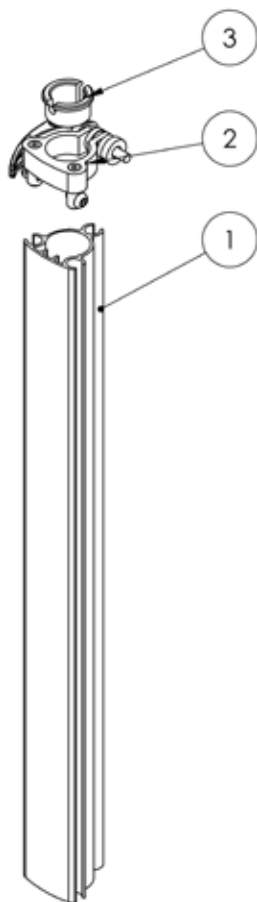


Pos.	Leckey Sales Code	Description
1	184-611	Inner Spine Assembly
2	184-712	Head Support Clamp Assembly SQ+
3	118-166	Head Support Collar

Squiggles+ Spare Parts - Chest Pad and Orb

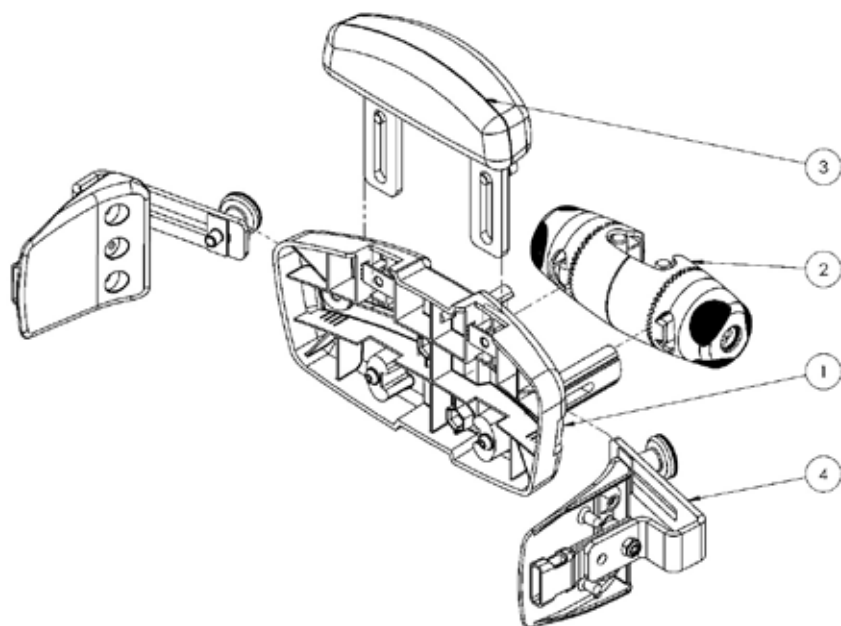


Pos.	Leckey Sales Code	Description
1	118-663	Chest Pad Assembly
2	118-680	Orb Assembly
3	118-664	Sternum Pad
4	118-666	Lateral Assembly [Single]

Squiggles+ Spare Parts - Hip Pad and Orb

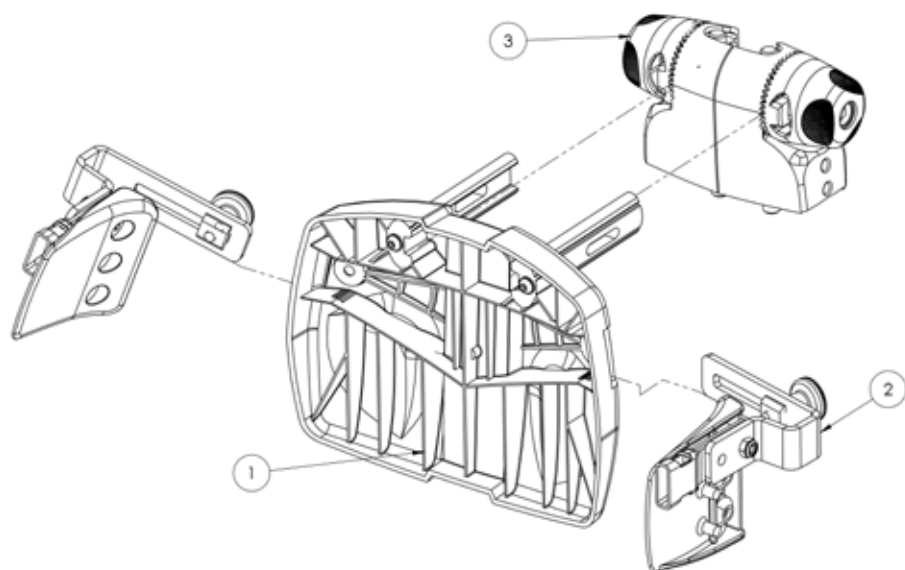


Pos.	Leckey Sales Code	Description
1	184-612	Hip Pad Assembly
2	118-666	Lateral Assembly [Single]
3	184-660	Orb Assembly

Squiggles+ Spare Parts - Knee Support

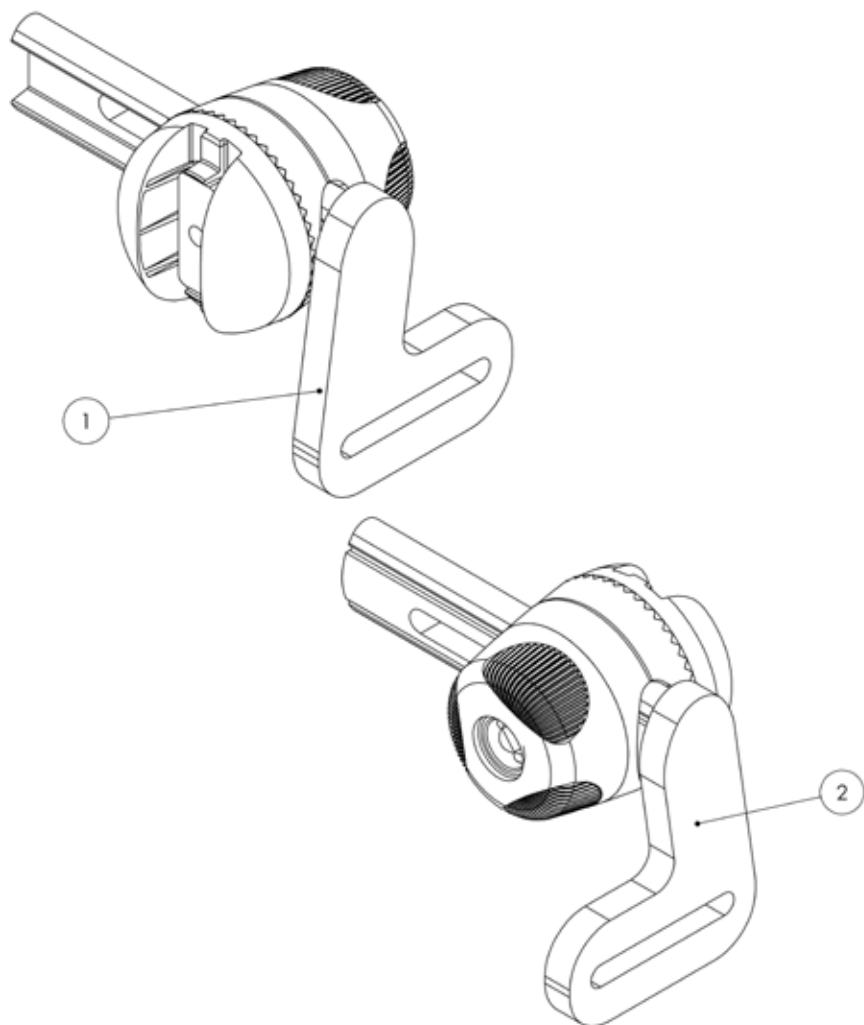


Pos.	Leckey Sales Code	Description
1	184-613-LHS	Knee Block Assembly Left
2	184-613-RHS	Knee Block Assembly Right

Squiggles+ Spare Parts - Footrest

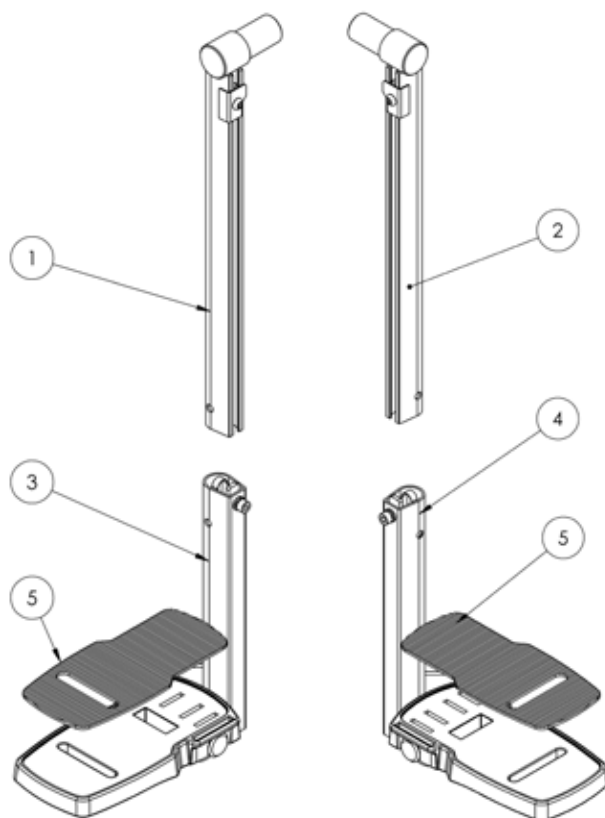


Pos.	Leckey Sales Code	Description
1	184-692	Calf Inner Support Assembly RH
2	184-693	Calf Inner Support Assembly LH
3	184-654	Calf Outer Support Assembly RH
4	184-655	Calf Outer Support Assembly LH
5	117-156	Footplate Rubber Matting (One Piece Only)

Squiggles+ Spare Parts - **Pivot Chassis**

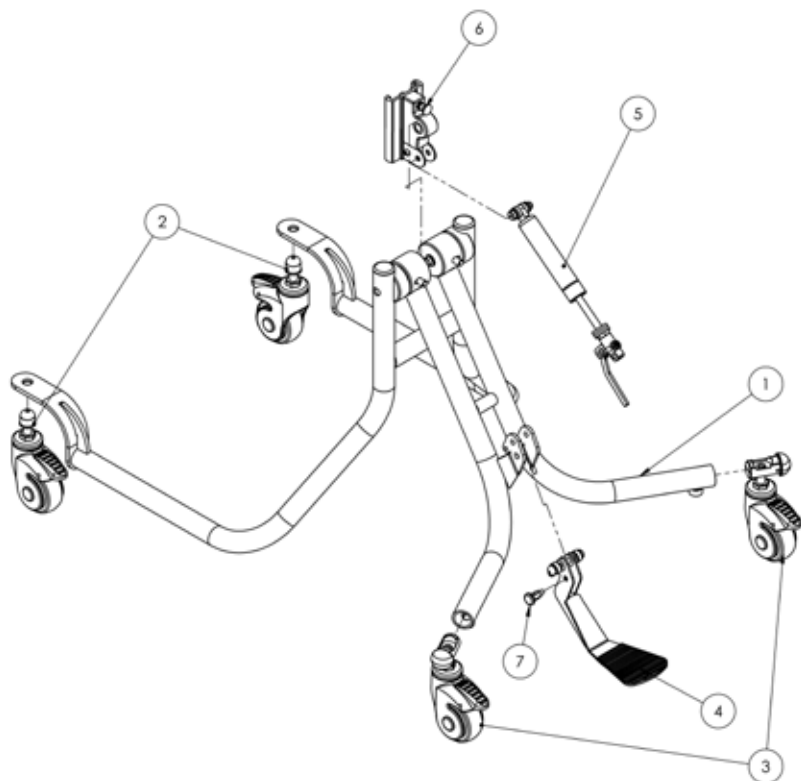


Pos.	Leckey Sales Code	Description
1	188-764	Pivot Chassis
2	184-715	Castor Assembly - Front [Pair]
3	184-716	Castor Assembly - Rear [Pair]
4	184-625	Pedal Assembly
5	188-717	Gas Spring Assembly
6	118-713	Pivot Interface Assembly
7	IA19	Pull Pin [Single]

Squiggles+ Spare Parts - Tray

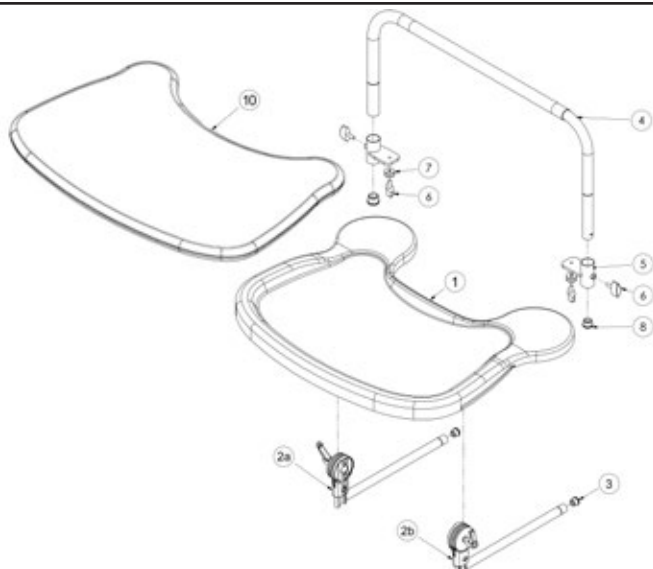

Pos.	Leckey Sales Code	Description
1	118-619	Tray Assembly
2a	118-710	Tray Depth Adjustment Assembly LH
2b	118-618	Tray Depth Adjustment Assembly RH
3	SF51	Round Tube Insert [Single]
4	117-769	Tray Grab Rail
5	118-646	Tray Grab Rail Height Adjuster [Single]
6	SF48	Locking Key M6 [Single]
7	SI09	M6 Washer [Single]
8	SF52	3/4" Round Tube Insert [Single]
9	SF53	7/8" Round Tube Insert [Single]
10	120-270	Padded Tray Insert

Squiggles+ Spare Parts - Tray Support

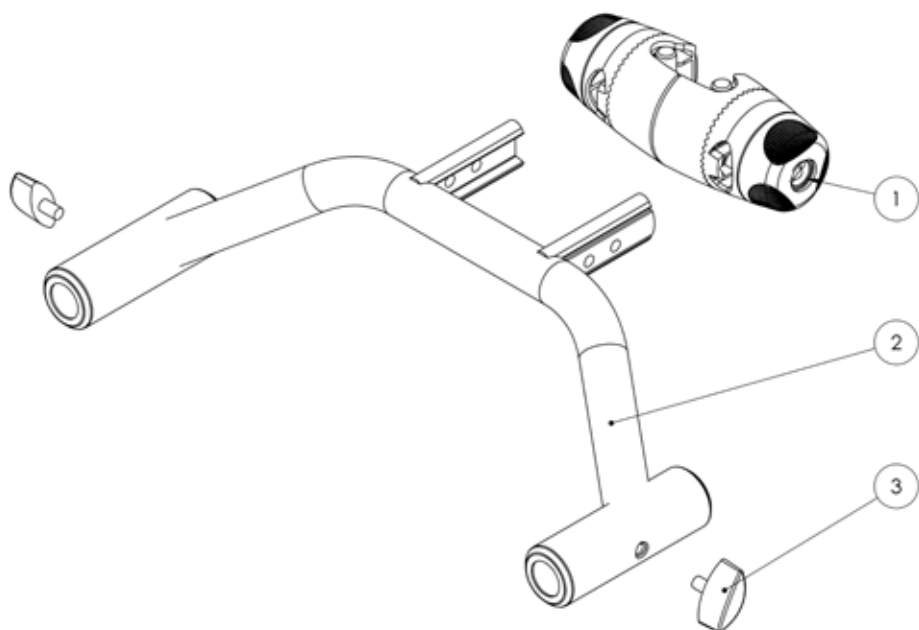

Pos.	Leckey Sales Code	Description
1	118-660	Orb Assembly
2	184-689	Tray Horn Assembly
3	SF48	Locking Key (M6x16) [Single]



Leckey
19c Ballinderry Road
Lisburn BT28 2SA
Northern Ireland

{+44} 28 9260 0750
hello@leckey.com

LS827-02

leckey.com

