



Squiggles TT

Spare Parts Catalogue




Supporting
Children

Squiggles TT Spare Parts - Covers




Pos.	Leckey Sales Code	Description	Colour Options			
			Blue	Purple	Red	Red Vinyl
		[Replace XX with colour]				
4	184-1820-XX	Knee Support [Pair]	-02	-04	-09	-09VIN
4	184-2810-XX	Knee Support Size 2 [Pair]				
5	188-1851-05	Lateral - Black [Pair]	-	-	-	-
6	188-1850-XX	Horizontal Support	-02	-04	-09	-09VIN
7	188-1852-XX	Upper Wrap Support	-02	-04	-09	-09VIN
8	188-1853-XX	Lower Wrap Support	-02	-04	-09	-09VIN
9	188-1854-05	Padded Tray Cover - Black	-	-	-	-
10	184-1838-XX	Spine Cap Cover	-02	-04	-09	-09VIN
11	188-703	Upholstery Bolt Pack [2 Pairs]	-	-	-	-

Squiggles TT Spare Parts - **Handlebar**

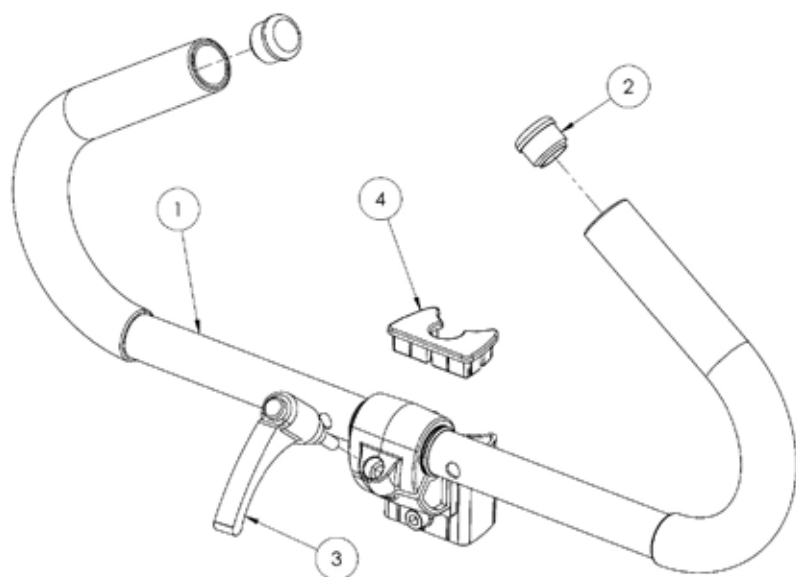


Pos.	Leckey Sales Code	Description
1	184-621	Push Handle Assembly
2	SF53	Round Tube Insert [Single]
3	SF95	Adjustable Handle
4	118-154	Extrusion Outer Cap

Squiggles TT Spare Parts - **Outer Spine**

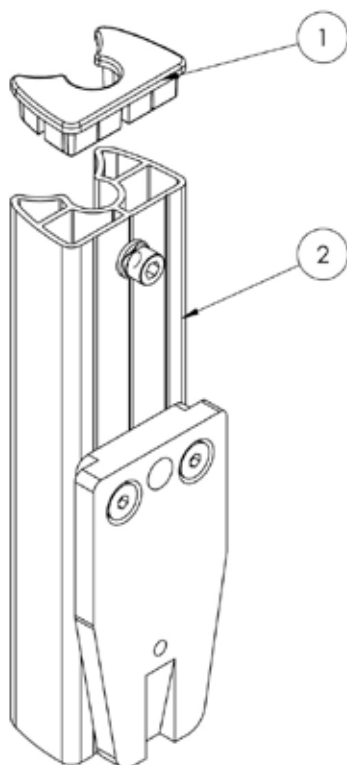


Pos.	Leckey Sales Code	Description
1	118-154	Extrusion Outer Cap
2	184-610	Outer Spine Assembly

Squiggles TT Spare Parts - Inner Spine

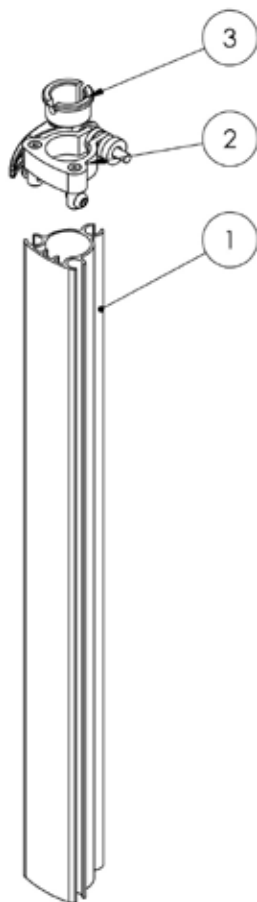


Pos.	Leckey Sales Code	Description
1	184-611	Inner Spine Assembly
2	184-712	Head Support Clamp Assembly SQ+
3	118-166	Head Support Collar

Squiggles TT Spare Parts - Head Support Spine

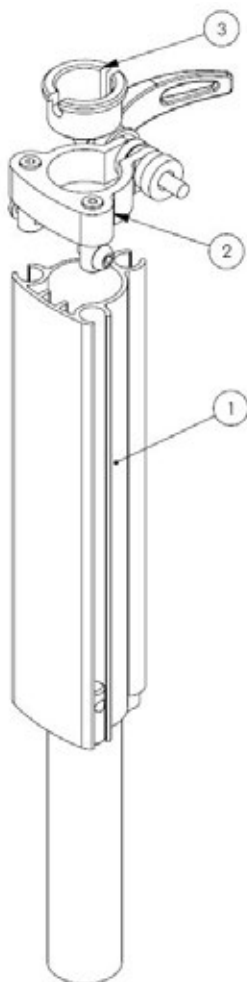


Pos.	Leckey Sales Code	Description
1	188-603	Horizontal Support Spine Assembly
2	184-712	Head Support Clamp Assembly SQ+
3	118-166	Head Support Collar

Squiggles TT Spare Parts - **Horizontal Support and Orb**

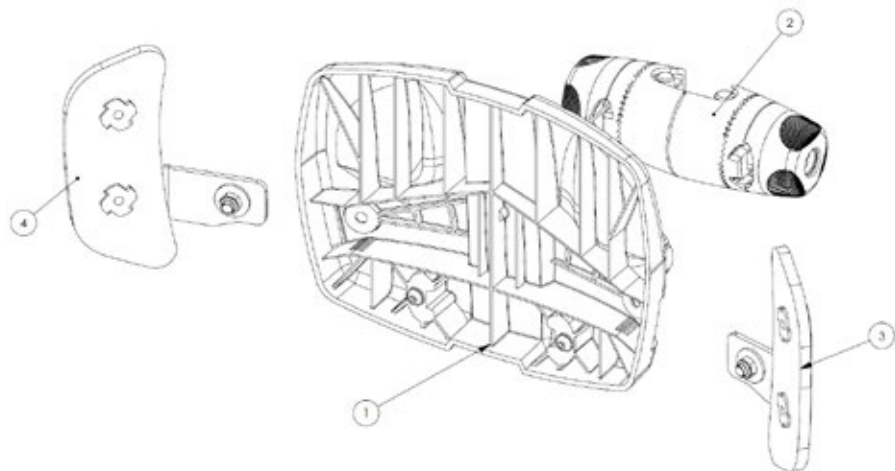


Pos.	Leckey Sales Code	Description
1	188-604	Horizontal Support Assembly
2	118-680	Orb Assembly
3	188-712	Lateral Assembly - LH [Single]
4	188-713	Lateral Assembly - RH [Single]

Squiggles TT Spare Parts - Upper Support and Orb

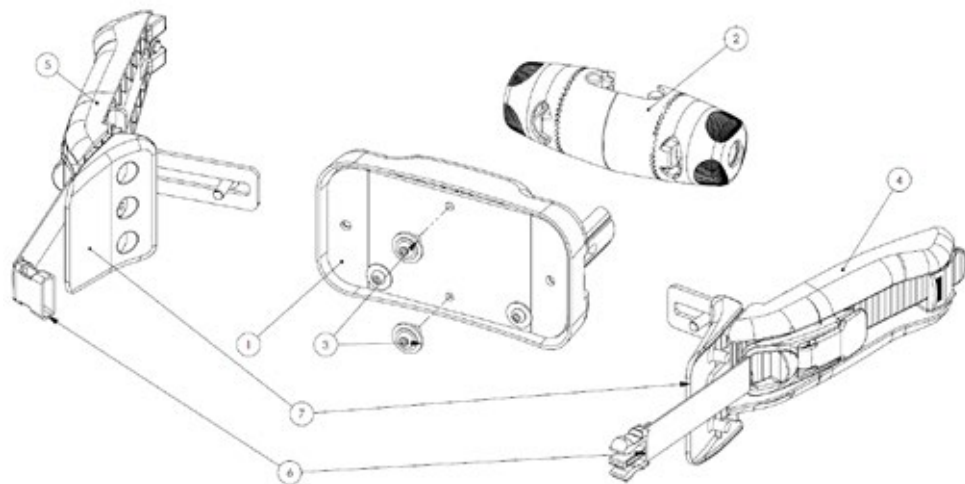


Pos.	Leckey Sales Code	Description
1	188-615	Upper Support Assembly
2	118-680	Orb Assembly
3	188-703	Upholstery Bolt Pack [2 pairs]
4	188-607	Lateral Assembly - LH [Single]
5	188-606	Lateral Assembly - RH [Single]
6	188-740	Lock and Buckle [Pair]
7	188-049	SQTT Side Lateral [Pair]

Squiggles TT Spare Parts - Lower Support and Orb

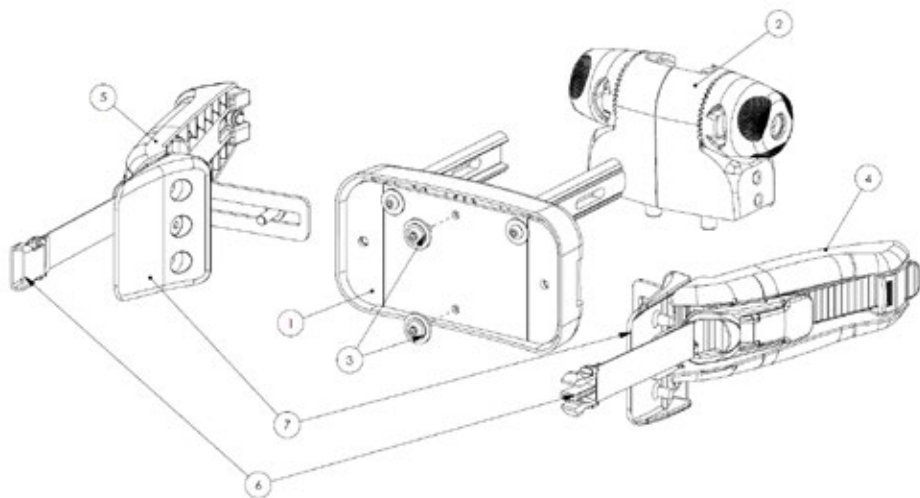


Pos.	Leckey Sales Code	Description
1	188-616	Lower Support Assembly
2	184-660	Orb Assembly
3	188-703	Upholstery Bolt Pack [2 Pairs]
4	188-607	Lateral Assembly - LH [Single]
5	188-606	Lateral Assembly - RH [Single]
6	188-740	Lock and Buckle [Pair]
7	188-049	SQTT Side Lateral [Pair]

Squiggles TT Spare Parts - Knee Support

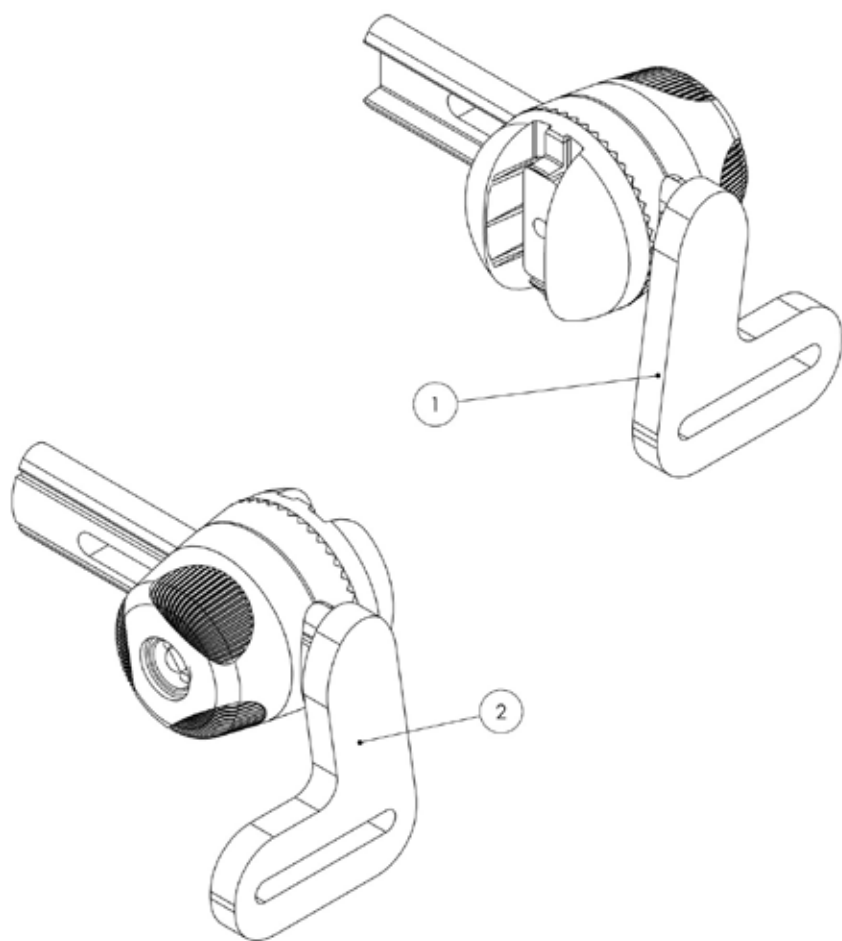


Pos.	Leckey Sales Code	Description
1	184-613-LHS	Knee Block Assembly Left
2	184-613-RHS	Knee Block Assembly Right

Squiggles TT Spare Parts - Footrest

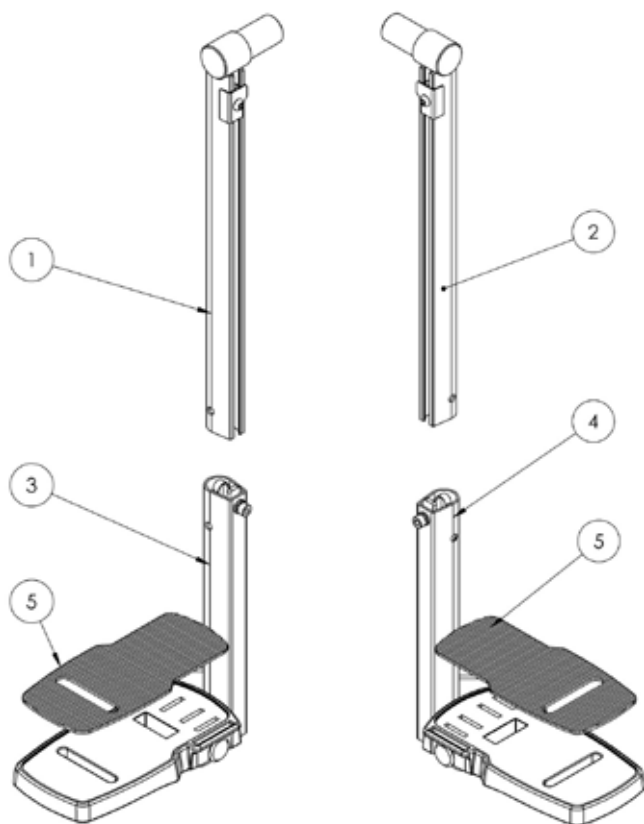


Pos.	Leckey Sales Code	Description
1	184-692	Calf Inner Support Assembly RH
2	184-693	Calf Inner Support Assembly LH
3	184-654	Calf Outer Support Assembly RH
4	184-655	Calf Outer Support Assembly LH
5	117-156	Footplate Rubber Matting (One Piece Only)

Squiggles TT Spare Parts - **Pivot Chassis**

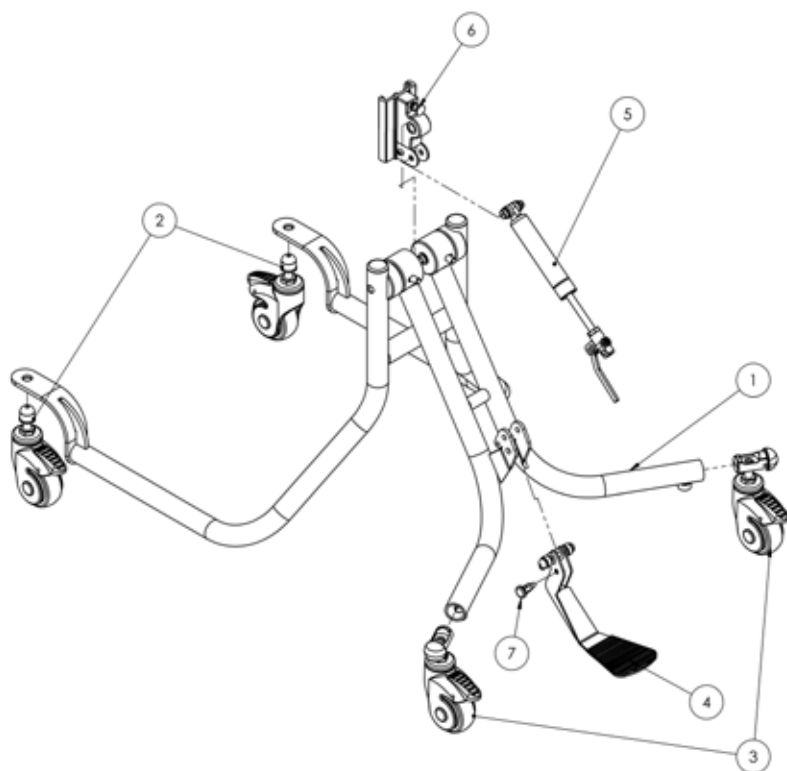


Pos.	Leckey Sales Code	Description
1	184-764	Pivot Chassis
2	184-715	Castor Assembly - Front [Pair]
3	184-716	Castor Assembly - Rear [Pair]
4	184-625	Pedal Assembly
5	184-717	Gas Spring Assembly
6	118-713	Pivot Interface Assembly
7	1A19	Pull Pin [Single]

Squiggles TT Spare Parts - **Angle Limit Adjuster**

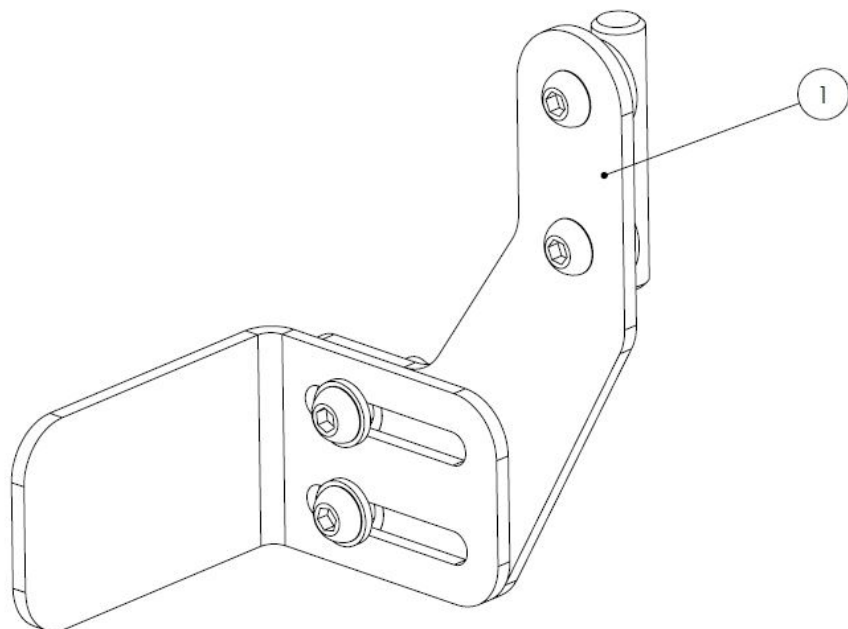


Pos.	Leckey Sales Code	Description
1	188-625	Angle Adjustment Limiter

Squiggles TT Spare Parts - Perturbation Rocker

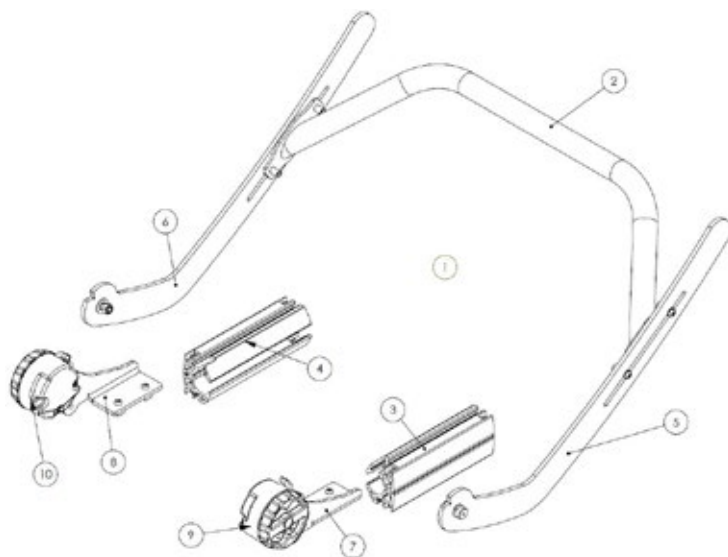


Pos.	Leckey Sales Code	Description
1	188-630	Full Perturbation Rocker Assembly
2	188-645	Crossbar
3	188-735	Chassis Clamp LH
4	188-745	Chassis Clamp RH
5	188-741	Perturbation Leg LH
6	188-742	Perturbation Leg RH
7	188-737	Perturbation Mechanism LH with Cover
8	188-738	Perturbation Mechanism RH with Cover
9	188-756	Mechanism Cover LH
10	188-757	Mechanism Cover RH

Squiggles TT Spare Parts - Tray

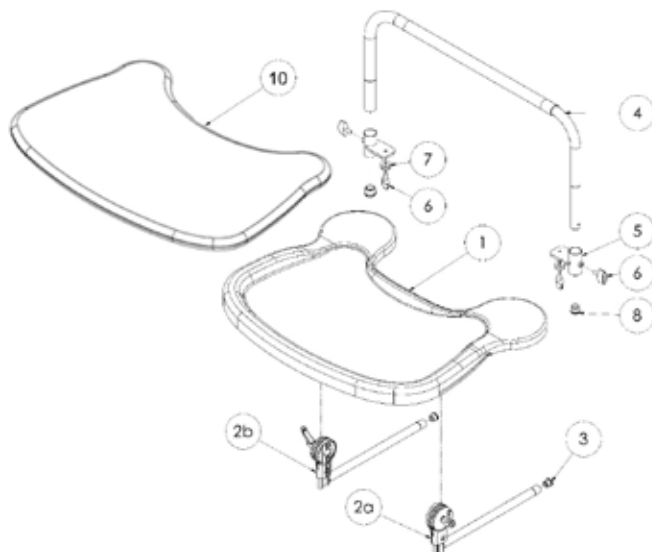

Pos.	Leckey Sales Code	Description
1	118-619	Tray Assembly
2a	118-710	Tray Depth Adjustment Assembly LH
2b	118-618	Tray Depth Adjustment Assembly RH
3	SF51	Round Tube Insert [Single]
4	117-769	Tray Grab Rail
5	118-646	Tray Grab Rail Height Adjuster [Single]
6	SF48	Locking Key M6 [Single]
7	SI09	M6 Washer [Single]
8	SF52	3/4" Round Tube Insert [Single]
9	SF53	7/8" Round Tube Insert [Single]
10	188-1854-05	Squiggles TT Padded Tray Cover - Black

Squiggles TT Spare Parts - Tray Support

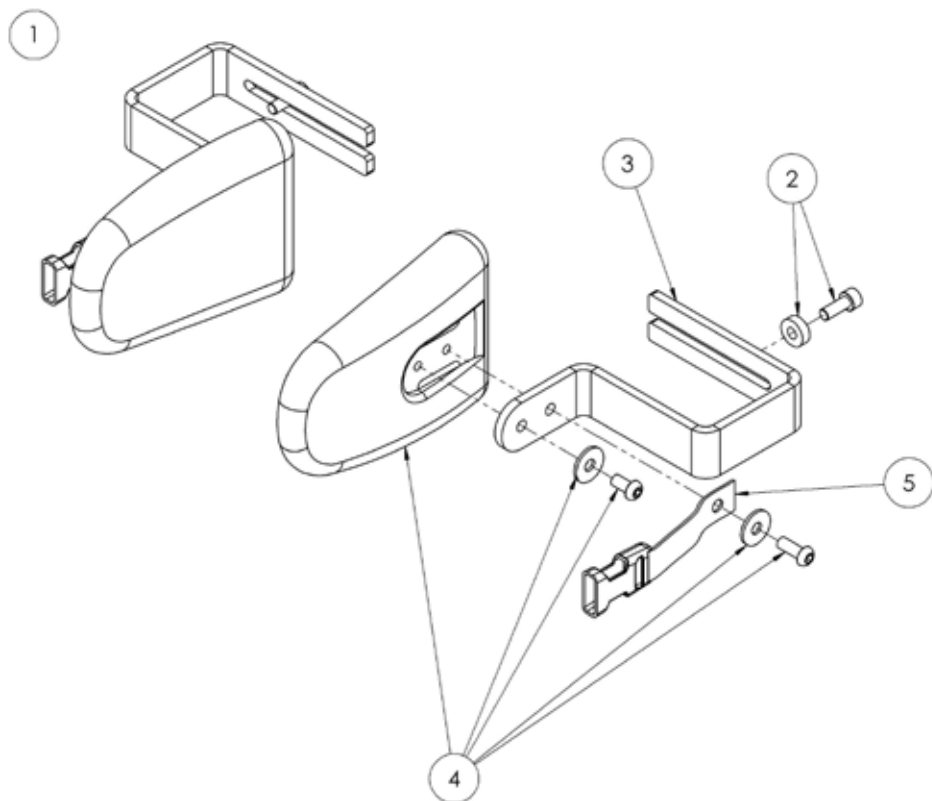

Pos.	Leckey Sales Code	Description
1	118-660	Orb Assembly
2	184-689	Tray Horn Assembly
3	SF48	Locking Key (M6x16) [Single]



Leckey
19c Ballinderry Road
Lisburn BT28 2SA
Northern Ireland

{+44} 28 9260 0750
hello@leckey.com

LS835-01

leckey.com

